

MOOG®

**CURLIN
INFUSION**

User Manual

for use with the

6000 CMS™ and 6000 CMS™ IOD™

Ambulatory Infusion System



Moog Medical Devices Group

tf 800.970.2337

ph 801.264.1001

fx 801.264.1051

www.moogmedical.com

Important Safety Note

- ▶ When using a Curlin Infusion Pump in a life sustaining situation, use AC Adapter, and provide an extra set of "C" cell batteries in addition to the batteries that are installed in the pump.
- ▶ Check pump between patient use per procedure (see maintenance chapter).
- ▶ Prior to Programming Load Dose, refer to Important Information in the appropriate Therapy chapters.



User Manual

for use with the

6000 CMS™ and 6000 CMS™ IOD™
Ambulatory Infusion Systems



Manufactured by
ZEVEX, Inc.
4314 ZEVEX Park Lane
Salt Lake City, UT 84123 USA
Phone 801.264.1001
Fax 800.778.3135
www.moogmedical.com

For 24-hour Clinical or Technical Support call toll free:
1-800-970-2337

Covered by U.S. Patents 5,575,631 5,683,233 5,791,881 5,924,852 6,164,921 6,371,732 B1
Des. 408,911 Des. 437,547 S. Other U.S. and Foreign Patents Pending
Copyright © 2025 Moog, Inc. All Rights Reserved

Revision L
Reorder Number 360-9009

TABLE OF CONTENTS

INTRODUCTION.....	1	Keypad Function.....	33
Delivery Modes	1	Description of the Info-On-Demand (IOD) Feature	34
Indications for Use	2	Info-On-Demand (IOD) Key Functions	35
Special Features of the <i>6000 Series</i> Pump	2	LED Light Indicators	37
Regulatory Agency Approvals.....	4	LED Light Indicators	38
Warnings, Cautions, Notices	4	Audio Indicator.....	38
List of Abbreviations	10	Display Screen.....	38
Pump Illustrations and Features.....	12	BASICS OF	
Disposable Administration Set Illustrations and Features	14	PROGRAMMING	40
Preparing Medication for Infusion/Changing IV Bags	15	Types of Display Screens	40
Opening and Closing the Pump Door	17	Sample of an Action Field Screen.....	40
Choosing and Installing the Moog Curlin Infusion Administration Set	18	Sample of a Selection List Field.....	40
Gravity Priming Set and Filter	18	Sample of a Data Entry Screen.....	41
Steps in Installing Administration Set into Moog Curlin Infusion Pumps	19	Sample of a Run Screen	41
Up Stream Occlusion Alarm Feature	20	Sample of User Notification Screen	41
Installing the Batteries and Using External Power	21	Using HELP	42
Battery Replacement, Alternate Battery Door	22	Starting the Pump	42
External Power Sources	24	Welcome Screen	42
AC Adaptor/Charger	30	Power Status Screen	43
Bolus Cord Use	31	Preventive Maintenance Screen.....	43
LEARNING PUMP		Select Program or Bio-Med Setup Screen	43
FEATURES	32	Access Code	44
Figure 2.1 Key Pad	32	Clinician Access Code Screen	44
		BioMed Setup	44
		BioMed Setup Menu	44
		Enable Therapy Menu.....	46
		Patient History Log	47
		History Event Log	48
		Clearing the History Log	49
		Clearing Patient Information	49
		Changing Date and Time.....	50
		Change Date and Time Screen.....	50
		Program.....	51

TABLE OF CONTENTS

<p>Program/BioMed Setup Menu..... 51</p> <p>Resume/Repeat Rx/New Program Menu 51</p> <p>Resuming a Therapy.....52</p> <p>Repeating a Pre-Programmed Therapy52</p> <p>How to go to “New Program”53</p> <p>Selecting a Therapy53</p> <p style="padding-left: 20px;">Select Therapy Screen 53</p> <p>Entering or Changing Data Fields.....53</p> <p>PROGRAMMING A 6000 CMS PUMP WITH PROTOCOL LIBRARY SAFETY SYSTEM (PLSS) VERSION 2 PROGRAM54</p> <p style="padding-left: 20px;">Programming a Pump with a Protocol Library.....55</p> <p style="padding-left: 40px;">Resume/Repeat Rx/New Program Menu 55</p> <p style="padding-left: 20px;">Programming a New Library Therapy55</p> <p style="padding-left: 20px;">Resuming a Library Therapy57</p> <p style="padding-left: 20px;">Repeating a Pre-Programmed Library Therapy57</p> <p>CONTINUOUS THERAPY59</p> <p style="padding-left: 20px;">Prescription Menus59</p> <p style="padding-left: 40px;">Continuous Pre-Prescription Menu Screen..... 59</p> <p style="padding-left: 40px;">Continuous MedLMTS™ Limit Screen..... 61</p> <p style="padding-left: 40px;">Continuous Prescription Menu..... 61</p> <p style="padding-left: 40px;">Run Options Screen 63</p> <p style="padding-left: 20px;">Prime63</p> <p style="padding-left: 40px;">Prime Direction Screen..... 63</p> <p style="padding-left: 20px;">Options64</p> <p style="padding-left: 40px;">Continuous Options Menu 65</p> <p style="padding-left: 40px;">Hourly Totals Screen..... 65</p>	<p style="padding-left: 20px;">Clear Hourly Totals Screen66</p> <p style="padding-left: 20px;">Continuous MedLMTS™ Menu66</p> <p style="padding-left: 20px;">Air-In-Line Sensor Off Message Screen.....67</p> <p style="padding-left: 20px;">Air-In-Line Sensor Set at 2mL Alert Screen67</p> <p>Reviewing or Changing the Program 68</p> <p>Starting the Infusion 68</p> <p style="padding-left: 20px;">Continuous Run Screen.....68</p> <p style="padding-left: 20px;">Continuous Information Screens69</p> <p>Interrupting an Infusion69</p> <p>Resuming the Current Container 70</p> <p>Hanging a New Container..... 70</p> <p>Entering a New Program 71</p> <p>Infusion Complete 71</p> <p style="padding-left: 20px;">Continuous Infusion Complete71</p> <p>Stopping the Infusion 72</p> <p>PCA THERAPY73</p> <p style="padding-left: 20px;">Prescription Menus73</p> <p style="padding-left: 40px;">PCA Pre-Prescription Menu Screen.....73</p> <p style="padding-left: 40px;">Administration Routes.....74</p> <p style="padding-left: 40px;">PCA MedLMTS™ Limits Screen.....75</p> <p style="padding-left: 40px;">PCA Prescription Menu.....76</p> <p style="padding-left: 40px;">Run Options Screen77</p> <p style="padding-left: 20px;">Prime77</p> <p style="padding-left: 40px;">Prime Direction Screen.....78</p> <p style="padding-left: 20px;">Options79</p> <p style="padding-left: 40px;">PCA Options Menu79</p> <p style="padding-left: 40px;">Clinician Dose Menu.....80</p> <p style="padding-left: 40px;">Clinician Dose Run Screen80</p> <p style="padding-left: 40px;">Shift Totals Screens81</p> <p style="padding-left: 40px;">Hourly Totals Screen.....81</p> <p style="padding-left: 40px;">Clear Hourly Totals Screen82</p> <p style="padding-left: 40px;">PCA MedLMTS™ Screen.....82</p> <p style="padding-left: 40px;">Air-In-Line Sensor Off Message Screen.....83</p> <p style="padding-left: 40px;">Air-In-Line Sensor at 2mL Alert Screen.....84</p>
---	---

TABLE OF CONTENTS

Reviewing or Changing the Program	84	Resuming the Current Container	101
Starting the Infusion	85	Hanging a New Container.....	101
PCA Run Screen.....	85	Entering a New Program	102
PCA Information Screens	86	Infusion Complete	102
Interrupting an Infusion	86	TPN Infusion Complete.....	102
Resuming the Current Container	86	Stopping the Infusion	103
Run Options Screen	86	Pause Menu Screen	103
Hanging a New Container.....	87	INTERMITTENT	
Entering a New Program	88	THERAPY	104
Infusion Complete	89	Prescription Menus	104
PCA Infusion Complete	89	Intermittent Pre-Prescription	
Stopping the Infusion	89	Menu Screen.....	104
Pause Menu Screen	89	Run Options Screen	105
TPN THERAPY WITH		Intermittent Prescription Menu	106
AUTOMATIC RAMPING	91	Run Options Screen	108
Prescription Menus	91	Prime	108
TPN Pre-Prescription Menu		Prime Direction Screen	108
Screen.....	91	Options.....	109
Run Options Screen	92	Intermittent Options Menu.....	110
TPN Prescription Screen	93	Hourly Totals Screen.....	110
Run Options Screen	94	Clear Hourly Totals Screen	111
Prime	94	Air-In-Line Sensor Off Message	
Prime Direction Screen	95	Screen.....	111
Options	96	Air-In-Line Sensor Set at 2mL	
TPN Options Menu.....	96	Alert Screen	112
Hourly Totals Screen.....	97	Reviewing or Changing the	
Clear Hourly Totals Screen	97	Program	113
Air-In-Line Sensor Off Message		Starting the Infusion	113
Screen.....	98	Intermittent Run Screen	113
Air-In-Line Sensor Set at 2mL		Intermittent Information Screens.....	114
Alert Screen	99	Interrupting an Infusion	114
Reviewing or Changing the		Resuming the Current Container	114
Program	99	Hanging a New Container.....	114
Starting the Infusion	99	Intermittent Dosing Schedule.....	115
TPN Run Screen	100	Entering a New Program	116
TPN Information Screens.....	100	Timing Rules When Interrupting	
Interrupting an Infusion	101	Intermittent Therapies	116
		Infusion Complete	117

TABLE OF CONTENTS

<p>Intermittent Infusion Complete..... 117</p> <p>Stopping the Infusion 118</p> <p style="padding-left: 20px;">Pause Menu Screen 118</p> <p>VARIABLE THERAPY 119</p> <p style="padding-left: 20px;">Prescription Menus 119</p> <p style="padding-left: 40px;">Variable Pre-Prescription Menu</p> <p style="padding-left: 60px;">Screen..... 119</p> <p style="padding-left: 40px;">Run Options Screen 120</p> <p style="padding-left: 40px;">Variable Prescription Menu 1..... 121</p> <p style="padding-left: 40px;">Variable Prescription Menu 2..... 122</p> <p style="padding-left: 40px;">Variable Prescription Menu 3..... 122</p> <p style="padding-left: 40px;">Variable Summary Screen 123</p> <p style="padding-left: 40px;">Run Options Screen 123</p> <p style="padding-left: 20px;">Prime 124</p> <p style="padding-left: 40px;">Prime Direction Screen..... 124</p> <p style="padding-left: 20px;">Options 125</p> <p style="padding-left: 40px;">Variable Options Menu 125</p> <p style="padding-left: 40px;">Hourly Totals Screen..... 126</p> <p style="padding-left: 40px;">Clear Hourly Totals Screen 126</p> <p style="padding-left: 40px;">Air-In-Line Sensor Off Message</p> <p style="padding-left: 60px;">Screen..... 127</p> <p style="padding-left: 40px;">Air-In-Line Sensor Set at 2mL</p> <p style="padding-left: 60px;">Alert Screen 127</p> <p style="padding-left: 20px;">Reviewing or Changing the</p> <p style="padding-left: 20px;">Program 128</p> <p style="padding-left: 20px;">Starting the Infusion 128</p> <p style="padding-left: 40px;">Variable Run Screen 1..... 128</p> <p style="padding-left: 40px;">Variable Run Screen 2..... 129</p> <p style="padding-left: 40px;">Variable Information Screens 129</p> <p style="padding-left: 20px;">Interrupting an Infusion 130</p> <p style="padding-left: 20px;">Resuming the Current Container 130</p> <p style="padding-left: 20px;">Hanging a New Container..... 130</p> <p style="padding-left: 20px;">Entering a New Program 131</p> <p style="padding-left: 20px;">Infusion Complete 131</p> <p style="padding-left: 40px;">Variable Infusion Complete 131</p> <p style="padding-left: 20px;">Stopping the Infusion 132</p> <p style="padding-left: 40px;">Pause Menu Screen 132</p> <p>TROUBLESHOOTING 133</p>	<p>Alerts, Alarms, and Error Codes 133</p> <p>ACCESSORIES..... 154</p> <p style="padding-left: 20px;">Soft Carry Packs..... 154</p> <p style="padding-left: 20px;">Detachable Pole Clamps..... 155</p> <p style="padding-left: 20px;">Remote Bolus Cord (340-2005) 155</p> <p style="padding-left: 20px;">Data Interface Cable (340-2011)..... 156</p> <p>CLEANING,</p> <p>ENVIRONMENTAL</p> <p>CONDITIONS, AND</p> <p>MAINTENANCE 157</p> <p style="padding-left: 20px;">Cleaning..... 157</p> <p style="padding-left: 20px;">Transport and Storage 158</p> <p style="padding-left: 20px;">Operating Environment 158</p> <p style="padding-left: 20px;">Maintenance..... 158</p> <p style="padding-left: 20px;">Limited Warranty 158</p> <p style="padding-left: 20px;">Procedure for Checking Pumps</p> <p style="padding-left: 20px;">Between Patients 160</p> <p>TECHNICAL</p> <p>SPECIFICATIONS..... 164</p> <p>CLINICIAN</p> <p>INFORMATION AND</p> <p>ACCESS CODE..... 175</p> <p style="padding-left: 20px;">Clinician Access Code..... 175</p> <p style="padding-left: 20px;">Lock Setting Table 176</p>
--	--

TABLE OF FIGURES

FIGURE 1.1 MOOG CURLIN INFUSION AMBULATORY PUMP—FRONT, TOP AND LEFT VIEW	12	FIGURE 10.3 DETACHABLE POLE CLAMP ..	155
FIGURE 1.2 MOOG CURLIN INFUSION AMBULATORY PUMP—BACK, BOTTOM, AND RIGHT VIEW	13	FIGURE 10.4 REMOTE BOLUS CORD	155
FIGURE 1.3 SAMPLES OF MOOG CURLIN INFUSION ADMINISTRATION SETS	14	FIGURE 10.5 DATA INTERFACE CABLE	156
FIGURE 1.4 SPIKING THE FLUID CONTAINER, GRAVITY PRIMING THE SET AND USING THE SLIDE CLAMP TO CLOSE THE TUBING	15		
FIGURE 1.5 MOOG CURLIN INFUSION ADMINISTRATION SET INTEGRAL FLOW-STOP WITH BREAKAWAY TAB.....	16		
FIGURE 1.6 INTENTIONALLY OPENING THE INTEGRAL FLOW-STOP	16		
FIGURE 1.7 OPENING THE PUMP DOOR	17		
FIGURE 1.8 CLOSING THE PUMP DOOR	17		
FIGURE 1.9 PROPER INSTALLATION OF THE MOOG CURLIN INFUSION ADMINISTRATION SET INTO THE PUMP	19		
FIGURE 1.10 TOP CUTAWAY VIEW OF THE MOOG CURLIN INFUSION PUMP WITH MOOG CURLIN INFUSION ADMINISTRATION SET IN PROPER INSTALLATION POSITION	20		
FIGURE 1.11 BATTERY REPLACEMENT, ALTERNATE BATTERY DOOR.....	22		
FIGURE 1.12 UNIVERSAL AC ADAPTOR	30		
FIGURE 1.13 ATTACHING AND REMOVING THE REMOTE BOLUS CORD	31		
FIGURE 2.1 KEY PAD	32		
FIGURE 2.2 IOD KEYPAD WITH INFO-ON-DEMAND FEATURE	34		
FIGURE 2.3 SAMPLE OF DISPLAY SCREEN....	39		
FIGURE 3.1 HISTORY EVENT LOG	48		
FIGURE 5.1 PCA ADMINISTRATION ROUTE DEFAULT SETTINGS	74		

Chapter

1

Introduction

*The **Moog Curlin Infusion** pumps have been designed with the user in mind and have the latest in friendly, simple-to-learn technology to allow fast and easy access to the features of these pumps. With a little introduction and training, an operator will quickly be able to program and use these pumps to deliver the therapies prescribed.*

Note: Throughout this manual the text will refer to the **Moog Curlin Infusion** pump. All the features described here apply equally to the **6000 CMS™** pumps, unless specifically otherwise stated.

Let's begin by introducing you to the **Moog Curlin Infusion** pump, an easy-to-use, ambulatory or pole mounted infusion device designed to meet the needs of the hospital or alternate site patient. It operates accurately in any position, has a low rate resolution of 2 microliters, and its small, compact, lightweight design allows mobility for ambulatory patients.

The **Moog Curlin Infusion** pump can be carried in any of the four convenient carryall soft-packs, pole mounted, placed into the locked "safety shells," or set on a suitable surface for use.

Delivery Modes

The pump features five therapy delivery modes:

1. **Continuous Infusion**—Designed to allow a constant, programmed rate of infusion.
2. **PCA or Patient Controlled Analgesia**—Designed for therapies that require a continuous rate of infusion, patient-controlled demand boluses, or both.
3. **TPN with Automatic Ramping**—Designed to allow a level rate of infusion of parenteral nutritional products with the option of tapering at the beginning, end, or both beginning and end of the infusion. This mode also has an early ramp-down feature.
4. **Intermittent Delivery**—Designed to deliver programmed intervals and rates of specified amounts of infusates and to optionally deliver small amounts of the infusate between doses to keep the patient's access site patent.
5. **Variable Program**—Designed to allow varying amounts, rates, and times of delivery of infusions up to twenty-four specified programs.

Indications for Use

The Moog Curlin infusion pump can be used for intravenous, intra-arterial, epidural, subcutaneous, or enteral infusions. Any therapy should be overseen by a physician or a certified, licensed healthcare practitioner. The infusion pump can provide therapy to patients/users who are ambulatory and needed an alternative to receiving their medication in a restricted manner (hospital or clinic). Typically, these patients are capable of being given instruction for performing basic tasks in operation the pump and maintaining medical care for themselves, or are assisted in these tasks by having a caregiver closely by, or living with them. All patients and caregivers must be instructed in the use of the pump by a qualified clinician and demonstrate an adequate level of proficiency in the use of the pump.

The pump is small, lightweight and is designed to have exceptional ease of use so that after learning the pump features and basics of programming, the pump can be operated and can be customized to the specific needs of each individual patient without the need to constantly refer to the User's Manual.

Special Features of the 6000 Series Pump

- The Info-On-Demand feature safely provides additional information to the user when the pump is in the "RUN" mode.
- Shift Totals that can be greater than 24 hours.
- Hourly Totals in all therapies
- Customized Maximum Basal Rate Limit.
- Customized Maximum Bolus and Dose Limits.
- Customized Lock Level setting.
- Large screen with Tallman Lettering.
- Larger font, single information display.
- MedLMTS™ features.
- PCA dose and MedLMTS™ schedules that can be maintained on Repeat.
- Pump will not permit the user to enter any programming value that is outside of its predetermined default range, or further restricted through MedLMTS™.
- Automatic "quick-repeat" function
- Small, accurate, ambulatory, volumetric infusion pump that provides reliable and safe delivery of infusion therapies.
- Cost effective, safe and easy-to-load disposable **Moog Curlin Infusion** administration sets featuring a unique, spring-activated, self-clamping, set-based Integral Flow-Stop device that automatically clamps the tubing when the door of the pump is opened and prevents inadvertent fluid flow.
- User-friendly, easy-to-teach programming that shortens staff inservice time and patient teaching time.
- Help screens and display messages.

¹ Moog cannot guarantee the $\pm 5\%$ accuracy claim when using syringes

- Three tamper-resistant lock settings to assist in maintaining patient compliance and safety.
- Patient and therapy specific programming features via the **Moog Curlin Infusion** BIOMED SETUP and Options Menus (when not limited by the institution).
- Ability, at the institution level, to configure the pump for specific administration route or units of measure:
 - Units (Factory, mL, mg, mcg, mL fixed, mg fixed or mcg fixed);
 - Route (Intravenous, Epidural, Subcutaneous, Intravenous fixed, Epidural fixed or Subcutaneous fixed).
- Retention of programmed infusion settings until cleared by the clinician, thus eliminating the need to reprogram before each use.
- Mandatory requirement of two independent key press actions before a running pump can be turned off, thus minimizing the accidental interruption of a therapy in progress.
- Powered by two readily available, cost-effective C-cell alkaline batteries.
- Attractive yet robust design that is impact resistant and water resistant.
- Audio alarms that can be adjusted from 1 (very quiet) to 9 (loudest) to meet specific patient and clinical setting needs but that cannot be totally disabled.
- Four sizes of convenient carry packs to assist the ambulatory patient in maintaining independence of lifestyle. Backpack will accommodate pump, AC Adaptor, and up to four-liter IV bag.
- Retained memory features that allow the pump to easily resume a therapy exactly from where it left off when it is interrupted before completion or to repeat a therapy using the same programmed prescription.
- Compact, lockable safety shells that provide additional security when needed and that are designed to be free standing, pole mounted, or placed in the convenient, soft, carry case.
- Backlit illumination for display screen. Backlight remains on continuously if AC Adaptor is in use.
- Pump will not permit the user to enter any programming value that is outside of its predetermined range.
- Integral Up as well as Down Occlusion alarms to notify user if an occlusion is present on either side of the pumping chamber.
- Integral High Upstream pressure alarm to notify user of excessive upstream pressure.
- Integral Air-In-Line alarm to notify user of presence of air in the tubing.
- Adjustable Down Occlusion settings of “High” and “Low” to customize the pump’s down pressure requirements.
- Patient side (downstream) “Line Pressure” displayed.

Regulatory Agency Approvals



INFUSION PUMP
WITH RESPECT TO ELECTRIC SHOCK, FIRE
AND MECHANICAL HAZARDS ONLY
IN ACCORDANCE WITH UL60601-1/CAN/CSA
C22.2 NO. 601.170TK

- Underwriters Laboratories has evaluated the Moog Curlin 6000 Infusion Pump with AC Adapters to EN 60601-1 and CSA C22.2 No. 601-1 for General Standards of Safety for Medical Equipment and found the Moog Curlin Infusion Pump to be compliant.
- Underwriters Laboratories has evaluated the Moog Curlin Infusion Pumps with accessory equipment's Battery Pack 360-2030 and the Curlin 6000 AC Adapters to EN 60601-2-24 Particular requirements for safety of infusion pumps and controllers and they were found to be compliant.
- Underwriters Laboratories has evaluated the Moog Curlin 6000 Infusion Pumps and the AC Adapters for Electromagnetic Compatibility, to regulations EN 60601-1 and EN 60601-2-24 and they were found to be compliant.

IMPORTANT INFORMATION

The indicator shown at the left appears throughout this manual to emphasize important information in the operation of the Moog Curlin Infusion pump. Please read these sections carefully.

Warnings, Cautions, Notices

EXPLANATION FOR SYMBOLS on the **Moog Curlin Infusion** Pump with the Classified UL 60601-1 and CSA C22.2 No. 601-1 Label.

The meanings for the symbols on the pump are as follows:




Attention, consult ACCOMPANYING DOCUMENTS.



Type CF Equipment

IPX1 Protected against dripping water.

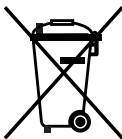

Separate collection of electrical and electronic equipment and batteries/accumulators.
Please do not dispose of this pump/accumulator in the regular bin.



INFUSION PUMP
WITH RESPECT TO ELECTRIC SHOCK, FIRE
AND MECHANICAL HAZARDS ONLY
IN ACCORDANCE WITH UL60601-1/CAN/CSA C22.2 NO.
601.170TK

INTERNALLY POWERED OR FOR USE WITH Curlin 6000 AC ADAPTERS.

DEFIBRILLATION PROOF TYPE CF APPLIED PARTS

EQUIPMENT PROTECTED AGAINST DRIPPING WATER, IPX1

EQUIPMENT NOT SUITABLE FOR USE IN THE PRESENCE OF A FLAMMABLE
ANESTHETIC MIXTURE WITH AIR OR WITH OXYGEN OR NITROUS OXIDE

MODE OF OPERATION CONTINUOUS

WARNINGS

- **In a life sustaining situation** it is recommended that your Curlin Infusion pump use the AC Adaptor and have a back-up pump available. Also provide an extra set of C-cell batteries in addition to the batteries that are installed in the pumps.
- User should be aware that the error in delivered drug volume may exceed $\pm 5\%$ when a syringe is used with the Curlin infusion pump.

A mechanical problem, known as “stiction,” may occur when the syringe is operated at low plunger speeds. The friction between the syringe plunger and the barrel causes the plunger to stick, then moves forward in a jerking motion, and the fluid will be delivered as a series of boluses. This effect will cause variations larger than 5% in the delivered volume. The fit between the syringe plunger and the barrel may vary from batch to batch and, consequently, the stiction effect may also vary.

Due to this potential error, the delivery of powerful drugs with short half lives, such as catecholamines is not recommended.

- Use only **Moog Curlin Infusion** administration sets in your **Moog Curlin Infusion** pump. Use of non-proprietary administration sets may alter accuracy in delivery of fluids and could result in over or under infusions, leading to possible patient injury or death.
- Visually inspect the pump, pumping chamber and administration set before use. Do not use any pump or administration set that appears to be damaged or tampered with or if there is any indication of improper function.

- Damage to the pump due to dropping or impact while the door is open can cause intermittent free flow conditions and loss of pump accuracy, but may not be visible through inspection. Volumetric testing is the only method to confirm proper function.

NOTE: Anti-siphon Valve (ASV) administration sets can be used to provide additional free-flow protection.

Do not subject the pump to dropping or hitting against a hard surface. If at any time the pump is dropped or hit, the pump must be checked for volumetric accuracy prior to reuse. The pump can only be brought back into service if the volumetric accuracy test passes per the user manual. If the pump fails volumetric accuracy it must be returned to Moog Medical Devices Group for evaluation.

Note: This procedure does not replace the recommended annual preventive maintenance. The pump should receive preventive maintenance by a qualified technician at least annually.

- Do not use this pump with a pressure cuff applied to the IV bag or medication reservoir bag, and do not unduly squeeze or compress the bag during a running infusion.
- Remove all air from the administration set and IV bag or reservoir before connecting it to a patient's access site.
- Do NOT prime the administration set while it is connected to the patient's access device. Doing so could result in overdosing the patient and could cause injury or death.
- Do not attempt to open the pump's housing. All service problems should be referred to an appropriate service technician.
- **Danger:** Risk of explosion if used in the presence of flammable anesthetic or explosive gasses.
- The CURLIN 6000 is NOT safe for use in an MRI environment. The CURLIN 6000 should not enter the MRI scanner room. The device presents a projectile hazard.

CAUTIONS

- U.S. federal law restricts this device to sale by or on the order of a physician or other licensed practitioner.
- Keep the pumping surfaces clean, dry, and free of fluid spillage at all times.
- This pump is fluid resistant and can withstand fluid spillage. It is **not**, however, designed for total submersion as moisture buildup within the case could cause damage to the operating components. Do not use the pump in the shower, sauna, or steam bath, and do not position the pump where it could accidentally be dropped into a container of fluid (e.g., basin, tub, or toilet). Avoid fluid contact with the power port and data port of the pump.
- Do not try to insert foreign objects into any of the pump connectors, as such objects may damage the pump.
- Use only **Moog Curlin Infusion** external power sources to power the **Moog Curlin Infusion** pump. Using other power sources may result in circuitry or microprocessor damage.
- When using the AC Adaptor, connect the plug only into a *grounded* AC outlet.
- Use only non-rigid, non-vented IV fluid containers unless an air vent adaptor is in place and the container is suspended from an IV pole.
- Do not unduly stretch the tubing of the administration set or leave the tubing in the pump for more than twenty-four hours when the pump is not running.
- Use **Moog Curlin Infusion** administration sets with appropriate air-in-line filters any time the Air-In-Line Sensor of the pump is disabled.
- If any signs or symptoms of infiltration or inflammation are noted at the infusion site, stop the infusion and report it to the appropriate healthcare provider.
- Always use the slide clamp on the administration set as an additional precaution to occlude the tubing before opening the door of the pump.
- Avoid leaving tubing and cords where infants, children, or those deemed at high risk for medical line entanglement, can become entangled. Be aware that if these items become wrapped around a patient's neck, there is an increased risk of strangulation or death. Caregivers of patients who are at risk for entanglement should discuss with their health care provider how to properly manage their lines and properly monitor patients based on their needs while in use.

NOTICES

- The **Moog Curlin Infusion** pump is not intended for the administration of blood or cellular blood products.
- Dispose of all used administration sets in accordance with all applicable regulatory and institutional policies.
- Teach users to check all tubing sites for proper aseptic connections and to check the administration sets for air leaks before and during the infusion.
- To minimize the potential hazard of air bubble formation or “out-gassing,” administer all medications at the proper temperatures, and remove medications from refrigerators as specified by the healthcare provider or pharmaceutical manufacturer.
- To avoid potential fire hazard, do not use the pump near flammable anesthetics, explosive gases or in an oxygen rich environment such as Hyperbaric Oxygen Therapy. Note: Use with supplementary oxygen and ventilation is allowed.
- Follow local governing ordinances and recycling plans regarding disposal or recycling of device components. When the C-cell batteries are to be discarded, do not incinerate them. Dispose of all used administration sets in accordance with all applicable regulatory and institutional policies and directions.
- Adhere to any warnings, precautions, or recommendations stated by drug manufacturers regarding the use of infusion pumps and disposable administration sets in the administration of their specific products.
- Administer all drugs selected for epidural administration in accordance with the indications included in the manufacturer's package insert accompanying the drugs.
- Note that the administration of medications into the epidural space is limited to delivery via specially designed indwelling catheters inserted by a qualified physician. Any patients receiving epidural infusions should be managed and monitored by medical professionals familiar with epidural administrations and their clinical management. **Moog** offers administration sets specifically designed for epidural infusions with 0.22 micron filters, no injection ports, and yellow striping to clearly identify the set for epidural use. Any administration set being used for epidural infusions should be clearly labeled as such.
- Clean the pump, remote bolus cord, AC adaptor, and battery pack with a soft, clean cloth dampened in any of the following:
 1. Warm soapy water (do not submerge)
 2. Isopropyl alcohol
 3. Household bleach, diluted 9:1 with water
 4. Commercial disinfectant
 5. When clean, dry pump and other items with a soft, clean, dry cloth.
- The soft carry packs are intended for single patient use. They may be cleaned in the gentle cycle of a washing machine using cold water, no chlorine bleach, and line dried. (They cannot be dried in a clothes dryer.)
- The bolus/data connector is to be used by the patient only for the connection of the bolus cord with remote switch. Other accessory equipment connected to this digital interface must be certified to the respective IEC/EN standards (i.e. IEC 950 for data processing equipment

and EN 60601-1 for medical equipment.) Furthermore, all configurations shall comply with the system standard EN 60601-1. Anyone who connects additional equipment to the signal input part or signal output part configures a medical system, and is therefore, responsible that the system complies with the requirements of the system standard EN 60601-1. If in doubt, consult the technical services department or your local **Moog** representative.

- The **Moog Curlin Infusion** pump has been tested and found to comply with the European Standard EN 60601-1-2:1993 and EN 60601-2-24:1994. These limits are designed to provide reasonable protection against harmful interference in a typical medical installation.
- This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to other devices in the vicinity. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to other devices, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one of more of the following measures:
 1. Reorient or relocate the receiving device.
 2. Increase the separation between the equipment.
 3. Connect the equipment into an outlet on a circuit different from that to which the other device(s) are connected.
 4. Consult the manufacturer or field service technician for help.
- *Prior to programming Load Dose, refer to Important Information in the appropriate Therapy chapters.*

Note: For information regarding the clinician access code, read the chapter entitled “Clinician Information and Access Code.” When entered correctly, the access code allows a user to change the lock settings of the pump and thereby change the settings of the pump.

IMPORTANT INFORMATION

Because the chapter entitled “Clinician Information and Access Code” contains access code information, the entire chapter should be removed before initial patient teaching and should not be left in the manual while in the patient setting. Doing so could jeopardize the security levels intended only for clinician use and access.

List of Abbreviations

#BOLS/hr	=	Number of Bolus Doses per hour In Patient controlled analgesia the number of boluses allowed per hour 0-15
Δ RATE	=	Amount rate may be changed
Δ TIME	=	Interval until rate may be changed
AMT	=	Amount
AMT	=	Amount to be infused
TBI		
BA	=	Bolus Attempted Options hourly screen shows amount of bolus doses attempted
BASL	=	Basal Basal rate or continuous basal infusion maximum varies by route of infusion, intravenous, epidural or subcutaneous
BG	=	Bolus Given Options hourly screen shows amount of bolus doses infused
BOL INT	=	Bolus Interval Allows clinician to enter RX amount of time 0-60 min between patient-activated bolus dosing for pain management
CLIN	=	Clinician Clinician dose is the option to give an additional prescribed dose by a clinician
CLRHRLY	=	Clear hourly Option to clear hourly totals of amount infused
CONCEN	=	Concentration Concentration is the amount of milligrams or micrograms per milliliter of solution
dd	=	Day To set date enter 2 digit day
DN Occlu	=	Down Occlusion Occlusion pressure in the line between the pump and the patient access site, as a kink in tubing, blockage in access site (infiltration) or clamp on
EPI	=	Epidural To inject anesthetic into the spinal canal
HH:MM	=	Hours and Minutes To set time enter Military 24 hour or AM/PM 12 hour
HX	=	History The patient history log is a cumulative record of the last 6,000 events that have occurred in the pump
INF	=	Infusion Fluid or medication pumping into the patient via intravenous, subcutaneous, or epidural routes
IV	=	Intravenous To give fluids or medication directly into the venous circulation via peripheral or central circulation
KVO	=	Keep Vein Open To infuse a very small amount of fluid to keep vein patent
LPRES	=	Line Pressure Shows downstream pressure to patient in millimeter of mercury
MAX	=	Maximum
MCG	=	Microgram
MedLMTS™	=	Sets limits for medication being infused
MG	=	Milligram
MIN	=	Minimum Minimum amount of time between patient boluses
mL	=	Milliliter A unit of liquid measure 1/1000 of a liter
mm		Month

MX	= To set date enter 2 digit month
	= Maximum
OPT	= Upper limit of a bolus or patient demand dose, or infusion rate
	= Options
	Once prescription is programmed, this key provides customizing features as locking programs from changes, alarm volume, battery level, amount of air detection, adjust occlusion alarm, hourly totals, shift totals, clinician dosing
PCA	= Patient Controlled Analgesia
	Pain control by continuous rate of infusion with options for patient controlled demand boluses
PT	= Patient
	Person receiving infusion
RX	= Prescription
	Written order for medication with instructions of how to administer as time, dose, amount, limits, and route
SENS	= Sensor
	Air-In-Line Sensor will detect the amount of air in the tubing
SQ	= Subcutaneous
	To give fluids or medication just below the skin, also (SC)
VOL	= Volume
	Amount of fluid in intravenous bag, medication reservoir or syringes
yy	= Year
	To set date enter 2 digit year

Pump Illustrations and Features

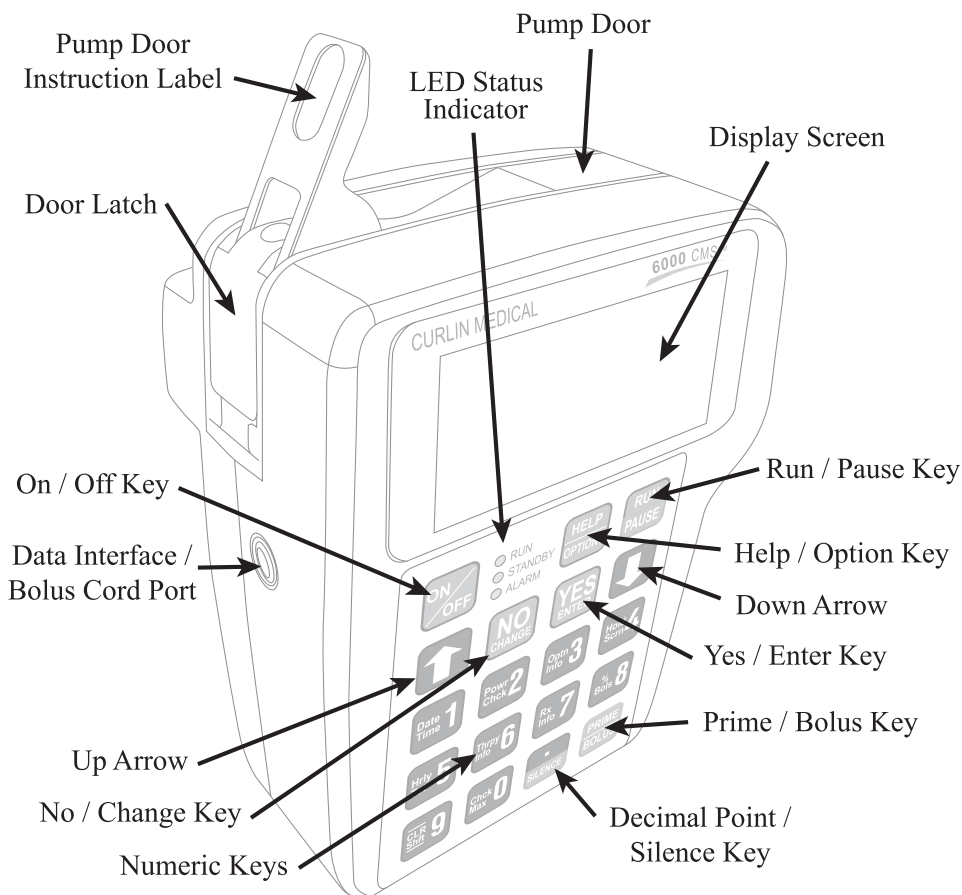


Figure 1.1 Moog Curlin Infusion Ambulatory Pump—Front, Top and Left View


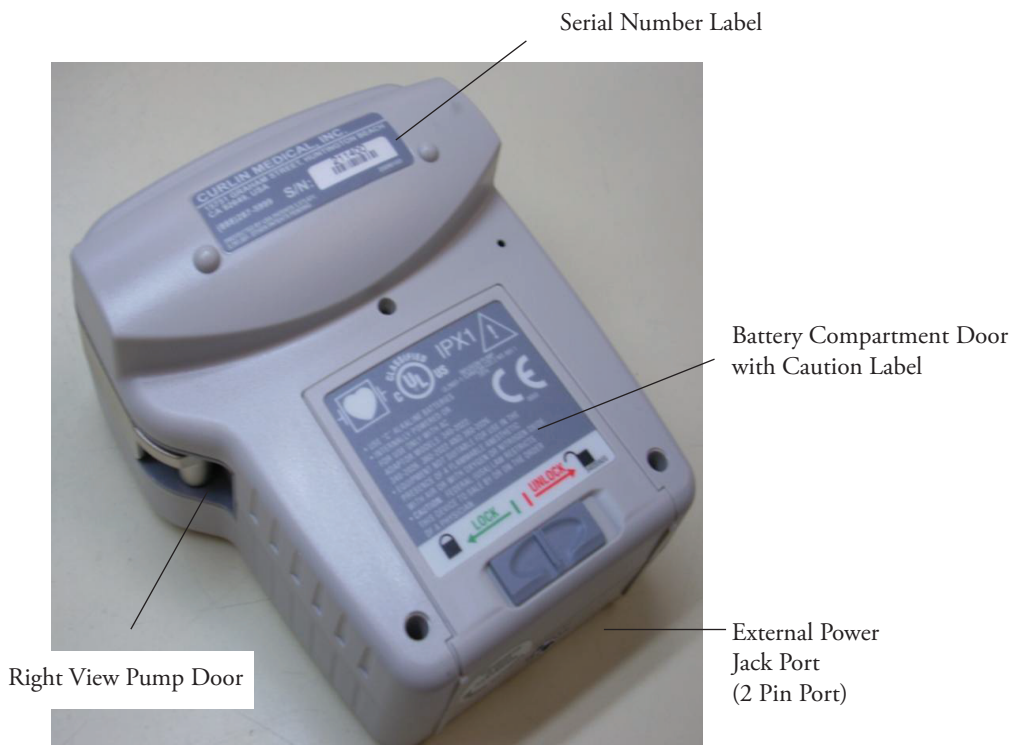


Figure 1.2 Moog Curlin Infusion Ambulatory Pump—Back, Bottom, and Right View

Disposable Administration Set Illustrations and Features

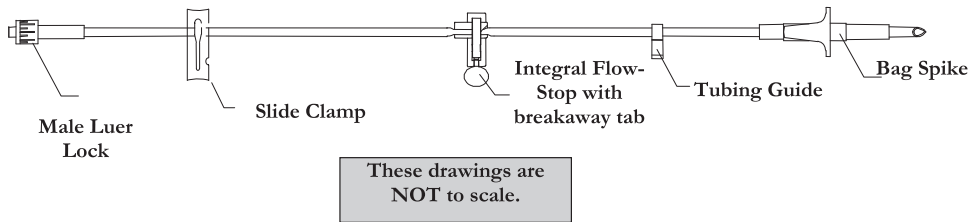

Note: An important feature of the Curlin Medical administration set is the unique design of the set-based Integral Flow-Stop. When the set is new, the Integral Flow-Stop has a breakaway tab that keeps the Integral Flow-Stop open and allows the tubing to be gravity primed. When you are ready to load the administration set into the pump, remove this tab, and the Integral Flow-Stop will automatically clamp the tubing. The Curlin Medical Integral Flow-Stop prevents inadvertent flow of medication to the patient whenever the door of the pump is opened because the Integral Flow-Stop automatically reacts to clamp the tubing. The Integral Flow-Stop can, however, be opened after the tab is broken away by intentionally squeezing down on the movable spring action section of the Integral Flow-Stop. See Figure 1.6 for an illustration of how to intentionally open the Integral Flow-Stop. For a complete listing of all available administration sets, see the Curlin Medical Product Catalog.

Note: Administration sets with anti-siphon valve (ASV) protection can also be used for additional free-flow protection.

CAUTION: USE ONLY MOOG CURLIN INFUSION ADMINISTRATION SETS WITH THIS PUMP

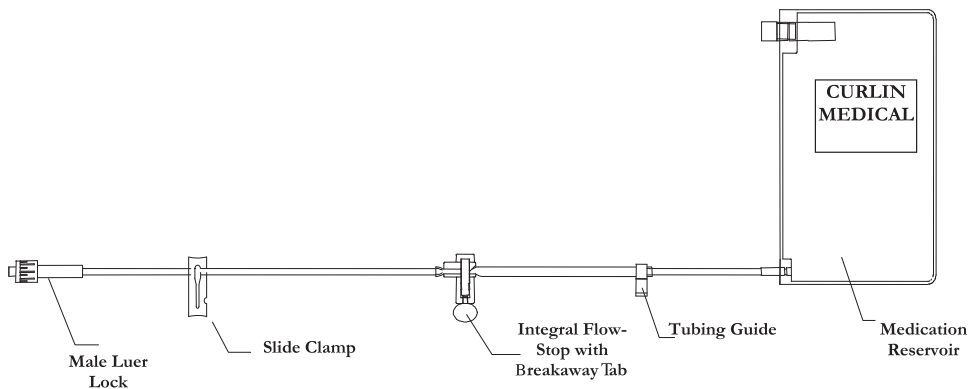


Figure 1.3 Samples of Moog Curlin Infusion Administration Sets

Preparing Medication for Infusion/Changing IV Bags

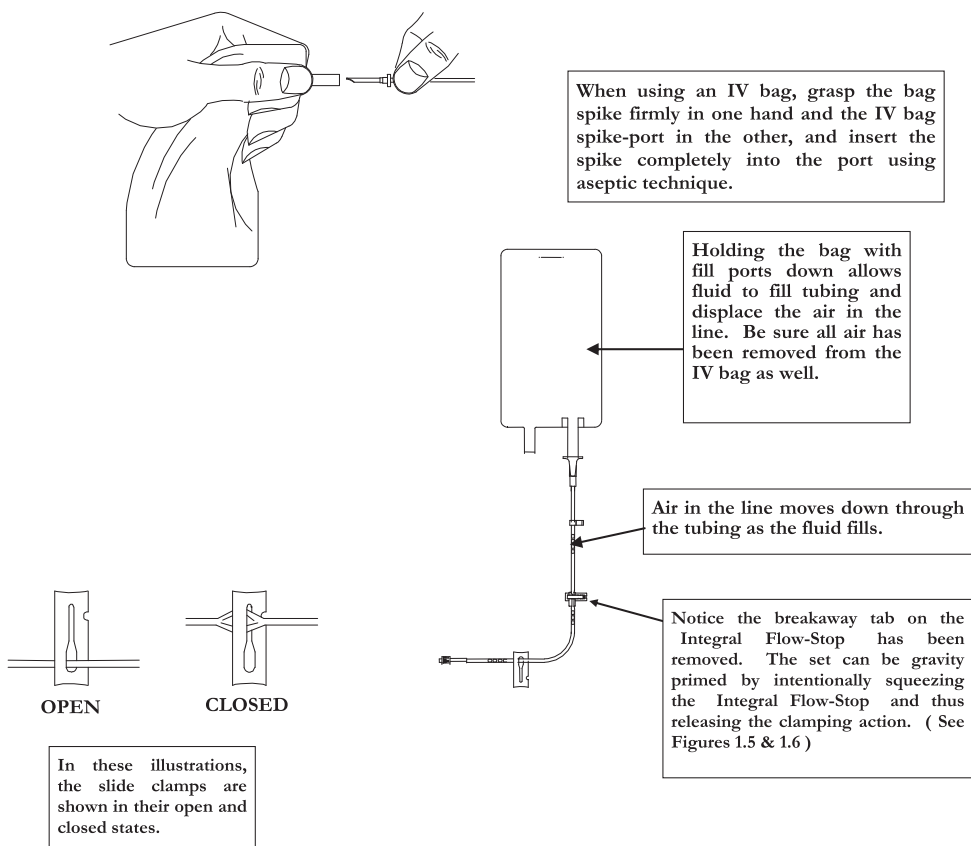


Figure 1.4 Spiking the Fluid Container, Gravity Priming the Set and Using the Slide Clamp to Close the Tubing

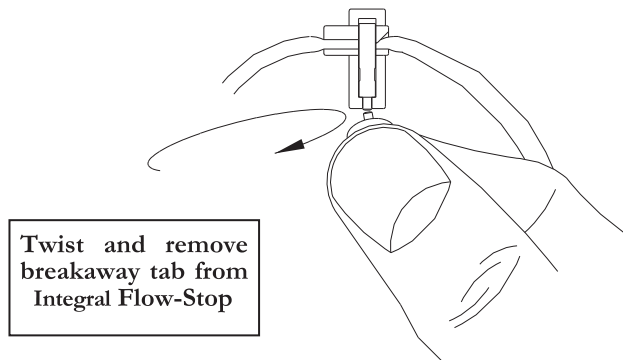


Figure 1.5 Moog Curlin Infusion Administration set Integral Flow-Stop with breakaway tab

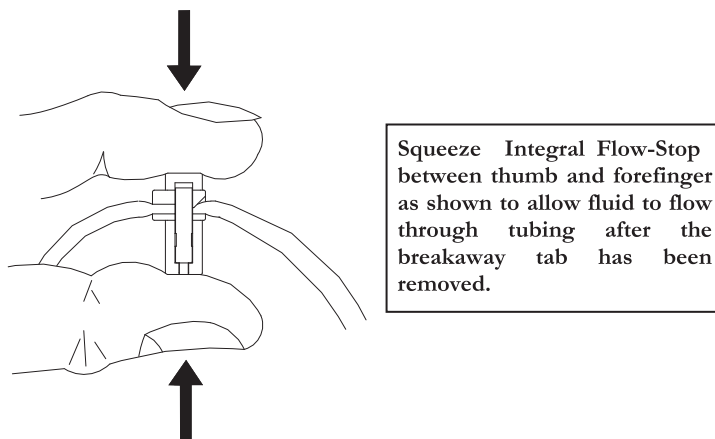


Figure 1.6 Intentionally Opening the Integral Flow-Stop

Opening and Closing the Pump Door

The pump door, located on the top of the **Moog Curlin Infusion** pump, has a sturdy “over-center” latching mechanism, which assists in securely closing the door. Review the following illustrations:

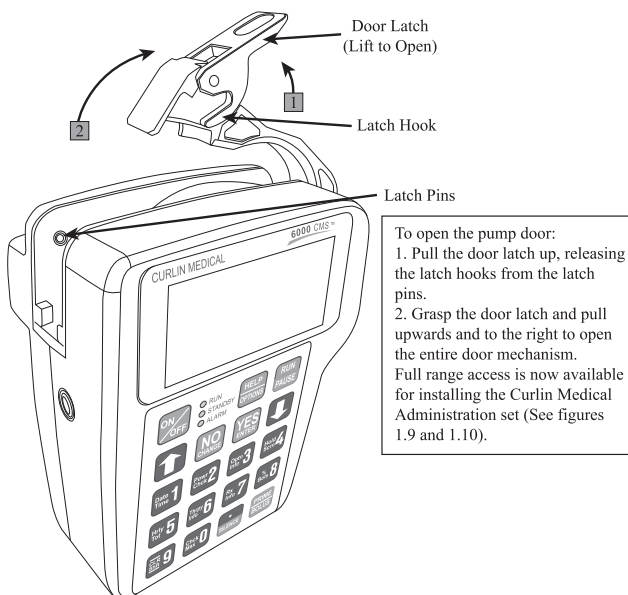


Figure 1.7 Opening the Pump Door

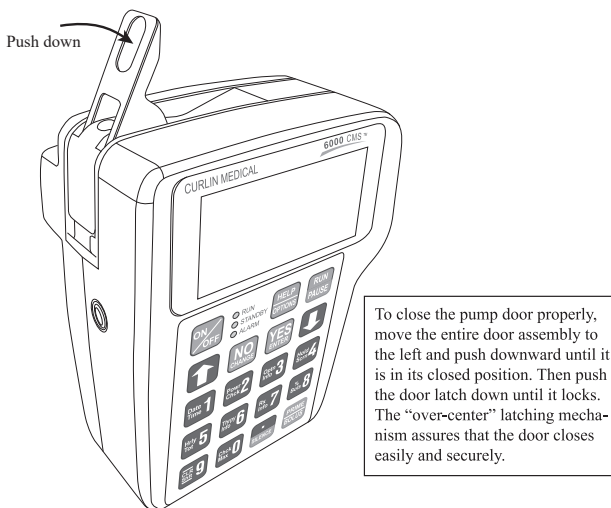


Figure 1.8 Closing the Pump Door

Choosing and Installing the Moog Curlin Infusion Administration Set

A number of distinctively designed **Moog Curlin Infusion** administration sets are available to deliver each prescription accurately. Each set is designed to meet the requirements of specific therapies, therefore, if a basic bag-spike set or a set with a medication reservoir is needed, with or without air eliminating filters, etc., there are several configurations from which to choose. Follow the healthcare provider's protocols for preparing the medication and the administration set before loading the set into the pump, remembering the following basic rules:

1. Examine the packaging and the administration set before use to assure the package integrity. Do not use the set if the protective caps are dislodged or if the package integrity is breached, as fluid path sterility cannot be guaranteed under those circumstances.
2. Notice that there is an instructional insert included in the administration set package. This insert includes written and graphic instructions for proper installation of the administration set into the pump. It also lists the "fill volume" of the administration set. Allow for the amount of the fill volume when determining the final IV bag volume.

Gravity Priming Set and Filter

1. To gravity prime the administration set, twist and remove the breakaway tab from the Integral Flow-Stop, then spike the IV bag. Squeeze the Integral Flow-Stop between your thumb and forefinger to allow fluid to flow through the tubing until all air is removed from the IV bag and administration set (See figures 1.5 and 1.6). Release the Integral Flow-Stop to automatically clamp the tubing.

Note: The set you are using may contain an air-eliminating filter. Depending on the filter type, you may need to hold the filter either upright or inverted while priming it. Refer to the administration set instructions for use for proper orientation of the filter during the priming steps. Clamp immediately after priming is complete to prevent emptying of line.

Note: Administration sets with integral anti-siphon valves cannot be gravity primed.

2. Close the slide clamp on the administration set before the door of the pump is opened. This step along with the automatic clamping properties of the Integral Flow-Stop prevents inadvertent fluid flow.
3. Use an administration set with an air-eliminating filter whenever the Air-In-Line Sensor is disabled or set to "OFF."
4. Follow CDC guidelines or Intravenous Nursing Standards of Practice Guidelines regarding the frequency of changing the administration sets.
5. Use aseptic technique and universal precautions as directed by the policies of your healthcare facility or institution.

6. Use only **Moog Curlin Infusion** administration sets with the **Moog Curlin Infusion** pump.
7. Dispose of administration sets in accordance with all applicable regulatory and institutional policies and directions.

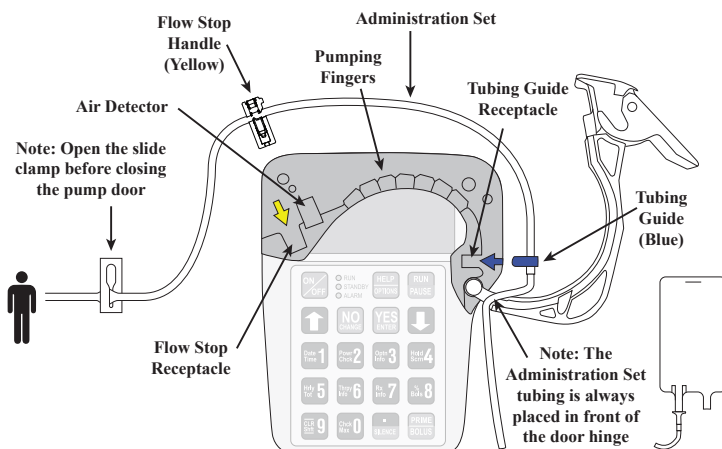


Figure 1.9 shows the Curlin Medical pump with a cut-away of the display screen allowing you to visualize the receptacles for the small *blue* Tubing Guide on the right side of the pump and the larger *yellow* Integral Flow-Stop on the left side of the pump. The arrows, in corresponding colors, are visible on the background surface of the pump head and indicate the proper direction for placing these locators into their receptacles. Also, note that the breakaway tab has been removed from the Integral Flow-Stop, allowing the administration set to be placed into the pump correctly.

Figure 1.9 Proper Installation of the Moog Curlin Infusion Administration Set into the Pump

Steps in Installing Administration Set into Moog Curlin Infusion Pumps

1. Fully open the door of the pump.
2. Close the slide-clamp on the administration set.
3. Activate the Integral Flow-Stop by twisting and removing the breakaway tab.
4. Insert the blue tubing guide into the receptacle on right side of pump in the direction of the blue arrow, positioning the tubing in **front** of the door hinge.
5. **Center the tubing in the middle of the pumping fingers, covering the yellow dot.**
6. Holding the yellow Integral Flow-Stop by the handle (see Figure 1.9), insert it on an angle into the receptacle on the left side of the pump in the direction of the yellow arrow.
CAUTION: Do not press down on the top of the Integral Flow-Stop when inserting it into the receptacle. This action could inadvertently open the Integral Flow-Stop and allow fluid to flow through the tubing.
7. Push the tubing slightly into the black air detector slot.
8. Fully close the door of the pump and latch it securely.
9. Open the slide-clamp on the administration set
10. If the set is not primed, prime the set according to prime section in the relevant therapy chapter.
11. Attach the primed set to the patient's venous access device per agency protocols.
12. Dispose of used administration sets in accordance with all applicable regulatory and institutional policies and directions.

Installing the Batteries and Using External Power

The **Moog Curlin Infusion** pump can be powered solely with the use of two C-cell alkaline batteries installed into the pump as shown in Figure 1.11. C-cell batteries are readily available at most retail facilities that carry alkaline batteries. A second alternative for power is the portable, rechargeable battery pack. The third alternative for power is to use the AC adaptor, allowing the pump to be connected to any approved, three-prong electrical wall outlet.

The C-cell batteries will last approximately 30 hours at a rate of 125 mL/hour and over 85 hours at rates of 2 mL/hr or less. Factors that influence battery life are the number of times the screen is illuminated with a key press, the viscosity of the fluid being delivered, the age and temperature at which the batteries have been stored. To extend the life of the batteries, patients can use the AC adaptor whenever they are stationary, e.g., at night.

The pump has a “low battery” alert that beeps periodically with a message appearing to tell you when the power is low in the C-cell batteries. This alert gives you a window of time to be sure you have back-up power (e.g. extra batteries, AC adaptor, or a recharged battery pack.) If you are infusing at high rates, you will have approximately one hour of warning; if you are infusing at low rates (below 10 mL/hr), you will have a significantly longer time interval before the batteries are completely empty.

The user should be taught that whenever the “low battery” alert appears, to pause the therapy, turn the pump off, and to replace both C-cell batteries. Then, turn the pump back on, select resume to continue the therapy.



When the “empty battery” alarm occurs the pump will go into an alarm state and the infusion will stop. If this occurs, turn the pump off, replace both of the C-cell batteries, turn the pump on, and go to the “Resume” function to continue the therapy. All programming information is retained. If the C-cell batteries are empty and you want to connect the AC adaptor to power the pump, turn the pump off first, plug in the AC adaptor and then turn the pump back on.

Note: If you are operating the pump solely on the battery pack or the AC adaptor (no C-cell batteries installed) and the power source is disconnected from the pump, the alarm will sound, the infusion will stop, and the display screen will be blank. Turn the pump off, reconnect the power source, turn the pump on, and resume the therapy. All programmed information is retained. However, as there is no low battery alarm for the external battery pack, the external battery pack should always be used with charged C-cell batteries.

When replacing batteries, **pause any infusion in progress by pressing the PAUSE key, and then turn the pump off.** Replace the batteries, and, when the pump is turned back on, select “RESUME” to continue the therapy from the point at which it was interrupted.

Battery Replacement, Alternate Battery Door

Figure 1.11

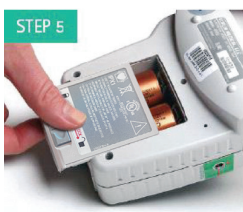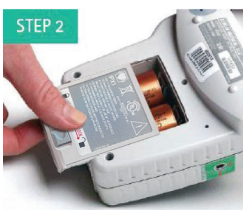

- The battery door compartment is located on the back of the pump.
- To open the door, slide the door latch to “open.”
- **Do not use a screwdriver or other sharp objects to open battery compartment door.**
- Slide the battery door down and remove it from the pump.
- If your pump contains a plastic tab between the batteries, pull up on the tab until the batteries can be removed. Otherwise:
- Using the small coin (penny) pry up and out on the top (+) edge of the battery to cause the battery to pop out.
- Use the same technique on the second battery.
- **Do not use a screwdriver or other sharp objects to pry batteries out of the pump.**
- Install two new C-cell batteries by depressing the coil spring with bottom (–) end of the battery and sliding the top of the battery into place.
- **Please note:** Both positive poles are at the top of the battery compartment. To depress the coil spring, insert the battery at a slight angle.
- Slide the battery door compartment back to its closed position and slide the door latch to “lock.”

To check battery levels while the pump is running;**

- Press the HELP/OPTIONS keys
- Scroll down to “PWR Ck?”
- Press NO to change to “ON”.
- Press YES to view battery level
- Press YES to “Accept OPT?” to exit options menu

CAUTION: If batteries are removed from the bottom (–) end first, damage to the springs in the bottom of the battery case can occur. Damaged springs can create a situation in which the battery overheats. If the battery or the pump becomes excessively warm to the touch, do not attempt to remove the batteries. Set the pump aside and contact your home infusion pharmacy.

* It is recommended to use Duracell® Procell®¹ Batteries in **Moog Curlin** pumps

** C-cell batteries only, this function will not check external battery pump power levels.

IMPORTANT INFORMATION

Whenever an external source is powering the pump, it is still important to always install the two C-cell batteries. In the event of a power failure or accidental power cord disconnection, the batteries will power the pump without interrupting the therapy in progress. If the batteries are in place when the power cord is disconnected, the display will say “External Power Disconnected,” the alert beep will sound, the batteries will take over powering the pump, and the therapy will continue infusing.

Handling and Storage:

Store at room temperature. Avoid mechanical or electrical abuse. Do not short, charge or install incorrectly. Do not mix with other battery types, as (alkaline-manganese dioxide with carbon-zinc). Replace both batteries at the same time. Do not store batteries loose. Refer to battery manufacturer’s material safety data sheet for specific product recommendations and precautions.

¹ Duracell and Procell are registered trademarks of Proctor & Gamble.

External Power Sources

Rechargeable Lithium Ion (Li-Ion) Battery Pack (360-2030) Overview

The Li-Ion Battery Pack (pictured below) is designed to be used with 2 fully charged C-cell batteries to power a model 2000, 4000, 6000 or PainSmart™ infusion pump for an extended period of time without being connected to a power outlet. The battery pack allows the user to operate the pump for a minimum of 24 hours (at 125mL/hr) in addition to the run-time supplied by the disposable C-cells in the pump. To ensure an adequate low battery warning period, the battery pack should always be used with 2 fully charged C-cell batteries installed in the pump.



The battery pack provides 5.2V with a capacity of 6000 mA-hrs. It is connected to the same port connector that the AC adaptor uses to power the pump. When the battery pack power is fully exhausted the pump will automatically revert to the internal C-cell batteries. Having new C-cell batteries in the pump is crucial to ensure an adequate low battery warning period and to ensure infusion continuity if the battery pack is accidentally disconnected. The red low battery LED on the battery pack will illuminate prior to the external battery running out of power. The pump will provide at least 30 minutes of audible and visual warning prior to the internal C-cell batteries running out power.


The battery pack is recharged by connecting it to an AC adaptor. Fully recharging the battery takes approximately 15 hours. While charging, the LED indicator will be yellow. When the battery is fully charged, the LED will be green. If a fault is encountered in the battery pack the LED will either flash yellow or not illuminate.

Warnings

- The external battery pack should be used with new C-cell batteries in the pump to avoid interruption of the infusion if the cord between the pump and battery pack becomes disconnected and to ensure proper operation of the pump's low battery alarm.
- Do not use this battery pack with the **Moog Curlin 340-2022 AC Adaptor** as the combination may exceed the allowable limits for emitted electromagnetic interference.
- Battery pack cable utilizes an attached cable. This can be a tripping or choking hazard, avoid use that may lead to entanglement around the neck or feet.

Cautions

- This battery pack is fluid resistant, but it is not designed to be water tight. Care should be taken to avoid any unnecessary contact with water or other fluids. Do not use the battery pack where it might fall into water.
- The power gauge located on the pump screen (pictured below) does not reflect the remaining charge in the battery pack. The displayed capacity on the pump will always show empty when a battery pack (with remaining capacity) is attached (see image below):
- Refer to the LED indicators on the pump to determine if the battery pack has been fully charged or is in a low battery condition.
- The battery pack should be unplugged from the pump when recharging. The AC adaptor cannot simultaneously power the pump and charge the battery pack. If AC operation is desired, connect the adaptor directly to the pump.

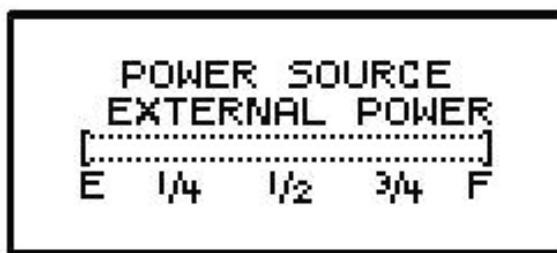

Environmental

Storage Temperature -20°C to 65°C; Atmospheric pressure 500hPa to 1060hPa; Humidity 10% to 90%

Operational Temperature 5°C to 40°C; Atmospheric pressure 650hPa to 1020hPa; Humidity 10% to 90%

Directions for Use

Charging the Battery Pack

The Moog Curlin 6000 AC Adapters can be used to recharge the battery. A fully discharged battery will recharge in less than 15 hours.


Note: Do NOT use Moog Curlin 360-2026 (Globtek GTM341-7.5-800), Moog Curlin 340-2026 (Globtek GTM341-7.5-800), or the Moog Curlin 340-2022 (POWDEC AC adaptor WM10-6.9V) AC adaptors to recharge the battery.

- If connected, disconnect the battery pack from the pump.
- Connect the AC adaptor to the power port of the battery pack.




Connect *AC adaptor*
here

- The charge LED will illuminate yellow if the battery needs to be recharged. See below:

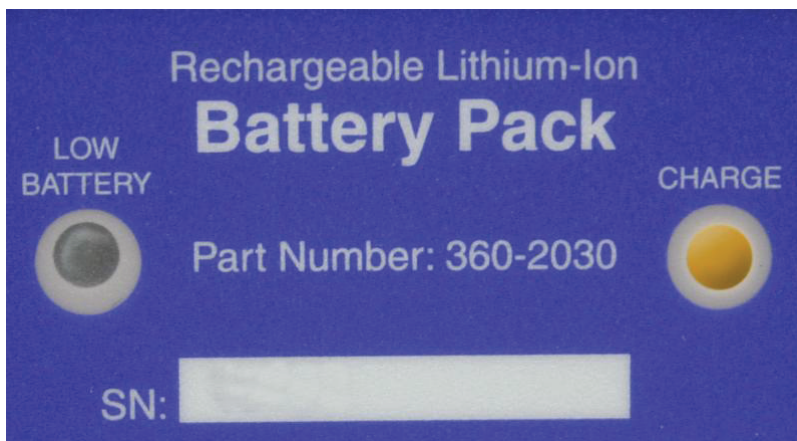

- Charge the battery pack for 15 hours or until the LED turns green indicating the battery is fully charged.

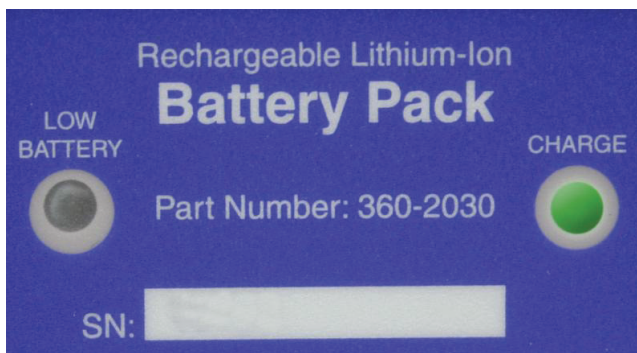

- Disconnect the battery from the AC adaptor.

Note: If there is a fault detected while charging the battery, the LED will either flash yellow or not illuminate. Most faults are due to either the AC adaptor becoming unplugged from the power outlet or the battery pack being at a location where the temperature is not between 5 and 40 degrees celsius. If the fault condition persists, then the battery pack should be returned to Moog Medical Devices Group for replacement of the battery.


Using the Battery Pack to Power the Pump

- Make sure fully charged C-cell batteries are loaded in the pump.
- Connect the battery pack to the AC adaptor port on the pump.



- Operate the pump in the desired infusion mode.
- Do not use the pump's display (see cautions) to predict external battery life.

- When the external battery pack is nearly depleted, the red LED indicator will be illuminated.

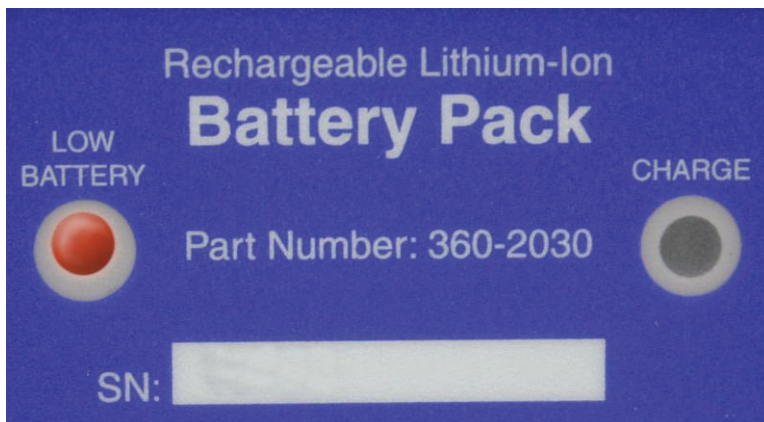

- The pump may continue to be used until the disposable C-cells are depleted.
- Disconnect battery pack from pump prior to recharging the battery pack.

Maintenance

There is no need to fully discharge the batteries periodically since Li-Ion batteries do not have a memory effect. The battery pack enclosure may be cleaned with a mild soap or disinfectant.

End of Life Determination

The battery in the battery pack degrades with charge cycles and time since manufacture. When the batteries are no longer holding their charge for a sufficient length of time, the battery pack should be replaced.

AC Adaptor/Charger

This accessory becomes the external power source when plugged directly into the **Moog Curlin Infusion** pump (see Figure 1.2 for external power port location on the bottom of the pump). The AC adaptor can also be used to recharge the portable battery pack by plugging the connecting cable into the recharging port on the battery pack (see Figure 1.13).

There are two AC adapters:

- The US Domestic Curlin 6000 AC Adapters have a three-prong plug to be plugged into a standard three-prong, grounded wall receptacle and a six-foot cable which plugs into the power outlet on the bottom of the pump. The two-pin connector plugs into the receptacle by aligning the red dots on each.
- The International Curlin 6000 AC Adapters are available for international use. The two-pin connector plugs into the receptacle by aligning the key on the connector.

Note: Pause the pump and then turn it off before connecting the AC adaptor.

Note: The connector is keyed with a “notch” at this red-dot location to guide alignment. To remove, pull back on the locking sleeve to unlock and then pull gently to remove the connector from the battery pack or pump’s port.

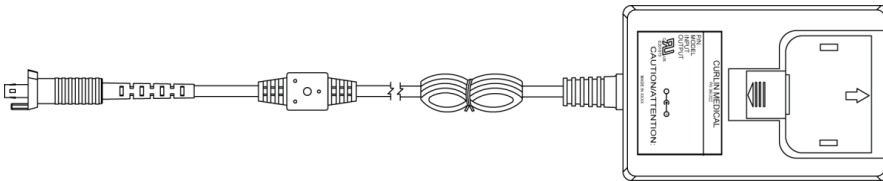


Figure 1.12 International AC adapter

Two-pin adaptor plugs into port on bottom of the pump for AC power or can be used to recharge the battery pack by plugging into the recharging port on the battery pack (see Figure 1.12). To remove, pull back on the locking sleeve.

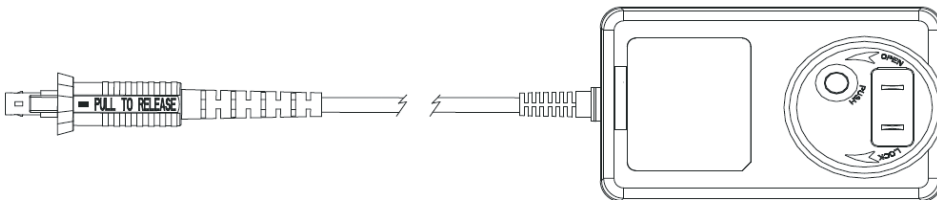


Figure 1.13 International AC Adapter

Bolus Cord Use

When the PCA therapy is used, patient bolus dosing can be programmed. To facilitate ease of patient self-dosing, the patient can use the Bolus key or the Remote Bolus Access Cord. When a bolus dose is requested two beeps will sound to indicate that the dose has started. If a bolus dose is requested and denied, one beep will sound. When a bolus dose is completed, no beep will occur.

The cord is five feet long and has a patient access button at the top of the remote cord. The four-pin adaptor plugs into the four-hole port on the left side of the pump (see Figure 1.13 below). This port may be covered with a plastic plug. If so, remove the plug to access the port. To correctly connect the bolus cord, align the red dot and the connector's keyed notch with the red dot and notch in the port on the left side of the pump. When the cord is plugged into the pump it is locked in place by the locking sleeve. To remove the cord, pull back on the soft, green locking sleeve and disengage the cord from the pump by pulling it straight out of the access port. Because bolus dosing is applicable only in PCA therapy, if the cord is left in place and the button is pushed while any other therapy is infusing, an alert message will appear on the display screen stating: "Not in PCA No Bolus Allowed."

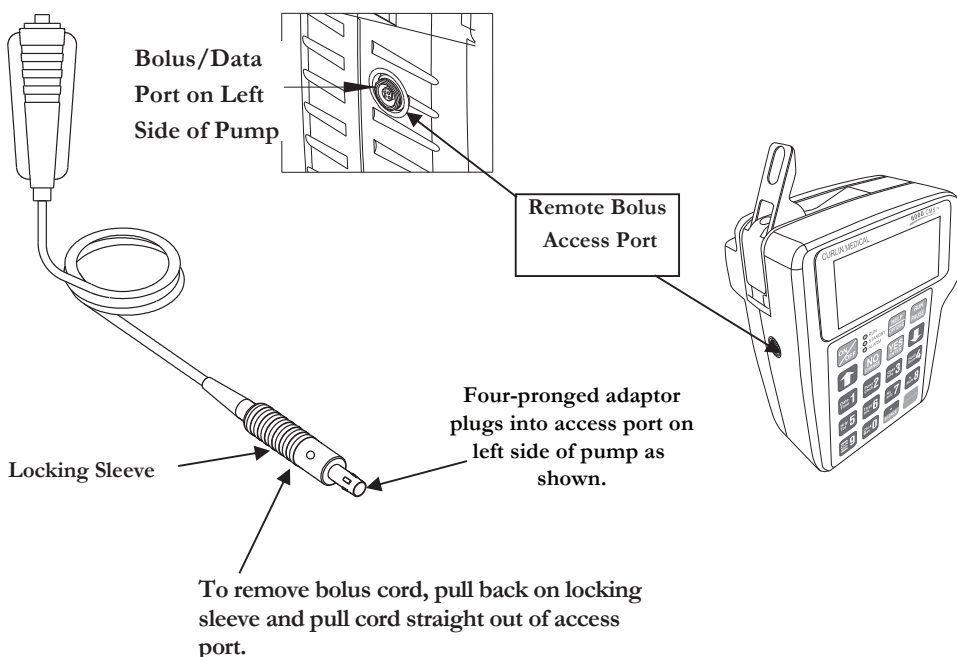


Figure 1.14 Attaching and Removing the Remote Bolus Cord

Learning Pump Features

*Learning to operate the **Moog Curlin Infusion** pump requires a few basic points of understanding that remain constant in every therapy. It may be helpful to read through these steps first, and then go back and use them to operate the **Moog Curlin Infusion** pump.*

First, examine the key pad and the functions of the individual keys. Knowing how the keys, especially the dual function keys, work will allow you to give commands to the pump.

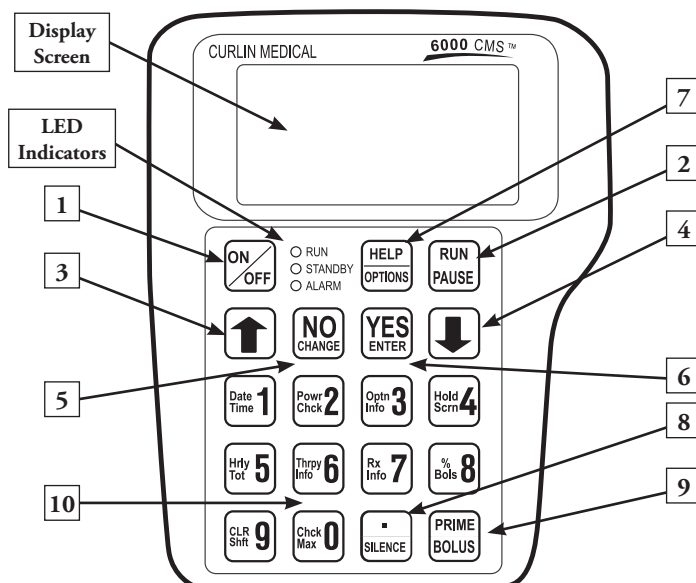


Figure 2.1 Key Pad

Keypad Function

1. **ON/OFF:** Used to turn the pump on and off. When the pump is running, however, the PAUSE key must be used to stop the infusion first and then the OFF key can be pressed to turn the pump off. This requirement prevents the interruption of an infusion by a single accidental press of the OFF key.
2. **RUN/PAUSE:** Used to start or pause the motor of the pump, which, in turn, starts or pauses fluid delivery. When the pump is running, pressing this key will pause the pump. When the pump is paused, pressing this key during the Run Options Menu will start the pump. This key is also used to terminate a pump alarm.
Note: If the cause of the alarm is not resolved when the pump is resumed, the alarm will recur.
3. **↑ Up arrow:** Used to scroll the highlight bar or field cursor *up* on the display screen.
Note: The Up arrow will not accept a selection which must be made with a YES/ENTER or NO/CHANGE key.
4. **↓ Down arrow:** Used to scroll the highlight bar or field cursor *down* on the display screen.
Note: The Down arrow will not accept a selection which must be made with a YES/ENTER or NO/CHANGE key.
5. **NO/CHANGE:** Used to respond “NO” to data or questions presented or to reject or to change highlighted data.
Note: The **Moog Curlin Infusion** pump requires you to actively change a selection. The Up arrow or Down arrow will not change a selection.
6. **YES/ENTER:** Used to respond “YES/ENTER” to data or questions presented or to “accept” highlighted data when prompted to press ENTER key.
Note: The **Moog Curlin Infusion** pump requires you to affirmatively accept a selection. The Up arrow or Down arrow will not accept a selection.
7. **HELP/OPTIONS:** Used to request a help screen when the pump is paused or is at the “Infusion Complete Screen.” It is also used to access the options screen when the pump is running. When the options screen is displayed, help is again available with this key. When in a Help Screen, use the arrow keys to scroll through the help message, and then press the Help key again to exit Help.
8. **./SILENCE:** Used as a decimal point when programming numeric data. Used also to temporarily silence the alarm (for one-minute intervals).
9. **PRIME/BOLUS:** Used to prime fluid through the tubing to clear it of air or to administer a patient demand bolus dose of medication in PCA therapy.
10. **0 - 9 Numeric keys:** - Used for data entry.

Description of the Info-On-Demand (IOD) Feature



Figure 2.2 IOD Keypad with Info-On-Demand Feature

To view any of the Info-On-Demand screens on the pump display, press the related key while the pump is in the “RUN” mode, i.e., running a therapy. The information requested will be displayed for 8 seconds and then the display will return to the previous infusion screen.

One exception is the HrlyTot /5 key.

The information for the HrlyTot /5 key will be displayed for 15 seconds and then the display will return to the previous infusion screen.


Another exception is the CLRShft /9 key.

The information for the CLRShft/9 key will be displayed for 11 seconds and it will allow the user to “CLEAR” or “EXIT” within the 11 seconds or the screen will revert to the “RUN” screen automatically.


Info-On-Demand (IOD) Key Functions

Designation and functionality of the ten numeric keys of the Info-On-Demand keyboard:


1. **Date and Time (DateTime):** The Date and Time Screen displays the current date and time according to the pump. Changes to this screen are not allowed.

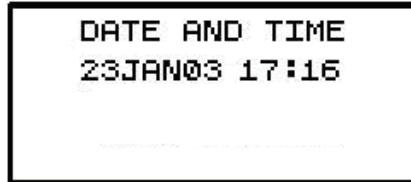


2. **Power Check (PowerChck):** The Power Check Screen displays the active power source of the pump, whether internal (batteries) or external. If the active source is the C-cell batteries, a bar graph will indicate the amount of power remaining in the batteries. This graph is only an approximation of the charge left in the batteries and not intended to give specific representation of the battery life. Furthermore, the bar graph should NOT be used to check the state of the external NiMH Battery Pack, because it will not provide with accurate results.

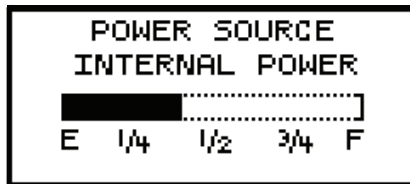

3. **Option Info (OptnInfo):** This screen provides information on selected options. The information displayed here includes the status of the "Lock Level," "Down Occlusion," "Air Sensors," and the "Audio Level". Changes to this screen are not allowed.

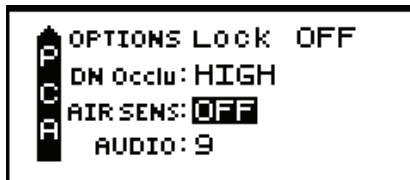

4. **Hold Screen (HoldScrn):** Pressing the Hold Scrn/4 key will hold the current Run Screen information for an additional 8 seconds and then the pump will return to the normal running screen. Changes to this screen are not allowed.

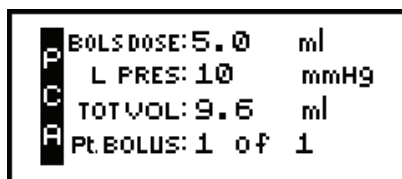

5. **Hourly Total (HrlyTot):** The information displayed here includes the hourly “Volume (amount) Infused,” the number of “Bolus Given” and the number of “Bolus Asked”. Within the allocated 15 seconds, the user can scroll up or down the screen. Changes to this screen are not allowed.

P C A	HR	VOL-ml	BG	BA
	19	9.0	0	0
	20	10.9	1	2
	214	4.8	0	0


6. **Therapy Information (ThrpyInfo):** The information displayed in this screen is from the pre-prescription menu and includes “Units,” “Concentration,” “Administration Route” and “Load Dose”. Changes to this screen are not allowed.

P C A	UNITS: mg		
	CONCEN:	0.0	mg/ml
	ADMINRT:	IV	
	LOADDOSE:	0.0	mg

7. **Rx Information (RxInfo):** This screen provides prescription menu information such as “Basal Rate,” “Patient Bolus,” “Bolus Interval,” and “Number of Bolus per Hour”. Changes to this screen are not allowed.

P C A	BASLRATE:	8.0	ml/hr
	PT.BOLUS:	5.0	ml
	BOLS INT:	6	MIN
	#BOLS/hr:	10	



8. **Percent Bolus (%Bols):** This screen provides Bolus Volume as percentage of Total Volume. “TBOL” is the total volume infused through bolus, “TVOL” is the total volume infused, and “% BOLS” is the percentage of TBOL over TVOL. Changes to this screen are not allowed.

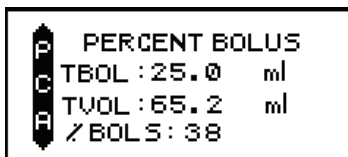


9. **Clear Shift Total (CLRShft):** Two SHIFT TOTAL screens are displayed for 11 seconds when this key is pressed. This key allows the user to view or CLEAR the SHIFT TOTALS (this action does not interfere with the operation of the pump).

To **CLEAR** the SHIFT TOTAL information to zero, press the CLRShft/9 IOD key, and then press the YES/ENTER button in response to the “DONE? CLEAR” question and the pump will go back to the running screen. Press the CLRShft/9 IOD key again to verify that the values are reset or cleared.

To **EXIT** the SHIFT TOTAL screens without clearing, press NO/CHANGE for the “DONE? CLEAR” question and then YES/ENTER for “DONE? EXIT” and return to the running screen. The pump will return to the running screen automatically after 11 seconds have elapsed since the pressing of the CLRShft/9 IOD key, even if no key is pressed.

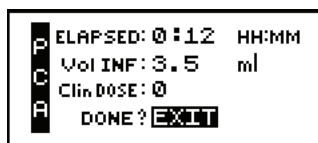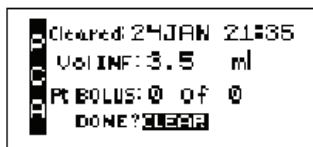

10. **Check Maximum Infusion (ChckMax):** The information displayed here is the maximum (rate or amount) that can be infused hypothetically, given the prescription parameters. This “Maximum Infusion per Hour” serves as a “approximation check,” after the pump has begun infusing. Changes to this screen are not allowed.

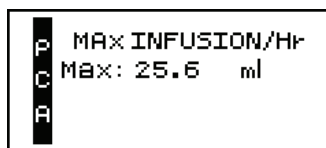

LED Light Indicators

This three-light feature gives the user feedback on the current status of the pump.

1. **Green** indicates a RUNNING status of the pump and blinks approximately once every three seconds when a therapy is in progress.
2. **Yellow** indicates a STANDBY status of the pump and blinks while the pump is paused.
3. **Red** indicates a STOPPED or ALARM status of the pump and blinks whenever an alarm status exists.
4. **Green and Red** indicates an Infusion Complete Alarm status with KVO or a Down Occlusion Alarm with automatic restart.
5. **Yellow and Green** indicates a “STANDBY” status with automatic start. This occurs during any delay start with a “0” KVO programmed or in PCA during a “0” basal rate and bolus only programmed.

Audio Indicator

An audible alarm or beep is used to alert you to a change in the status of the pump or a situation requiring user action or notification. A message will appear on the display screen describing the situation. The rate, number of beeps, and volume further specify status. This pump offers nine different volume levels from which to select for the audio feature; however, the beep cannot be totally disabled. The audio beep can be silenced for one-minute intervals by pressing the SILENCE”key except during an Error Code, at which time the alarm continues and the pump must be turned off. The chapter of this manual entitled, *Troubleshooting* includes tables with the alerts, alarms, and error codes, the conditions causing them, the message displayed on the screen, and the actions required for resolution. If an alert or alarm occurs, please refer to these tables.

Display Screen

The display screen provides a maximum of four lines of text, or fields, at any one time for feedback regarding the pump status and data entry. It notifies the user with prompts or messages when necessary. The screen has backlighting that remains:

- on if connected to an AC outlet
- lit for 30 seconds if using batteries or 120 seconds during any programming key press sequences
- on with alert/alarm messages

When no keys are pressed, the light goes off to conserve battery power. Backlighting is resumed with any key press.

In most displays, a “Therapy Identifier” bar appears at the far left of the screen, indicating which therapy is running or which programming screen is being considered. This identifier also has another function, and that is to alert you when more text exists either above the top line or below the bottom line on the display screen. The identifier bar will have an arrow configuration either pointing up or down or both whenever there is more text, or it will appear as a squared-off block if no further text exists.

The screen also has a cursor or highlight bar to tell you which field is being considered. Use the up and down arrow keys to move the cursor. When the pump is running, a specially designed “running screen” appears to provide information regarding the status of that particular infusion. In the following chapters, each display screen will be explained.

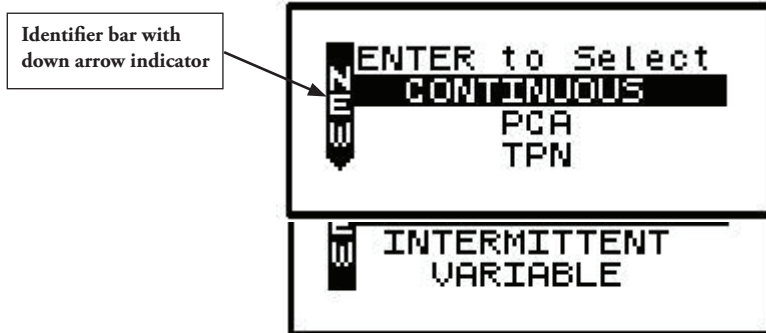


Figure 2.3 Sample of display screen

Basics of Programming

The **Moog Curlin Infusion** pump allows the operator to customize this device to meet the specific infusion needs of each individual patient. This chapter will explain the customizing features along with the programming steps.


Programming the **Moog Curlin Infusion** pump requires initial orientation and training, but the programming steps have been designed to be user friendly and self-prompting to minimize training time.

Several different types of screens and fields may appear on the displays. Each of the different types of screens is explained below. Whenever you are required to interact with the pump, a highlight bar or cursor will be present on the screen.

Types of Display Screens

1. An Action Field Screen allows you to select a pump action from the menu.

Sample of an Action Field Screen

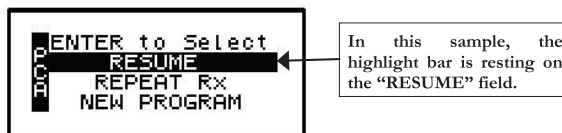

In this sample, the highlight bar is resting on the "RESUME" field.

To select a field other than the one highlighted, use the arrow keys to move the cursor up or down to the desired field. When the cursor is highlighting the correct field and you press the YES/ENTER key, the pump accepts the selection and executes that action. In the example above, the pump would proceed with steps to resume the therapy.

2. A Selection List field allows you to select from a predetermined list of values:

Sample of a Selection List Field




The above example allows you to select which unit to use in programming—e.g., units: mL, mg or mcg. Only one selection is displayed at a time. When the cursor blinks it is indicating that no selection has yet been accepted. The selection displayed on the screen changes each time you press the NO/CHANGE key. Pressing the YES/ENTER key accepts the selection displayed and moves the cursor to the next field.

When programming the pump outside of the LIBRARY mode, the word "Unidentified" will be displayed on the top line of several of the screens. This indicates that the pump has been manually programmed. "Unidentified" is a place holder where the drug name would be displayed if the program had been chosen from the library.

3. Data Entry Screens are used to specify prescription parameters.


Sample of a Data Entry Screen

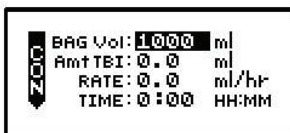

In the above example, the cursor rests on a field requiring numeric data entry for specific prescription information, e.g., the volume in the IV bag. The operator enters the correct information using the numeric keys and decimal point key. When the correct data is displayed and the YES/ENTER key is pressed, the pump accepts the selection if it is a valid entry value and moves the cursor down to the next field. If the down arrow key is used to move to the next field, the value reverts back to default or the last programmed value.

4. Run Screens display information to the operator about the progress of the specific infusion while the pump is running:


Sample of a Run Screen

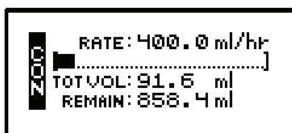

The standard Run Screen displays the Mode Identifier, the rate of pumping, a graphical bar that indicates the progress of the infusion, the volume already infused, and the time remaining for the infusion. No action is required on a run screen, therefore, no highlight bar appears. In some therapies a field may toggle to give the operator more information, i.e. the “REMAIN” field may change from *HH:MM* (time remaining) to *mL* to tell you how much volume is remaining as well. The information on the run screen will alternate between a four-line display and a large font display of each line of information on the screen.

5. User Notification Screens are messages displayed to the user for information.

Sample of User Notification Screen

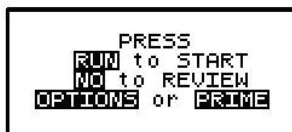

Words representing keys are displayed in reverse video (shown in this manual as bold text). The action requested by a notification screen is either to press a specified key or to resolve an alert, alarm, or error code.

For all events such as alerts, alarms, or error codes, specific notification screens will also be displayed. (See chapter entitled, “Troubleshooting” for specific alert, alarm, and error code messages and how to resolve them). The audio alarm also beeps and the LEDs blink whenever an alarm or error code occurs. In the event of an error code, user directions will be displayed on the screen, most often, this will require turning the pump off and back on again.


Using “Help”

Help Screens are available for data entry/selection screens, during Action Field screens (refer to sample screen on pg. 39), and at the Infusion Complete screen. To access help, press the HELP key. Use the arrow keys to scroll through the help message, and then press the HELP key again to exit help. When a help screen is displayed and there is more text to read which is not currently on the screen, a small up (▲) or down (▼) arrow indicator will appear to let you know you can scroll up or down to read the complete help message.

Starting the Pump

Start the pump by pushing the orange ON key. The following screen displays:

W e l c o m e S c r e e n

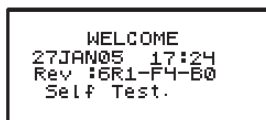

The above WELCOME screen gives the date, time, and a message that the pump is performing a self-diagnostic test. The display also lists the software revision number for this pump and if library is enabled, the library name will appear at the bottom of the screen.

During this test, you will hear three beeps as each of the LED indicators blink. If any one of the green, yellow, or red LED lights fails to light or to blink, return the pump for service. If the self-test fails, the alarm will sound, and a new screen will appear stating: “ERROR CODE:#” (See “Troubleshooting” chapter for a list of error notices and how to resolve them). The error notice will include a numeric code. At any error notice, the only user option is to turn the pump off.


IMPORTANT INFORMATION

If, at any time, there is indication of pump damage, error code, or messages requiring the pump to be turned off, call the healthcare agency or facility as directed to report the condition.

Note: If an error notice appears, turn the pump off and back on again. If the pump passes its start-up diagnostics tests, you can safely resume the therapy. If the error notice recurs, however, turn the pump off and notify the healthcare agency responsible for monitoring the pump. Some error notices may reset the pump back to its default settings and therefore will require reprogramming.

When the **Moog Curlin Infusion** pump successfully passes its self-test, a brief message of “System OK” appears, and the display moves to the next screen:

Power Status Screen

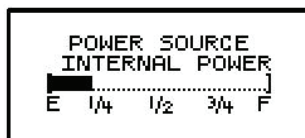


This screen displays the active power source of the pump, whether internal (batteries) or external power. If the active source is the C-cell batteries, a bar graph will indicate the amount of power remaining in the batteries. This graph is only an *approximation* of the charge left and is not intended to give a specific representation of battery life. The display will again automatically move to the next screen:

Preventive Maintenance Screen



The above alert screen will appear after the Welcome Screen *only* when the maintenance due date has arrived and will continue to appear until the date has been reset during the actual maintenance performance. The pump may continue to be used after the maintenance due date but should be returned for its routine maintenance at the earliest opportunity.

Select Program or BioMed Setup Screen

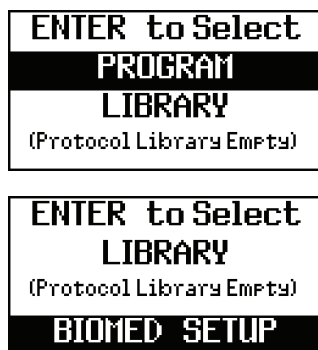


This screen is the first requiring you to act. It gives two selections from which to choose, “PROGRAM,” “LIBRARY” or scroll down two screens from Library to “BIOMED SETUP.” Using the up or down arrows, move the cursor to the desired field and then press the ENTER key. Selecting “PROGRAM” allows you to proceed with choosing features and entering data required for the pump to deliver a prescribed therapy.

Selecting “LIBRARY” allows you to enter customized library and choose pre-programmed prescription to deliver therapy. Library is an optional feature on some pumps.

Selecting “BIOMED SETUP” allows the pump to be programmed and customized by the clinician to perform a number of special functions including setting the system clock (See “BIOMED SETUP”) Accessing “BIOMED SETUP” requires you to enter a clinician access code. (Refer to the chapter entitled, “Clinician Information and Access Code,” of this manual.)

Access Code

Clinician Access Code Screen

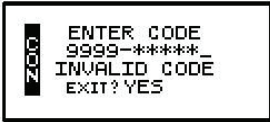

The clinician enters a four-digit personal identification number followed by the five-digit access code; and, if the code is valid, the display will automatically move to the requested screen or field. The four-digit personal identification number appears on the screen but the access code numbers do not; instead five asterisks appear. **The personal identification number is recorded in the history file for the purpose of providing an audit trail.** If an invalid code is accidentally entered, a message displays: “Invalid Code,” and the clinician can reenter the correct code.

Note: After entering the access code, DO NOT press the YES/ENTER key. The pump will automatically recognize the correct code and give you access to the secured section.


IMPORTANT INFORMATION

If an access code screen displays and the access code is not known, use the down arrow to move the cursor to the “EXIT?” field. Press the YES/ENTER key at this field to return to the previous screen and make another selection. Also, remember that the HELP key is available when the pump is paused.

Look first at the BIOMED SETUP menu and its special customization features:

BioMed Setup

BioMed Setup Menu

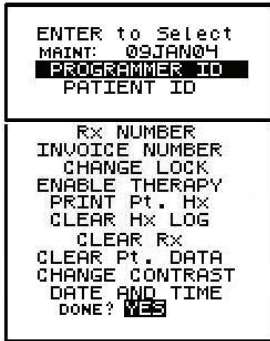

The BIOMED SETUP menu lists special optional customization and performance functions of the pump:

**IMPORTANT
INFORMATION**

You may skip any of these functions by using the down arrow to move the cursor to only the desired selection. When the desired selection is highlighted, press the YES/ENTER key. A secondary screen will be displayed. When all BIOMED SETUP functions are completed, move the cursor to the last field, “DONE? YES,” and press the YES/ENTER key to exit this menu.

1. PREVENTIVE MAINTENANCE DATE: The first field displays the date the pump requires its annual routine preventive maintenance check.

Note: You cannot change this date; it is changed only when a factory technician performs the routine pump maintenance.

The clinician that initially programs the pump for a patient should note the maintenance date. Once this date arrives, a “Perform Routine Maintenance” message will briefly appear after the welcome screen. If the maintenance due date will occur during the time the patient will be on service, the clinician **may** wish to select another **Moog Curlin Infusion** pump for this patient. This action will avoid any undue concern on the patient’s part at the notice of “Maintenance Due” and eliminate the need to recall a pump in service for maintenance. The pump, however, can continue to be used without disrupting service when this message appears, but should be returned for service as quickly as possible.

2. PROGRAMMER ID: This feature gives the programmer the option to use the numeric keypad to enter his or her assigned ID number (up to ten digits). If this option is not to be used, it can be left blank and skipped, using the down arrow to scroll to the next field. When the PROGRAMMER ID number is satisfactory, press the YES/ENTER key at the “ACCEPT? YES/NO” field and the number will be stored in the pump history log.
3. PATIENT ID: This feature gives the programmer the option to use the numeric keypad to enter a patient ID number (up to ten digits). If this option is not to be used, it can be left blank and skipped, using the down arrow to scroll to the next field. When the PATIENT ID number is satisfactory, press the YES/ENTER key at the “ACCEPT? YES/NO” field and the number will be stored in the pump history log.
4. Rx NUMBER: This feature gives the programmer the option to use the numeric keypad to enter a prescription number (up to ten digits). If this option is not to be used, it can be left blank and skipped, using the down arrow to scroll to the next field. When the Rx NUMBER is satisfactory, press the YES/ENTER key at the “ACCEPT? YES/NO” field and the number will be stored in the pump history log.
5. INVOICE NUMBER: This feature gives the programmer the option to use the numeric keypad to enter an invoice number (up to ten digits). If this option is not to be used, it can be left blank and skipped, using the down arrow to scroll to the next field. When the INVOICE

NUMBER is satisfactory, press the YES/ENTER key at the “ACCEPT? YES/NO” field and the number will be stored in the pump history log.

6. **CHANGE LOCK:** This selection allows the clinician to change the lock setting of the pump. This feature is also available under the **OPTIONS** menu of each therapy. There are four choices in this field: “OFF,” “1,” “2,” or “3”. When the lock is set to “OFF,” the clinician has access to all pump functions. Each level higher limits the options available to the patient or caregiver (see each lock option screen in the individual therapy sections and/or the lock level table in the chapter entitled “Clinician Information and Access Code”.) If the desired lock does not appear on the display, press the NO/CHANGE key until the correct lock does appear. Then, press the YES/ENTER key to accept and move the cursor to the next field.
7. **MANUAL PROGRAM:** This field is used to customize the **Moog Curlin Infusion** pump to ENABLE/DISABLE the PROGRAM menu option on the main menu. There are two choices: “OFF” and “ON”. When MANUAL PROGRAM is set to “OFF” the PROGRAM menu option is not available on the main menu.
 - a. To enable MANUAL PROGRAM, press the NO/CHANGE key until the selection states “ON.”
 - b. To disable MANUAL PROGRAM, press the NO/CHANGE key until the selection states “OFF.”
 - c. When the “ON/OFF” status is satisfactory, move the cursor to the last field, “ACCEPT?” and press the YES/ENTER key to accept. To abort the enable therapy process, press the NO key to exit this menu and return to the BIOMED SETUP Menu.
8. **ENABLE THERAPY:** This field is used to customize the **Moog Curlin Infusion** pump to the specific therapy ordered for the patient. When you select this function by pressing the YES/ENTER key, the menu below appears, listing all five of the possible therapies. The pump’s default setting is to enable all therapies. You can choose, however, to *enable or disable* any of the therapies listed. Once a therapy is disabled, no further prompts will appear in programming for that therapy.

Enable Therapy Menu

ENABLE THERAPY COM: OFF PCA: ON TPN: OFF
INT: OFF VAR: OFF ACCEPT? YES/NO

- d. To enable a therapy, move the cursor to that therapy selection and press the YES/ENTER key when the selection states “ON.”
- e. To disable a therapy, move the cursor to that therapy selection and press the NO/CHANGE key. The “ON” status will change to “OFF.” Press the YES key to accept the “OFF” or disable status.

f. When the “ON/OFF” status of each therapy is satisfactory, move the cursor to the last field, “ACCEPT?” and press the YES/ENTER key to accept. To abort the enable therapy process, press the NO key to exit this menu and return to the BIOMED SETUP Menu.

g. At least one therapy must remain enabled before you exit this screen.

Note: If a therapy that has previously been programmed is disabled, the program is cleared, and the default settings are reestablished.

Patient History Log

The patient history log is a cumulative record of the approximately last 6,000 events that have occurred in the pump. Example: When prescription information is saved using the YES/ENTER key at the “DONE” or “ACCEPT” fields, a copy of the information also gets ‘date and time stamped’ and is recorded in the patient history log. Alarms, error codes, certain alerts, and certain key press events also get logged when they occur. The patient history log can be printed using the **Moog Curlin Infusion** print package and by selecting the “Print Pt. Hx” option. Selecting the “Clear Hx Log” option will erase the patient history log. When the history log is full, it will overwrite on a “first-in, first-out” basis.

Note: The patient history log should be printed before clearing occurs if a hard copy record is required. The history log, when printed, lists the most recent event first. Clearing the history log does not affect the saved patient information.

When printed, the patient history log includes a heading at its beginning and a footer at its end. The heading includes the pump serial number, the patient history log title and revision number information of the software and hardware of the pump. The footer contains information about how much of the history log has been used. The information is printed with the most recent event first, and organized for easy transfer to a spreadsheet for sorting and reporting. A partial sample of a history log is given below:

History Event Log

```

SERIAL NUMBER          :101293
MANUFACTURE DATE       :10AUG00
MAINTENANCE DUE DATE   :16JAN04

HISTORY EVENT LOG
22JAN03 15:38 ml/Vin/mmH/BV:9070.6/2.2/8/0.76
22JAN03 15:18 ml/Vin/mmH/BV:8937.3/2.2/10/0.76
22JAN03 14:58 ml/Vin/mmH/BV:8803.9/2.2/10/0.76
~~~~~
21JAN03 16:58 ml/Vin/mmH/BV:2.1/3.1/0/0.78
21JAN03 16:57 ml/Vex/mmH/BV:0.0/6.7/0/0.78
21JAN03 16:57 START PUMP      :0.0 ml
21JAN03 16:57 START THERAPY
21JAN03 16:57 AIR FILTER ? :YES
21JAN03 16:57 RUN/PAUSE KEY
21JAN03 16:57 CON RUN OPTION
Options          :Lock OFF
Lock             :OFF
Hourly Tot       :OFF
Clear Hrly       :OFF
Dn Occlu         :HIGH
Air Sens         :OFF
Audio            :9
FWR CK           :OFF
Accept OPT       :YES/NO
21JAN03 16:57 CON Rx
Bag Vol          :9900 ml
Amt TBI          :9500 ml
Rate             :400.0 ml/hr
Time             :23:45 HH:MM
KVO Rate         :0.1 ml/hr
Done?            :YES
21JAN03 16:57 CON PRE Rx
Units            :ml
Delay            :OFF
Medlimit         :OFF
Next?            :YES
21JAN03 16:57 REPEAT Rx
21JAN03 16:56 ON PUMP
21JAN03 16:33 OFF PUMP
Patient Hx Log Used (bytes): 1693 of 61440

```

Figure 3.1 History Event Log

9. PRINT PATIENT HISTORY: To print the patient history log, connect the Data Interface Cable to the communication port on left side of the pump before selecting this command. (FOLLOW DIRECTIONS THAT ACCOMPANY THE DATA INTERFACE CABLE.)

The pump will display:

```

PRINT Pt. Hx
ACCEPT? YES/NO


```

When you select “YES,” the display will read “Printing Patient History Record.” When the printing is completed, the pump display returns to the BIOMED SETUP Menu. To abort the printing process, press the NO/CHANGE key on the pump’s keypad, and the display will return to the BIOMED SETUP Menu. If the Patient History Log stops before all data is transferred, exit and restart the “Print Pt. Hx” command.

Note: To utilize the print feature of the **Moog Curlin Infusion** pump, you must have a PC running the hyperterminal program within MS Windows™, and use the Moog Curlin **Infusion** Data Interface Cable which is purchased separately. Detailed user instructions are provided with the cable.

Clearing the History Log

10. CLEAR Hx LOG: This feature is used to clear the patient history log from the pump. When you select YES/ENTER, the pump displays:

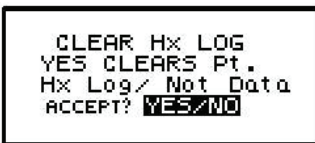


```
CLEAR Hx LOG
YES CLEARS Pt.
Hx Log/ Not Data
ACCEPT? YES/NO
```

When you select “YES,” the history log is cleared and the display returns to the BIOMED SETUP Menu. Selecting “NO” aborts the feature and also returns to the menu.

Clearing Patient Information


11. CLEAR Rx: When this command is selected, prescription information for all currently programmed therapies is cleared; however, the patient data is retained. When you select YES/ENTER the pump displays:

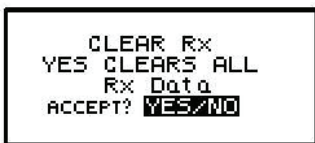

```
CLEAR Rx
YES CLEARS ALL
Rx Data
ACCEPT? YES/NO
```

When you select “YES,” all prescription programs are cleared, and the display returns to the BIOMED SETUP Menu. Selecting “NO” aborts the feature and also returns to the menu.

12. CLEAR Pt. DATA: **Selecting this feature clears all data programmed. The history log is NOT cleared.** When you select YES/ENTER at this field, the following screen displays:

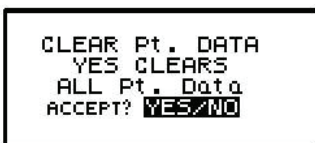

```
CLEAR Pt. DATA
YES CLEARS
ALL Pt. Data
ACCEPT? YES/NO
```

When you select “YES,” all patient data clears and the display returns to the BIOMED SETUP Menu. Selecting “NO” aborts the feature and also returns to the menu.

13. CHANGE CONTRAST: The contrast on the display screen is automatically optimized and is no longer changeable with this option.

Changing Date and Time

14. CHANGE DATE AND TIME: When this field is selected, the screen on the left appears:

Change Date and Time Screen

DATE AND TIME CLOCK: 12:15 TIME: 17:15 HH:MM DATE: 23JAN03 ACCEPT? YES/NO	DATE AND TIME CLOCK: 12hr AM/PM: AM TIME: 17:15 HH:MM DATE: 23JAN03 ACCEPT? YES/NO
---	---

This screen is used to set the system Real Time Clock in the pump and will change the date and time displayed on the Welcome Screen.

Note: The time must be reset to accommodate daylight savings time changes and also at any time the pump is moved into a different time zone.

- a. Time Format: You have the option of selecting twelve-hour or twenty-four-hour time format. Press the NO key until the correct selection appears on the field and then press the YES/ENTER key to accept the selection and move the cursor to the next field.
- b. AM/PM: The second field allows you to select AM or PM when the twelve-hour time format is selected in the previous field. If the twenty-four-hour format was selected, this field will not appear. Press the NO key until the correct selection appears on the field, and then press the YES/ENTER key to choose the selection and move the cursor to the next field.
- c. Time: Use the numeric keys to enter the correct time and then press the YES/ENTER key to accept and move the cursor to the next field.
- d. Using the numeric keys, enter the correct mm/dd/yy, using two digits for each (e.g., if the month is January, enter 01; if it is a single digit date, use the number preceded by a 0 and then enter the last two digits of the year code, 00, 01, etc.). When the date is correct, press the YES/ENTER key to accept and move to the next field.
- e. A final prompt will be given to “ACCEPT? YES/NO.” If any data needs to be corrected, use the arrow keys to locate the correct field, press NO to change the data, enter the correction(s), and then press YES/ENTER again to accept the correction. Move the cursor back to the “ACCEPT? YES” field and press YES/ENTER. This action enters the new time and date and moves to the next field in the BIOMED SETUP Menu. Pressing the NO key on this field aborts the change date and time selection and returns to the BIOMED SETUP Menu.

IMPORTANT INFORMATION

If a therapy is in progress, the date and time changing functions will not be accessible until the therapy is complete. If “***ALERT*** DATE AND TIME CHANGE NOT ALLOWED” appears you will need to go to CLEAR PATIENT DATA/Rx above date and time and choose “enter” and then “yes” to clear Pt data and Rx. Then date and time change will be allowed.

15. When BIOMED SETUP is complete, move the cursor to the “DONE?” field and press the YES/ENTER key to return to the Program or Set-Up Menu Screen.

Program

Program / BioMed Setup Menu



ENTER to Select
PROGRAM
LIBRARY
(Protocol Library Empty)

To select “PROGRAM” use the arrow keys to move the cursor to the field and press the YES/ENTER key. If no therapy is already programmed, a “Select Therapy Screen” appears prompting you to select a therapy. If a therapy has been programmed, the following screen appears:

Repeat / Resume Rx / New Program Menu



ENTER to Select
RESUME
REPEAT RX
NEW PROGRAM

There are three different ways that you can start a therapy:

- RESUME: Should be used if a therapy is interrupted. The pump picks up where it left off, and continues the therapy.
- REPEAT Rx: Should be used to hang a new bag and repeat the same therapy over again. The infusion history displayed is reset (volume infused, boluses requested, boluses received, etc). Shift Totals and Hourly Totals continue to accrue.
- NEW PROGRAM: Should be used to program a totally new Rx. All the therapy parameters in the pump are set to factory default and must be re-entered by the clinician.

The judicious choice of the option can help speed up programming. This screen gives you all three options:

Resuming a therapy

RESUME - If a therapy is interrupted and the pump is turned off or paused before completion of the infusion, you may select “RESUME” by moving the cursor to this field and pressing the YES/ENTER key. A notification screen displays, and a prompt is given to run the pump. At this time, you can continue to infuse the prescription from the exact point at which it was interrupted.

Note: Patients should be taught proper time frames or limitations for leaving their IV bags or medication reservoirs “hanging” during an interrupted state and, if indicated, to change to a new bag according to the healthcare provider’s policies and drug manufacturer’s instructions.

Repeating a Pre-Programmed Therapy

REPEAT Rx - Use this feature if an infusion is complete and you wish to repeat the same prescription over again or if a therapy is programmed and the pump is turned off before the infusion is started. Select this field by moving the cursor to “REPEAT Rx,” and press the YES/ENTER key.

If the pump has been programmed for more than one therapy and you wish to select a therapy at this time, move the cursor to “REPEAT Rx,” and press the YES/ENTER key. This action will take you to the “Select Therapy Screen” and allow you to choose which of the programs you now wish to repeat.


Note: If you are choosing a therapy that is different than the last one selected, Shift Totals and Hourly Totals will be cleared.

If only one therapy is programmed and “REPEAT Rx” is selected, the cursor will automatically advance through the prescription menus. You can stop the auto-advance function by pressing any key (except ON/OFF) and then complete the confirmation of the prescription by pressing the YES key at the remaining fields.

Note: REPEAT Rx does not allow any changes to the Pre-Prescription Menu. If changes need to be made in Pre-Prescription selections (units, concentration, etc.) select “New Program” and reenter the entire prescription.

The lock settings affect the use of “REPEAT Rx.” Lock 1 allows REPEAT with changes to the prescription screen only. Lock 2 allows REPEAT but does NOT allow the user to make changes in the prescription. Lock 3 does not allow REPEAT. (See lock setting table.)

If a therapy has not completed and the “REPEAT Rx” function is selected, the following screen will display:

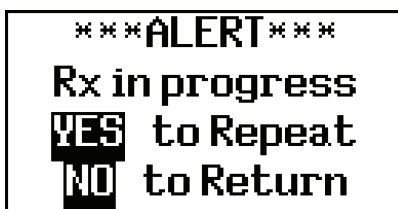


If the current infusion is to be discontinued and a new IV bag started, press YES/ENTER to repeat. If the current infusion is to continue, press NO, and you will be prompted to resume the therapy.

How to go to “New Program”

NEW PROGRAM—If the cursor is moved to this field and the YES/ENTER key is pressed, the display changes to allow you to select which therapy you wish to program:

Selecting a Therapy

Select Therapy Screen

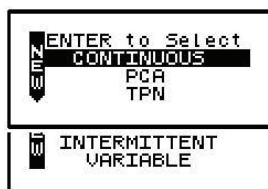

This screen displays only those therapies enabled from the BIOMED SETUP Menu. Move the cursor to the appropriate therapy and select it by pressing the YES/ENTER key. If only one therapy is enabled, this screen will not display, and the pump will automatically move to the first screen for specific therapy programming.

Entering or Changing Data Fields

When prescription data fields appear, use the numeric keys to enter specific patient prescription information and, when correct, press the YES/ENTER key to accept. If a field on the menu is not “accepted” by the program, a context sensitive Alert prompt will appear and the cursor returns you back to the fields that require the change, to help you resolve the problem.

The following screen is the general alert prompt:



Each individual therapy will now be considered separately in this manual and will have its own chapter heading.

Programming a 6000 CMS pump with Protocol Library Safety System (PLSS) Version 2 program

Refer to Protocol Library Safety Software manual, part number 360-7042

YES key is used to verify or enter

NO key is used to change, erase or leave screen

Turn the pump on, watch self test and power source bar.

Set tubing must be installed for pump to run.

Press the \uparrow and \downarrow keys to scroll through the menu, and press the **YES** key to accept/confirm a field entry and the **NO** key to reject/change a field entry.

Note: Library therapies may have soft or hard limits set for various values.

```

ENTER to Select
PROGRAM
LIBRARY
(Protocol Library Empty)
    
```

```

ENTER to Select
LIBRARY
(Protocol Library Empty)
BIOMED SETUP
    
```

For PCA therapies: Soft limit settings may be set for Max Basal Rate, Max Patient Bolus Amt, and Min Bolus Interval. Hard limits may be set for Max Load Dose.

Note: A warning screen will display if a value is entered that is out of range of the soft limit setting(s) for a downloaded therapy. The screen will ask the user to override the soft limit value.

Pressing **YES** will accept the new value, example below;

```

***ALERT***
VALUE ↑ LIMIT
5      ↑ 0.4
OVERRIDE? YES/NO
    
```

Programming a Pump with a Protocol Library

Note: The ability to program, resume, or repeat a Library Protocol is restricted to the **LIBRARY** menu option.

Press ↓ to scroll down to the **LIBRARY** function.

Highlight **LIBRARY**, and then press **YES**, the following screen appears:

Resume/Repeat Rx/New Program Menu



Note: The “RESUME” and “REPEAT Rx” options may not appear if a therapy has not been previously programmed or has run to completion.

There are three different ways that you can start a therapy:

- **RESUME:** Should be used if a therapy is interrupted. The pump picks up where it left off, and continues the current therapy.
- **REPEAT Rx:** Should be used to hang a new bag and repeat the same therapy over again. The infusion history displayed is reset (volume infused, boluses requested, boluses received, etc). Shift Totals and Hourly Totals continue to accrue.
- **NEW PROGRAM:** Should be used to program a totally new Library Rx.

Programming a New Library Therapy

NEW PROGRAM—If the cursor is moved to this field and the **YES/ENTER** key is pressed, the display changes to allow you to select which Library therapy you wish to program:

1. If the desired Category/Care Area is not already listed on the first screen, select **YES** to answer “Page Down.” This will bring up additional screen(s) with additional choices.
2. If Category/Care Area is already listed on first screen press ↓ to scroll down to the choice you wish to select.
3. When the Category/Care Area that you wish to choose is highlighted, press **YES** to select that option.
4. If the Library Therapy is not already listed on the first screen select **YES** to the right of “Page Down.” This will bring up additional screen(s) with additional choices.
5. If the Library Therapy is already listed on first screen press ↓ to scroll down to choice you wish to select.
6. When the Library Therapy that you wish to choose is highlighted, press **YES** to select that option.
7. Press **YES** again to **ACCEPT** this Library Therapy selection.

8. An optional ADVISORY <message> may appear, Press YES to continue or NO to return to main LIBRARY screen.
9. To use the Library Therapy as pre-programmed in the pump, select **YES** in each highlighted area to accept the information. To change any of the preselected drug therapy options when highlighted, press **NO** and enter correct information.

Note: Some options may be “hard locked” so that changes cannot be made or there may be preset limits that cannot be exceeded for safety.

Note: **YES** must be selected after each field when flashing to be accepted, if **YES** is not pressed after each field the pump will say: “***ALERT*** FIELD NOT ACCEPTED” and will not allow you to continue programming.

Continue through each field until the RUN to START screen appears after selecting done.

11. In the RUN to START screen, you have four options:

RUN to START	to begin the infusion
NO to REVIEW	to check orders and make changes
OPTIONS	to access additional features such as air sensor, down occlusion setting, audio
PRIME	to prime the set tubing via the pump

12. In the RUN to START screen choose **NO** to review, to validate therapy with another clinician. Press **YES** at each highlighted field to confirm choices, **NO** to change. (Note this should be done prior to priming the set tubing on the pump)
13. When you reach the RUN to START screen for the second time you can now prime the set tubing.

To prime the set tubing via the pump press the PRIME button at the bottom of the pump, follow on-screen instructions and hold the PRIME key down. Keep the button depressed until you are finished priming. Release the PRIME button to stop and press **YES** to **EXIT** to return to the RUN screen.

14. You can now begin your infusion by pressing **RUN to START**.

Resuming a Library Therapy

RESUME—If a therapy is interrupted and the pump is turned off or paused before completion of the infusion, you may select “RESUME” by moving the cursor to this field and pressing the YES/ENTER key. A notification screen displays, and a prompt is given to run the pump. At this time, you can continue to infuse the prescription from the exact point at which it was interrupted.

Note: Patients should be taught proper time frames or limitations for leaving their IV bags or medication reservoirs “hanging” during an interrupted state and, if indicated, to change to a new bag according to the healthcare provider’s policies and drug manufacturer’s instructions.

Repeating a Pre-Programmed Library Therapy

REPEAT Rx—Use this feature if an infusion is complete and you wish to repeat the same prescription over again or if a therapy is programmed and the pump is turned off before the infusion is started. Select this field by moving the cursor to “REPEAT Rx,” and press the YES/ENTER key.

If the pump has been programmed for more than one therapy and you wish to select a therapy at this time, move the cursor to “REPEAT Rx,” and press the YES/ENTER key. This action will take you to the Select Therapy Screen and allow you to choose which of the programs you now wish to repeat.


Note: If you are choosing a therapy that is different than the last one selected, “Shift Totals” and “Hourly Totals” will be cleared.

If only one therapy is programmed and “REPEAT Rx” is selected, the cursor will automatically advance through the prescription menus. You can stop the auto-advance function by pressing any key (except ON/OFF) and then complete the confirmation of the prescription by pressing the YES key at the remaining fields.

Note: REPEAT Rx **does not allow any changes to the Pre-Prescription Menu.** If changes need to be made in Pre-Prescription selections (units, concentration, etc.) select “New Program” and reenter the entire prescription.

The lock settings affect the use of “REPEAT Rx.” Lock 1 allows REPEAT with changes to the prescription screen only. Lock 2 allows REPEAT but does NOT allow the user to make changes in the prescription. Lock 3 does not allow REPEAT. (See lock setting table.)

If a therapy has not completed and the REPEAT Rx function is selected, the following screen will display:

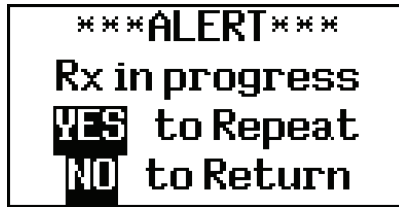

If the current infusion is to be discontinued and a new IV bag started, press YES/ENTER to repeat.

If the current infusion is to continue, press NO, and you will be prompted to resume the therapy.

Continuous Therapy

Continuous Therapy is designed to allow a constant programmed rate of infusion.

Begin programming a Continuous Therapy by selecting “Continuous” from the Select Therapy Screen. The following display screen appears and requires you to enter information needed to program this therapy:

Prescription Menus

Continuous Pre-Prescription Menu Screen

Z000 UNITS: mL DELAY: OFF Medlimit: OFF NEXT? YES	UNITS: mg CONCEN: 0.1 mg/ml DELAY: ON Medlimit: OFF	UNITS: mcg CONCEN: 0.1 mcg/ml DELAY: ON Medlimit: ON
	NEXT? YES	NEXT? YES

These menu items allow you to customize the program to match each individual prescription. The pump's default settings are displayed in the **left** screen. Each field is individually explained below.

Note: Some of the lines are abbreviated to fit on the display screen; the full descriptions appear below.

1. **UNITS:** Allows you to select from units of mL, mg, or mcg to match the patient's prescription. The pump will allow a volume range of 0.1 to 400 mL/hr; therefore, if mg or mcg is selected, the concentration requested in the next field will determine the upper and lower programming limits for mg and mcg. Use the NO/CHANGE key until the desired units appears on the display, and then press the YES/ENTER key to accept and move to the next field.
2. **CONCEN (Concentration):** This field is active whenever mg or mcg is selected in the “units” field and requires data entry of the concentration of mg or mcg per mL. When the correct data is entered using the numeric keys, press YES/ENTER to accept and move the cursor to the next field. (If mL is chosen in the “units” field, this line will not display.) The range for this field is 0.1 to 999.
3. **DELAY Start:** This field allows you to delay the start of the infusion to the programmed time from one minute up to twenty-four hours in the future. If the infusion is **NOT** to be delayed, accept the “OFF” setting by pressing the YES/ENTER key. The cursor will move to the


next field. If the infusion **IS** to be delayed until a later time, press the NO/CHANGE key. The delay start will change from “OFF” to “ON.” Press the YES/ENTER to accept and the infusion start time screen will appear.

The time format (12 hr AM/PM or 24 hr Military) selected in the BIOMED SETUP Menu will apply in this table; and, if military was selected, the “AM/PM” field will not appear here. If the entire infusion start screen is to be aborted, press the NO key at the “ACCEPT” field. When the start time and date are entered and the data is accepted after the YES/ENTER key is pressed, the infusion is set to start at the programmed time. The display then returns to the next field in the Pre-Prescription Menu.

After the entire prescription program is accepted, if the pump is turned off and back on, press YES at “PROGRAM” and YES to “REPEAT Rx.” The cursor will automatically scroll through the program. No changes will be allowed in the Pre-Prescription Menu during this time, thus maintaining the originally programmed Delay Infusion Start time. To change the Delay Infusion Start time, press the NO/CHANGE key when the Run Options Screen is displayed. This will allow the clinician to go back to review the program and at this point a change can be made to the delay start time.

Note: If the pump is not connected to the patient’s access site at this time, the patient or caregiver should be taught proper techniques to access the infusion site before the scheduled start time.

When the RUN key is pressed, the green and yellow LED lights will flash to indicate that the pump will automatically start when the delay start countdown period is up. During the delay start time, the following screen appears:

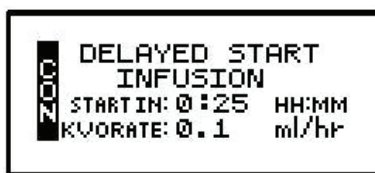

If the KVO rate is set to “0,” the pump will beep ten minutes prior to the beginning of the Continuous Therapy start time. To stop the audible beep, press the SILENCE key.

IMPORTANT INFORMATION

During the delay start period, if the KVO rate is set to “0,” no fluid is infusing. Be sure the patient’s access device is patent and has been flushed per facility policy to maintain patency during this time period. If a KVO rate is programmed, then that KVO rate will be used in the Delay Start period.

4. MedLMTS™: Allows the programmer to set MedLMTS™ limits into the prescription settings of the pump. If there is no MedLMTS™ prescription, select “OFF” and press YES/ENTER to accept. The cursor will move to the next field. If a MedLMTS™ prescription has been written, press the NO key to select “ON” and press the YES/ENTER key to accept. The following screen will appear to program the MedLMTS™ parameters:

Continuous MedLMTS™ Limit Screen

The screen displays 'medLIMITS SETUP' at the top. On the left, the word 'CON' is written vertically. The main area shows 'MAX RATE: 2' followed by a blacked-out box, with 'mL/hr' below it. At the bottom, it says 'ACCEPT? YES/NO'.

- MaxRATE (Maximum Rate) limits: Use the numeric keys to enter the upper limits of the infusion rate. When the maximum MedLMTS™ rate is entered correctly, press YES/ENTER and the cursor moves to the next field. The range for this field is from 0.1 to 400 mL/hr.
- ACCEPT?: If there are any changes to be made, use the up or down arrows to move to the desired field, use the numeric keys to enter the corrected data, and press YES/ENTER to accept the new values. Move the cursor back to the “ACCEPT?” field and enter YES/ENTER. The MedLMTS™ Limit is now entered, and the display returns to the Pre-Prescription Screen. If the entire MedLMTS™ Limit Screen is to be aborted, press NO to return to the Pre-Prescription Screen.

- NEXT?: If there are any changes to be made, use the Up or Down arrow keys to locate the field to be changed, use the numeric keys to enter the corrected information, and press YES/ENTER to accept the new data. If all the fields are acceptable, move the cursor to the “NEXT?” field and press the YES/ENTER key to move on to the next screen.

At the following screen, the prescription information is entered to complete the programming of the pump for a Continuous Therapy. Note that the Screen Identifier, at the left side of the display, gives the abbreviation “CON” for Continuous Therapy and has a down arrow configuration, which indicates that there are more than four fields of text or lines of information to be considered.

Continuous Prescription Menu

The screen is divided into three main sections. The top section has three identical boxes, each containing: 'BAG Vol: 0.0 ml', 'AmtTBI: 0.0 mg', 'RATE: 0.0 mg/hr', and 'TIME: 0:00 HH:MM'. The bottom section contains 'KVO RATE: 0.1 mL/hr' and 'DONE? YES'. On the left, the word 'CON' is written vertically next to a down arrow.

- BAG Vol (IV Bag Volume): Use the numeric keys to enter the amount of fluid in the IV Bag or Medication Reservoir (include the overfill volume).

Note: The volume used for priming via the pump or for KVO is deducted from the Bag Volume. Therefore, Bag Volume should be equal to or greater than the Amount To Be Infused to start the infusion.

When the correct amount has been entered, press the YES/ENTER key to accept this field and

move on to the next. The range for this field is 1 to 9999.

2. **Amt TBI (Amount To Be Infused):** Use the numeric keys to enter the actual amount of infusion to be delivered excluding any KVO amounts. The “Units” selected in the prescription menu will automatically appear in this field, whether mL, mg, or mcg. When the amount is correctly entered, use the YES/ENTER key to accept this field and to move to the next. (If the Amt TBI entered is greater than Bag Volume, the pump will beep once. A message of “AMT TBI GREATER THAN BAG VOL” will appear to remind you to recheck and change the number before the YES/ENTER key will accept the value and move to the next field.) The range for this field is 1 to 9999.
3. **RATE:** The **Moog Curlin Infusion** pump can deliver from 0.1 mL to 400 mL per hour. Use the numeric keys to enter the prescribed rate. If the time of the infusion is entered first, the pump will automatically calculate the rate. When this field is correct, use the YES/ENTER key to accept the rate, and move to the next field.
4. **TIME:** Displays the time required to complete the infusion. When the pump is given the Amount To Be Infused and the RATE at which to infuse, it will automatically calculate the Time and display it here. If the time is not acceptable, press the NO key to edit this field, and enter the new TIME with the numeric keys. (Remember, by changing the time of the infusion, a new RATE will be calculated and will have to be accepted before exiting this menu. The “Amount To Be Infused” will remain constant.) When the TIME field is satisfactory, press the YES/ENTER key and move to the final field. The range for this field is 0:01 to 288:00 HH:MM.

Note: The time calculated by the pump may be less than a minute; however, one minute is the minimum entry and display value.


5. **KVO Rate (Keep Vein Open Rate):** This field requires data entry using the numeric keys to set the rate at which fluid will be delivered in a “Keep Open” state. The pump defaults to a rate of 0.1 mL/hour, can be set from 0 to 10 mL/hour, but cannot be set at a rate to exceed the therapy-operating rate. When the desired KVO rate has been entered, press the YES/ENTER key to accept and move the cursor to the next field.
6. **DONE?:** If any changes are to be made, use the up or down arrow keys to locate the field to be changed, use the numeric keys to enter the corrected information, and press YES/ENTER to accept the new data. If all the fields are acceptable, move the cursor to the “DONE?” field and press the YES/ENTER key to move on to the next screen.

**IMPORTANT
INFORMATION**

Before the infusion is started, it is important to consider a few remaining issues, such as checking the administration set to be sure it was primed or purged of all air, or perhaps the pump may need to be put into a security lock setting for additional safety.

The next screen that appears when the programming is complete offers four choices:

Run Options Screen

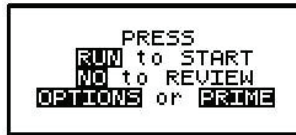

Note that you do not need to answer yes or no questions or enter data with the numeric keys as previously requested. This screen requires only that you press the corresponding key on the keypad to begin the requested function. The following explains each selection beginning with prime:

Prime

The IV bag and administration set should be inspected and be free of all air and air bubbles before they are connected to the patient's access site. Priming may have been done by gravity when the administration set was first connected to the bag (see Chapter 1, "Preparing Medication for Infusion"). However, if it was not done then or if there are any remaining air bubbles to be removed, the pump's PRIME function will assist in this process. To prime the set by using the pump, follow the directions below.

Note: The lock setting may restrict the ability to prime using the pump.

1. Press and release the PRIME key at the Run Options Screen.
2. The following screen appears:

Prime Direction Screen



3. Disconnect the administration set from the patient's access site using aseptic protocols.

IMPORTANT INFORMATION

Priming with the set connected to the patient could result in delivery of air or overdose cause injury or even death to the patient.

4. Release any clamps on the administration set.
5. Press and hold the PRIME key. Fluid will be pumped at a rate of 300 mL/hr through the administration set as long as the PRIME key is pressed or until 6 mL is pumped. The pump will then beep once and require repressing and holding the PRIME key again until 6 more milliliters is delivered. Continue this process until all air has been purged from the IV bag and the administration set.

Note: If the bag volume is depleted during priming, a subsequent priming event cannot be initiated.


While the PRIME key is depressed, the following screen appears:



```
PRIMING
Release PRIME
key to Stop
Prime Amt: 0.0 ml
```

The amount used for the prime session is accumulated and displayed on the bottom field of this screen.

6. When priming is complete, release the PRIME key. The following screen will display:

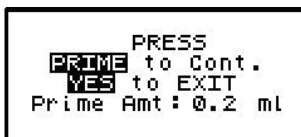

```
PRESS
PRIME to Cont.
YES to EXIT
Prime Amt: 0.2 ml
```

7. To continue priming, press the PRIME key again. If priming is complete, press YES/ENTER to exit and return to the Run Options Screen. The total amount of solution used in this priming session displays on the bottom field.

Note: The amount of fluid used in the priming process will be deducted from the Bag Volume Total but will NOT be counted in the amount infused because this amount is not to be delivered to the patient. The bag volume should still be equal to or greater than the Amount TBI before you can proceed with the infusion. If it is not, an alert will appear stating, “AMT TBI GREATER THAN BAG VOL.” You have the option to continue by pressing the YES/ENTER key or return to the Run Options Screen by pressing the NO key.

Options

Once the prescription is programmed, there are more customizing features available for each therapy. Press the OPTIONS key from the Run Options Screen to access these options. The OPTIONS menu can also be accessed while the pump is running.



Note: After the infusion has started, any changes made through the OPTIONS menu involving:

- “HRLY TOT” and
- “CLR HRLY”

will return you directly to the “ACCEPT OPT?” field.

The following screen displays the Continuous Therapy “Options” available. When you open this menu, use the up and down arrows to move the highlight bar to the desired option. Lock settings may require an access code before the option can be changed. Any new setting, except CLEAR HOURLY screen acceptance becomes effective when the OPTIONS screen is exited by pressing YES/ENTER at the “ACCEPT OPT?” field.

Continuous Therapy

 OPTIONS Lock OFF LOCK: OFF HRLY TOT: OFF CLR HRLY: OFF	1 2 3 ON ON
 DN Occlu: LOW AIR SENS: 0.5 ml AUDIO: 9 PWKCK: OFF ACCEPT OPT? YES/NO	HIGH 2.0 OFF 0.1 1 - 9 ON


Note: The pump's default settings are indicated above in the **left** screen.

1. **LOCK:** When the lock is set to “OFF ” the clinician has access to all of the pump functions. After the pump is completely programmed, change the lock to meet the security needs of this therapy. There are four choices: “OFF,” “1,” “2,” or “3.” Each level higher limits the available options. To change the lock setting, press the NO/CHANGE key until the correct lock setting appears. Then, press YES/ENTER key to accept the new lock selection. Once the pump has been set into a locked condition, any subsequent request to change the lock will require a clinician access code. (See “Clinical Information and Access Code” chapter for access code information, lock setting table and available features for each lock setting.)
2. **HRLY TOT (Hourly Totals):** To view hourly infusion totals press the NO/CHANGE key to change OFF to ON and then press YES/ENTER. The following screen will appear:

Hourly Totals Screen


 HR VOL-ml 10 _____ 11 _____ 124 0.0	AMT-mg _____ _____ _____ _____	AMTmcg _____ _____ _____ _____
--	---	---

The screen displays hourly infusion delivery for each hour (00-23) of the last 24 hours of therapy in the unit selected.

- The highlighted field has a marker  indicating the current hour in progress.
- Use the up arrow key to scroll back through the last 24 hours.
- If the infusion was not running, a series of ----- will appear for that hour.
- To exit this screen, press the NO/CHANGE key.


3. CLR HRLY (Clear Hourly Totals): To clear Hourly Totals press the NO/CHANGE key to change OFF to ON and then press YES/ENTER. The following screen will appear:

Clear Hourly Totals Screen

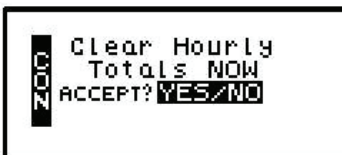

- To clear the Hourly Totals, press the YES/ENTER key. All Hourly Totals will be cleared and the cursor will return to the Options Menu.
 - To maintain the Hourly Totals, press the NO/CHANGE key. Hourly Totals will be maintained and the cursor will return to the Options Menu.
 - A lock level 2 or 3 will require an access code entry before allowing hourly total reset.
4. TitrRx (MedLMTS™ Prescription): If this feature was **not** selected in the Pre-Prescription Menu, it will not appear on the Options Menu. If a MedLMTS™ prescription was programmed and if the therapy is to be Medlimitd now, select “ON” and then press the YES/ENTER key to accept and to go to the Continuous MedLMTS™ menu:

Continuous MedLMTS Menu

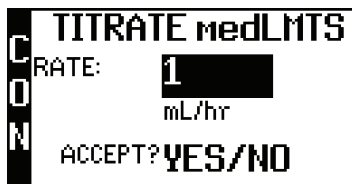

This screen allows the user to change the rate within the predefined prescription limits.

- RATE: Using the numeric keys, enter the new desired rate. When the new rate has been entered, press the YES/ENTER key to move to the next field. If the MedLMTS™ option is used to change the rate, the new rate will then appear on the run screen and on the prescription menu whenever it is reviewed.
 - ACCEPT? YES/NO: Use the Up/Down arrows to move to a field and change the data entered at this time. To abort this screen entirely, press NO at this prompt, and the display will return to the options menu. If the rate is correct, press the YES/ENTER key, and the new rate will automatically be accepted. The display returns to the Options Menu when this screen is exited.
 - New values will be in effect when the Options Menu is exited.
5. DN Occlu (Down Occlusion): This feature allows the pump to sense an occlusion pressure in the line between the pump and the access site— e.g., a kink in the tubing between the pump and the patient, a blockage in the access site, a clamp left on the distal tubing. There are two options for this field: the “Low” setting (default setting) triggers the alarm when it

detects in-line pressures at 400 mmHg \pm 150 (8 \pm 3 psi “pounds per square inch”), or the “High” setting alarms when it detects in-line pressures at 900 mmHg \pm 150 (18 \pm 3 psi). Press NO/CHANGE to change the selection. When the correct pressure setting appears, press the YES/ENTER key to accept it and move the cursor to the next field. If setting is changed, the settings will remain in effect until a new therapy is programmed.

6. AIR SENS (Air In Line Sensor): This feature allows you to select the amount of air the pump will detect in the administration set tubing before it goes into an alarm state. The choices are 0.1 mL, 0.5 mL (the default setting), 2 mL, and OFF. Press the NO/CHANGE key to change the options until the desired setting appears on the display, and then press YES/ENTER to select and move the cursor to the next line. If setting is changed, the settings will remain in effect until a new therapy is programmed.

**IMPORTANT
INFORMATION**

Whenever the Air-In-Line Sensor is set to OFF, Moog Curlin Infusion recommends the use of an air-eliminating filter.

When the Air-In-Line Sensor is set to “OFF” and the RUN key is pressed at the Run Options Screen, the alert message below will appear on the screen. It will also appear if the Air-In-Line is changed to “OFF” from the OPTIONS menu when an infusion is in progress and the AIR SENS field is accepted.


Air-In-Line Sensor Off Message Screen



This message alerts you to the Air-In-Line Sensor being set to “OFF.” The message will require answering YES or NO to whether the administration set being used has an air-eliminating filter. If you press the YES key to indicate that there is an air-eliminating filter in use, the Air-In-Line Sensor will be completely disabled and the pump will start the infusion.

If you press the NO key to indicate a filter is not in use, the Air-In-Line Sensor will be set to detect the presence of any contiguous 2-milliliter air in the line. The alert message below will then briefly appear on the screen. If the alert occurred at the Run Options Screen, the pump will start the infusion.

Air-In-Line Sensor Set at 2mL Alert Screen

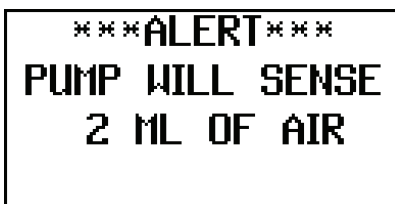

Also, when the Air-In-Line Sensor is completely disabled and the infusion goes to an “Infusion Complete” or “Empty Bag” status, the Air Sensor setting will automatically change to 2 mL.

7. **AUDIO (Audio Volume):** This feature allows you to adjust the audio volume of the pump’s alarm setting. You can enter a number from 9 (the loudest setting) to 1 (the quietest setting). When you select a number, the pump will beep at that audio level. Press the NO key and enter a new setting (1 – 9). Repeat until the desired setting is reached, and then press YES/ENTER to select it and move the cursor to the next field.
8. **PWR CK? (Power Check):** This feature allows you to periodically recheck the battery charge. It will display the active power source of the pump, whether internal batteries or external power. If the active source is the batteries, a graph will indicate approximately how much charge remains in the batteries. To access this information, select “ON” and press YES/ENTER. The Power Source Screen will display for five seconds and return to the Options Menu.
9. **ACCEPT OPT?:** If the changes made in the Options Menu are acceptable, press the YES/ENTER key. If the changes are not acceptable, use arrow keys to cursor up to any field for corrections, or abort all changes made in this menu by pressing the NO key and return to the previous screen.

There are two remaining selections to consider from the Run Options Screen, **NO** to **REVIEW**, and **RUN** to **START**.

Reviewing or Changing the Program

To review or make any changes in the prescription programming, press the NO/CHANGE key at the Run Options Screen, and the prescription screens will reappear (changes may be inhibited by lock setting). Changes to the prescription can be made before the start of the therapy; however, once a therapy has started, no changes can be made to the Pre-Prescription Screen. It will appear only for review purposes. Attempts to change the Pre-Prescription Menu during a new program review will cause an alert screen to be displayed, notifying the programmer that changes will require reprogramming of the prescription.

Starting the Infusion

RUN: When all parameters are correct and the pump has been primed properly and connected to the access site, press RUN at the Run Options Screen to start the therapy. If a delayed start is programmed, the “Delay Start Count Down Screen” will appear. If no delay start is programmed or the delay has elapsed, the pump will begin infusing and the following screen will display the status of the therapy:

Continuous Run Screen



This screen displays as long as the infusion is running. Notice the information on each field:

1. The first line displays the rate of the infusion in whichever units/hour has been selected. This information toggles with the programmed “Amount To Be Infused.”
2. The second line displays a graphic bar representing the percentage of the completed infusion. **The graphic bar will toggle with Line Pressure (“Line Press”) which displays the current patient side (downstream) pressure in the administration set in millimeter of mercury (mmHg) units.**
3. The third line displays the “Total Volume Infused” and, if mg or mcg is the selected unit, this field will toggle to display the total (mg or mcg) infused as well.
4. The last line displays the “Remaining Time” to the completion of the infusion toggled with the remaining amount to be infused.
5. In addition to the display screen giving information regarding the status of the infusion, the green LED light blinks whenever the pump is infusing.

The information on the run screen will alternate between a four-line display and a large-font display of each line of information on the screen.

Continuous Information Screens

Amt TBI 500.0 mL	Remain 1:14 HH:MM
Tot Vol 74.5 mL	Rate 400.0 mL/hr
Remain 425.5 mL	Line Pres 10 mmHg

Note: The top line of these running screens will display the name of the running protocol when programmed in the Library mode, or it will display "Unidentified" when programmed outside of the library.

Interrupting an Infusion

The infusion can be paused at any time by pressing the PAUSE key. The yellow standby LED light blinks, and the Resume/Repeat Rx/New Program Menu screen is displayed.

The screen offers you three options:

Resuming the Current Container

1. RESUME: To resume the therapy in progress from where it left off, press the YES/ENTER key at this field. The pump will return to the Run Options Screen, and you can resume the therapy.

Note: If you wish to hang a new container, move the cursor to “REPEAT Rx” (see following section).

- a. Press the RUN key to resume the therapy.
- b. Press the NO key to go back to the programming screen to make changes in the therapy program (this feature may be blocked by a security lock, thus not allowing changes to the program unless an access code is given). Also, remember that no changes are allowed in the first prescription menu if an infusion is in progress.
- c. Press the OPTIONS or PRIME keys if either of these functions is desired at this time.


Hanging a New Container

2. REPEAT Rx: To repeat the present therapy and add another IV container with the same prescription, press the YES/ENTER key at this field. If more than one therapy is programmed, a Select Therapy Screen will list all preprogrammed therapies. Select which therapy to repeat by moving the cursor to the therapy of choice and pressing the YES/ENTER key.

The pump will then display the program screen with the last programmed values of the therapy and the cursor will automatically advance through the prescription menus. You can stop the auto-advance function by pressing any key (except ON/OFF) and then complete the confirmation of the prescription manually by pressing the YES key at the remaining fields.

Note: The previously programmed AIL setting is also retained


When the program is accepted, the display returns to the above Run Options Screen. If the current therapy is still in progress when the “REPEAT Rx” function is selected, the following screen will appear:

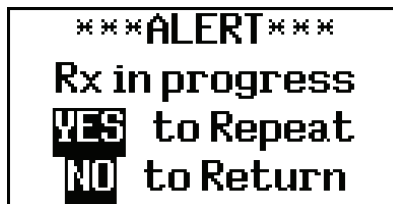

- a. This screen is a reminder that the infusion is not complete. If the therapy is to be terminated and a new IV container added, press YES/ENTER to go to the programming screen, confirm the program and proceed with the repeat infusion.
- b. If the incomplete therapy is to be continued, press NO and the Pause Menu screen will be displayed, then press YES and the Run Options Screen will be displayed. To resume the unfinished therapy, press RUN.

Entering a New Program

3. **NEW PROGRAM:** To program an entirely new prescription or a different therapy, press the YES/ENTER key at this field (the ability to change these parameters may be controlled by the lock setting). The display will return to the Select Therapy Screen if more than one therapy mode is enabled, or to the Pre-Prescription Screen of the single therapy mode. If the current container has not completely infused, the following screen will appear:
- This screen is a reminder that the infusion is not complete. If you want to return to the current container and finish infusing it, press the NO/CHANGE key to return to the Pause Menu Screen.
 - To abort the current container and enter a new program, press the YES/ENTER key. If a prescription has previously been entered for the therapy mode you wish to program, the following screen will appear:

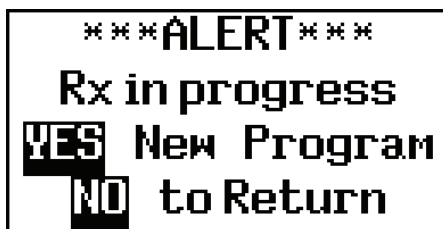

- If you want to save the current prescription, press the NO/CHANGE key to return to the Pause Menu Screen. If you wish to erase the current prescription and enter a new program, press the YES/ENTER key to proceed. All previous prescription values will be erased.

Note: Selecting “NEW PROGRAM” will also reset the OPTIONS selected to their default settings except for the LOCK and AUDIO settings which must be changed manually.

Infusion Complete

When the infusion is complete, the audio alarm will beep, and the following screen will display:

Note: To temporarily silence the alarm, press the SILENCE key.

Continuous Infusion Complete



This screen displays:

- “INFUSION COMPLETE.”
- A graphic representation (completely filled in bar graph) indicating infusion complete, or the smaller “dot” configuration (shown here) if a KVO RATE is programmed.

3. TOT Vol (Total Volume): If units are mg or mcg, this field will also toggle to give the Total Infused.
4. KVO Rate: The preset KVO rate displays.
5. Infusion Complete is considered an alarm status, and the audio alarm will beep periodically (every 3 seconds if there is no KVO, every 20 minutes if there is a KVO) to alert the user to this condition. Additionally, the LED lights blink to alert you to the status of the pump. The green light blinks when the KVO rate is infusing (if one is programmed) and the red light blinks to indicate that the “Infusion Complete” alarm exists.

Stopping the Infusion

To halt the infusion, at “Infusion Complete,” press the PAUSE key and then choose to either:

1. Turn off the pump by pressing the OFF key, or
2. Select one of the options from the Resume/Repeat Rx/New Program Menu screen:
 - a. RESUME: Is not available
 - b. REPEAT Rx: To repeat the present therapy and add another container with the same prescription, press the YES/ENTER key at this field. If more than one therapy is programmed, a Select Therapy Screen will list all preprogrammed therapies. Select which therapy to repeat by moving the cursor to the therapy of choice and pressing the YES/ENTER key.

The pump will then display the program screen with the last programmed values of the therapy and the cursor will automatically advance through the prescription menus. You can stop the auto-advance function by pressing any key (except ON/OFF) and then complete the confirmation of the prescription manually by pressing the YES key at the remaining fields.
 - c. NEW PROGRAM: To program an entirely new prescription or different therapy, press the YES/ENTER key when this field is highlighted and the display will return to the Select Therapy Screen. (The lock setting can control access to this function.)

Note: The therapy is complete; therefore, “RESUME” is not an option at this time and is not displayed.

Chapter

5



PCA Therapy

PCA or Patient Controlled Analgesia is designed for therapies requiring a continuous rate of infusion, patient controlled demand boluses, or both.

Begin programming a PCA Therapy by selecting “PCA” from the Select Therapy Screen. The following display screen will appear, requiring you to enter information needed to program this therapy.

Prescription Menus

PCA Pre-Prescription Menu Screen

 UNITS: mg CONCEN: 0.0 mg/ml ADMINRT: IV LOADDOSE: 0.0 mg	UNITS: mcg CONCEN: 0.0 mcg/ml ADMINRT: EPI LOADDOSE: 0.0 mcg	UNITS: mL ADMINRT: SQ LOADDOSE: 0.0 mL Medlimit: ON
 Medlimit: OFF NEXT? YES	Medlimit: OFF NEXT? YES	NEXT? YES

These menu items allow you to customize the program to match each individual prescription. The pump's default settings are listed in the **left** screen. Each field is individually explained below.

Note: Some of the lines are abbreviated to fit on the display screen; the full descriptions appear below.

1. **UNITS:** Allows you to select from units of mg, mcg, or mL depending on how the prescription is written. In the PCA therapy, the pump will allow rates as defined in figure 5.1; therefore, if mg or mcg is selected, the concentration requested in the next field will determine the upper and lower programming limits for mg and mcg. Use the NO/CHANGE key until the desired units appears on the display, and then press the YES/ENTER key to accept and move to the next field.
2. **CONCEN (Concentration):** This field is active whenever mg or mcg is selected in the UNITS field and requires data entry of the concentration of mg or mcg per mL. When the correct data is entered, press YES/ENTER to accept, and move the cursor to the next field. If mL is chosen in the UNITS field, this line will not display. The range for this field is 0.1 to 999.

3. ADMIN Rt (Administration Route): Allows a selection from three routes of administration: the default setting of IV (intravenous), EPI (epidural), and SQ (subcutaneous). If Epidural is selected, the Therapy Identifier Bar in the Prescription Menu will display “EPI” instead of “PCA.” Each of the routes has defined settings as illustrated in the following table:

Administration Routes						
Administration Route	Occlusion Setting mmHg (psi)	Max. Basal Rate (mL/hr)	Rate During Bolus (mL/hr)	Max. Bolus Dose (mL)	Max. Clin Dose (mL)	Max. Load Dose (mL)
Intravenous	900 or 400 (18 or 8)	9.9	125	9.9	9.9	9.9
Epidural	900 or 400 (18 or 8)	25	90	25	25	25
Subcutaneous	900 or 400 (18 or 8)	5	60	5	5	5

Figure 5.1 PCA Administration Route Default Settings

4. Load Dose (Loading Dose): Allows the clinician to program in a pre-infusion loading dose if it is prescribed. If a loading dose is not ordered, select the default setting of “0” by pressing YES/ENTER. If it is ordered, enter the dose in this field, and press the YES/ENTER key to accept and move to the next field. The maximum load dose is determined by the administration route (refer to Figure 5.1 above).

Note: The “Units” will correspond to the choice previously selected.

The loading dose will be delivered at the fixed bolus delivery rate when the RUN key is pressed to start the PCA infusion. The amount of the loading dose is included in the displayed “amount infused” and is logged into the history file. The bolus lockout time is set from the end of the loading dose. Because the loading dose is only used on initial start up of an infusion, it will revert to a value of “0” whenever the “REPEAT Rx” function is selected or when the pump is turned off, before delivery of a loading dose..

IMPORTANT INFORMATION

To terminate a loading dose in progress but save the rest of the prescription, pause the pump and select “REPEAT Rx.” When the RUN key is pressed, the prescription will restart without the loading dose.

5. Medlimit: Allows the clinician to program MedLMTS™ limits into the prescription settings of the pump. If there is no MedLMTS™ prescription, select “OFF,” and press YES/ENTER to accept, and move the cursor to the next field. If a MedLMTS™ prescription has been written, select “ON”, press the YES/ENTER key to accept, and the following screen will appear to program the MedLMTS™ parameters:

PCA Medlimit Limits Screen

P C A ✓	medLIMITS SETUP	
	MAX RATE: 0	ms/hr
	△ RATE: 0	ms
	△ TIME: 0:00	HH:MM
P C A ✓	Max BOLUS: 0	ms
	Min INT: 0	MIN
	#BOLS/hr: 0	
FI	ACCEPT? YES/NO	

- MAX RATE (Maximum Rate) limits:** Use the numeric keys to enter the upper limits of the infusion rate. When the correct maximum MedLMTS™ rate is entered, press YES/ENTER and the cursor will move to the next field. The minimum rate is 0, and the maximum rate must be equal to or less than maximum Basal Rate listed in Figure 5.1 Administration Routes table.
- ΔRate (Delta Rate):** Some prescriptions include incremental levels or *delta* limitations (e.g., “Do not raise the rate more than 1 mL in 12 hours”). If the prescription has these components, use the ΔRate field and the Δ Time field to enter those limits. Example: Enter the incremental MedLMTS™ amount level of 1 mL on this field, and press YES/ENTER to move to the next field.
- ΔTime (Delta Time):** Enter the incremental or delta time limit. (Example: Enter 12:00 hours.) If there are no incremental or delta level components to the prescription, leave these fields (ΔRate and ΔTime) set at their default settings of “0” and press YES/ENTER to move to the next field.
- Max BOLS (Maximum Bolus Dose):** Use this field to set the upper limit of the Bolus or Patient Demand dose. If there is no bolus dose, enter “0”. If a value is given for the Bolus Dose, use the numeric keys to enter it on this field, press the YES/ENTER key to accept, and move to the next field. The value entered for this field must be equal to or less than maximum Bolus Dose listed in Figure 5.1 Administration Routes table.
- MinINT (Minimum Bolus Interval):** Use this field to set the minimum amount of time allowed between bolus doses. When the correct time is entered, press the YES/ENTER key to accept, and move to the next field.
- #BOLS/hr (Number of Boluses allowed per hour):** Use this field to enter the maximum bolus/hr. When correct, press the YES/ENTER key to accept, and move to the next field.
- ACCEPT?:** If there are any changes to be made, use the up or down arrows to move to that field, use the numeric keys to enter the corrected data, and press YES/ENTER to accept the new values. Move the cursor back to the “ACCEPT?” field and press YES/ENTER. The MedLMTS™ Limits are now entered and the display returns to the

Pre-Prescription screen. If the entire MedLMTS™ Limit screen is to be aborted, press NO to return to the Pre-Prescription screen.

6. NEXT? If any field is not satisfactory, use the arrow keys to locate the field to be changed, enter the changes, press the YES/ENTER key to accept the new values and then go back to the “NEXT?” field. When all fields are satisfied with correct settings, press the YES/ENTER key to exit this screen, and move on to the next PCA Programming Menu.

At the following screen, the patient’s prescription information is entered to complete the programming of the pump for a PCA Therapy. Remember that there may be additional limits to rates and doses based on limits for the route of administration selected (See Figure 5.1).

Note that the screen identifier gives the abbreviation “PCA” for Patient Controlled Analgesic Therapy (or EPI if Epidural route is selected) and has a down arrow configuration, which indicates that there are more than four fields of text or information to be considered.

PCA Prescription Menu

BAG Vol: 0.0 ml BASL RATE: 0.0 ml/hr BOLUS: 0.0 ml BOLUS INT: 0 MIN	BAG Vol: 0.0 ml BASL RATE: 0.0 mg/hr BOLUS: 0.0 mg BOLUS INT: 0 MIN	BAG Vol: 0.0 ml BASL RATE: 0.0 mcg/h BOLUS: 0.0 mcg BOLUS INT: 0 MIN
#BOLUS/hr: 0 DONE? YES		

Note: The PCA therapy can be programmed for Basal Rate (continuous infusion) only, for Bolus (patient demand dose) only, or for both. At least one of these settings must be entered, though, before this menu can be accepted at the “DONE?” prompt.

1. BAG Vol (IV Bag Volume): Using the numeric keys, enter the amount of fluid in the IV Bag or Medication Reservoir (include the overfill volume).

Note: The volume used for priming via the pump is deducted from the Bag Volume. Therefore, Bag Volume should be equal to or greater than the Amount To Be Infused to start the infusion.

When the correct amount has been entered, press the YES/ENTER key to accept this field and move on to the next. The range for this field is 1 to 9999.

2. BASL RATE (Basal Rate): The **Moog Curlin Infusion** pump in PCA Therapy can be set for a continuous basal infusion (see Figure 5.1) Use the numeric keys to enter the prescribed rate (the “units” chosen in the Pre-Prescription Menu will appear here).

Note: If the rate falls out of the range of the pump (depending on the administration route selected), a beep will sound, and one of the following alert messages will appear:

ALERT GREATER THAN MAX FIELD LIMIT MAX=XX.X	ALERT LESS THAN MIN FIELD LIMIT MIN=XX.X
--	---

This field will not be accepted with the YES/ENTER key until the rate is adjusted to fall within pump's range. When the rate is correct, use the YES/ENTER key to accept, and move to the next field.

3. **BOLUS (Bolus Dose):** Use the numeric keys to enter the prescribed bolus (patient demand dose) and when correct, press the YES/ENTER key to accept and move on to the next field (see Figure 5.1).

Note: If a bolus dose entered falls out of the range of the pump, the above “Greater than Max Field Limit” or “Less than Min Field Limit” alert will also appear and display the limit.


4. **BOLS INT (Bolus Interval):** This field allows the clinician to enter the minimum amount of time (from 0 to 60 min) prescribed between patient-activated bolus dosing. Enter the time using the numeric keys and, when correct, press the YES/ENTER key to accept and move the cursor to the next field.
5. **# BOLS/hr (Number of Bolus Doses per hour):** Using the numeric keys, enter the number (0 to 15) of boluses allowed per hour and, when correct, press the YES/ENTER key to accept and move to the last field.
6. **DONE?:** If the programming is not done, use the up or down arrow keys to locate the field to be changed, use the numeric keys to enter the corrected information and press YES/ENTER to accept the new data. If all the fields are acceptable, at the “DONE?” prompt, press the YES/ENTER key to move on to the next screen.

IMPORTANT INFORMATION

Before the infusion is started, it is important to consider a few remaining issues, such as checking the administration set to be sure it was primed or purged of all air, or perhaps the pump may need to be put into a security lock setting for additional safety.

The following screen offers you four choices:

Run Options Screen

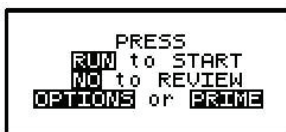

Note that you do not need to answer yes or no questions or enter data as previously requested. This screen requires only that you press the corresponding key on the keypad to begin the requested function. The following section explains each selection beginning with Prime:

Prime

The IV bag or syringes and administration set should be inspected and be free of all air and air bubbles before they are connected to the access site. Priming may have been done by gravity when the administration set was first connected to the bag. However, if it was not done then or if there are any remaining air bubbles to be removed, the pump's “PRIME” function will assist in this process. To prime the set by using the pump, follow the directions below.

Note: The lock setting may restrict the ability to prime using the pump.

1. Press and release the PRIME key at the Run Options Screen.
2. The following screen appears:

Prime Direction Screen

```

Disconnect Pt.
Release Clamps
HOLD PRIMEkey
YES to EXIT
  
```

3. Disconnect the administration set from the patient's access site using aseptic protocols.

**IMPORTANT
INFORMATION**

Priming with the set connected to the patient could result in delivery of air or overdose and may cause injury or even death to the patient.

4. Release any clamps on the administration set.
5. Press and hold the PRIME key. Fluid will be pumped at a rate of 300 mL/hr through the administration set as long as the PRIME key is pressed or until 6 mL is pumped. The pump will then beep once and require repressing and holding the PRIME key again until 6 more milliliters is delivered. Continue this process until all air has been purged from the IV bag and the administration set.

Note: If the bag volume is depleted during priming, a subsequent priming event cannot be initiated.

While the PRIME key is depressed, the following screen appears:

```

PRIMING
Release PRIME
key to Stop
Prime Amt: 0.0 ml
  
```

The amount used for the prime session is accumulated and displayed on the bottom field of this screen.

6. When priming is complete, release the PRIME key. The following screen will display:

```

PRESS
PRIME to Cont.
YES to EXIT
Prime Amt: 0.2 ml
  
```

7. To continue priming, press the PRIME key again. If priming is complete, press YES/ENTER to exit and return to the Run Option Screen. The total amount of solution used in this priming session displays on the bottom field.

Note: The amount of fluid used in the priming process will be deducted from the Bag Volume Total but will NOT be counted in the amount infused because this amount is not to be delivered to the patient. The bag volume should still be equal to or greater than the Amount TBI before you can proceed with the infusion. If it is not, an alert will appear stating, “AMT TBI GREATER THAN BAG VOL.” You have the option to continue by pressing the YES/ENTER key or return to the Run Options Screen by pressing the NO key.

Options

Once the prescription is programmed, there are more customizing features available for each therapy. Press the OPTIONS key from the Run Options Screen to access these options. The OPTIONS menu can also be accessed while the pump is running by pressing the Help/Options key.

Note. After the infusion has started, any changes made through the OPTIONS menu involving:

- “Clin DOSE”,
- “SHFT TOT”
- “HRLY TOT” and
- “CLR HRLY”

will return you directly to the “ACCEPT OPT?” field.

The following screen displays the PCA Therapy Options available. When you open this menu, use the up and down arrows to move the highlight bar to the desired option. Lock settings may require an access code before the option can be changed. Any new setting, except “CLEAR HOURLY” screen acceptance and “SHIFT TOTAL” clear screen acceptance becomes effective when the Options Screen is exited by pressing YES/ENTER at the “ACCEPT OPT?” field.

PCA Options Menu

OPTIONS Lock OFF LOCK: OFF ClinDOSE: OFF SHFTTOT: OFF	1 2 3 ON ON
HRLYTOT: OFF CLRHRLY: OFF TitRr: OFF DN Occlu: HIGH AIRSENS: 0.5 ml AUDIO: 9 PWRCK? OFF ACCEPT OPT? YES/NO	ON ON ON LOW 2.0 OFF 0.1 1 - 9 ON


Note: The pump’s default settings are indicated above in the **left** screen.

1. LOCK: When the lock is set to “OFF,” the clinician has access to all of the pump functions. After the pump is completely programmed, change the lock to meet the security needs of this therapy. There are four choices: “OFF,” “1,” “2,” or “3.” Each level higher limits the available options. To change the lock setting, press the NO/CHANGE key until the correct lock setting appears. Then, press YES/ENTER key to accept the new lock selection. Once the pump has been set into a locked condition, any subsequent request to change the lock will

require a clinician access code. (See “Clinical Information and Access Code” chapter for access code information, lock setting table and available features for each lock setting.)

2. ClinDOSE? (Clinician Dose): Allows an additional prescribed dose to be administered by a clinician at any time except during a Load Dose or Bolus Dose. The Maximum Clinician Dose is determined by the administration route (see Figure 5.1).

Clinician Dose Menu

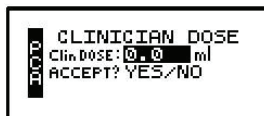

Note: If a lock setting of 1, 2 or 3 is selected, an access code will be requested. If you wish to give a clinician dose at this time, press the NO/CHANGE key to change OFF to ON and press YES key. Enter the amount of clinician dose to be delivered. Press YES to accept the clinician dose menu. The Clinician Dose will begin when the options screen is accepted and the pump is running. The following Run Screen will appear until it is completed:

Clinician Dose Run Screen



The graphic bar now depicts the clinician dose in progress and when this dose is complete, reverts back to reflecting the container in progress. **The graphic bar will toggle with Line Pressure (“L Press”) which displays the current patient side (downstream) pressure in the administration set in millimeter of mercury (mmHg) units.**

The “Volume Infused” will toggle with “Dose Infused” if mg or mcg are the selected unit. Remaining dose time will toggle with remaining dose volume to infuse.

IMPORTANT INFORMATION

When the clinician dose is complete, the pump will return to the previous running rate, and resume the PCA therapy. The lockout time for bolus dosing will restart at the end of the clinician dose to prevent a bolus from being delivered too quickly after a clinician dose.

Once the clinician dose is given, the clinician-dose field returns to its default of “0,” and any subsequent clinician doses will need to be reentered.

To Cancel a Clinician Dose:

Press the PAUSE key, to pause the pump and select RESUME. Press the OPTIONS key to access the clinician dose. Press the YES/ENTER key in the clinician dose field and change the dose amount to zero “0”. Press the YES/ENTER key to accept the change and options.

3. SHFT TOT (Shift Totals): This feature allows the clinician to track the infusion for a selected period of time, such as an 8 or 12-hour shift. To access Shift Totals, press the NO/CHANGE key to change OFF to ON and then press YES/ENTER. The following screens will appear alternately:

Shift Totals Screens

Cleared: 24JAN 21:35 Vol INF: 3.5 ml BOLUS: 0 of 0 DONE?	ELAPSED: 0:12 HH:MM Vol INF: 3.5 ml Clin DOSE: 0 DONE? EXIT
---	---

- Cleared/ELAPSED: This field displays the date and time the shift total was last cleared. It toggles with the amount of time elapsed since that clearing.
 - Vol INF/Amt INF (Volume/Amount Infused): This field displays the volume (mL) and amount (mg or mcg) infused since the shift total was cleared.
 - BOLUS/Clin DOSE (Clinician Dose): This field displays the number of clinician doses given since the total was cleared. It toggles with the number of patient activated boluses received of those requested.
 - To exit the shift total feature without clearing the totals, press the YES key at the highlighted "EXIT" field.
 - To clear the Shift Totals, press the NO key at the "EXIT" field. The field will change to read "CLEAR," press the YES key now to clear. A lock level 3 will require an access code entry before allowing the user to clear the Shift Totals.
4. HRLY TOT (Hourly Totals): To view hourly infusion totals press the NO/CHANGE key to change OFF to ON and then press YES/ENTER. Either of the following two screens will appear:

Hourly Totals Screen

HR	VOL-ml	BG	BA
19	----	--	--
20	----	--	--
21	4.8	0	0


HR	AMT-mg	BG	BA
20	----	--	--
21	4.0	0	1
22	4.6	1	0

HR	AMTmcg	BG	BA
20	----	--	--
21	----	--	--
22	4.0	1	1

The screen displays hourly infusion delivery for each hour (00-23) of the last 24 hours of therapy in the unit selected.


- The highlighted field has a marker ◀ indicating the current hour in progress.
 - "Volume" or "Amount" will show in the first column, "BG" (Bolus Given) in the second column and BA (Bolus Attempted) in the third column.
 - Use the Up arrow key to scroll back through the last 24 hours.
 - If the infusion was not running, a series of ----- will appear for that hour.
 - To exit this screen, press the NO/CHANGE key.
5. CLR HRLY (Clear Hourly Totals): To clear Hourly Totals press the NO/CHANGE key to change OFF to ON and then press YES/ENTER. The following screen will appear:

Clear Hourly Totals Screen

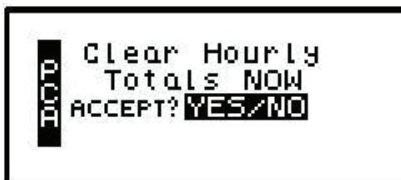

- a. To clear the Hourly Totals, press the YES/ENTER key. All Hourly Totals will be cleared and the cursor will return to the Options Menu.
 - b. To maintain the Hourly Totals, press the NO/CHANGE key. Hourly Totals will be maintained and the cursor will return to the Options Menu.
 - c. A lock level of 2 or 3 will require an access code entry before allowing Hourly Totals reset.
6. MedLMTS™ (Medication Limits): If this feature was **not** selected in the Pre-Prescription Menu, it will not appear on the Options Menu. If MedLMTS™ parameters were programmed previously, this field will appear; and if you want to titrate now, select “ON” and press the YES/ENTER key to go to the PCA Titrate MedLMTS™ menu.

PCA MedLMTS™ Screen

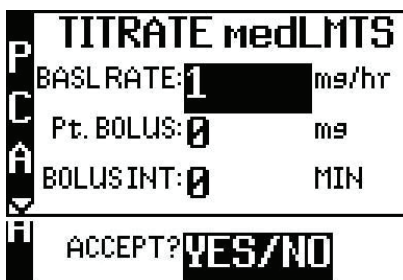

This screen allows you to change the basal rate and/or the bolus parameters within the predefined prescription limits. Whenever any value is changed using the MedLMTS™ option, the new value will appear on the prescription menu when it is reviewed. If a delta time/rate limitation was previously programmed, the rate can be increased within those parameters up to 12 times during the delta time period. If a 13th attempt is made an alert will display: “MAX CHANGES FOR PERIOD REACHED.”

- a. BASL RATE: Use the numeric keys to enter the new desired rate. When the correct rate has been entered, press the YES/ENTER key to accept the rate, and move to the next field (see Figure 5.1).
- b. BOLUS: Use the numeric keys to enter the new bolus dose and, when correct, press YES/ENTER to accept and move to the next field (see Figure 5.1).
- c. BOLS INT (Bolus Interval): Use the numeric keys to enter the new bolus time interval and, when correct, press YES/ENTER to accept and move to next field. The range for this field is 0 to 60.
- d. #BOLS /hr (number of boluses per hour): Use the numeric keys to enter the new number

- of boluses per hour and, when correct, press YES/ENTER to move to the final step in the MedLMTS™ menu. The range for this field is 0 to 15.
- e. ACCEPT? YES/NO: If necessary, use the arrow keys to move to any field and change the data entered. To abort this screen entirely, press NO at this prompt. Any data entered on this display will be aborted and the display will return to the options menu. If the parameters entered are correct, press the YES/ENTER key, and the new rate or rates will be accepted. The display returns to the Options Menu when this screen is exited.
 - f. New values will be in effect when the Options Menu is accepted.
7. DN Occlu (Down Occlusion): This feature allows the pump to sense an occlusion pressure in the line between the pump and the access site, e.g., a kink in the tubing between the pump and the patient, a blockage in the access site, or a clamp left on the distal tubing. There are two options for this field: the “Low” setting triggers the alarm when it detects in-line pressures at 400 mmHg \pm 150 (8 \pm 3 psi “pounds per square inch”), or the “High” setting (default setting) alarms when it detects in-line pressures at 900 mmHg \pm 150 (18 \pm 3 psi). Press the NO/CHANGE key to change the selection. When the correct pressure setting appears, press the YES/ENTER key to accept it and move the cursor to the next field. If setting is changed, the new settings will remain in effect until a new therapy is programmed.
 8. AIR SENS (Air In Line Sensor): This feature allows you to select the amount of air the pump will detect in the administration set tubing before it goes into an alarm state. The choices are “0.1 mL,” “0.5 mL” (the default setting), “2 mL,” and “OFF.” Press the NO/CHANGE key to change the options until the desired setting appears on the display, and then press YES/ENTER to select and move the cursor to the next line. If setting is changed, the new settings will remain in effect until a new therapy is programmed.

IMPORTANT INFORMATION

Whenever the Air-In-Line Sensor is set to OFF, Moog recommends the use of an air-eliminating filter.

When the Air-In-Line Sensor is set to “OFF” and the RUN key is pressed at the Run Option Screen, the alert message below will appear on the screen. It will also appear if the Air-In-Line is changed to “OFF” from the OPTIONS menu when an infusion is in progress and the AIR SENS field is accepted.

Air-In-Line Sensor Off Message Screen




This message alerts you to the Air-In-Line Sensor being set to “OFF.” The message will require answering yes or no to whether the administration set being used has an air-eliminating filter. If you press the YES key to indicate that there is an air-eliminating filter in use, the Air-In-Line

sensor will be completely disabled and the pump will start the infusion.

If you press the NO key to indicate a filter is not in use, the Air-In-Line Sensor will be set to detect the presence of any contiguous 2-milliliter air in the line. The alert message below will then briefly appear on the screen. If the alert occurred at the Run Options Screen, the pump will start the infusion.

Air-In-Line Sensor at 2mL Alert Screen

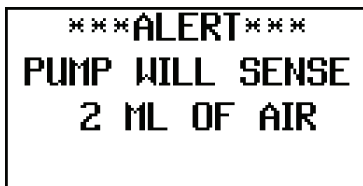

Also, when the Air-In-Line Sensor is completely disabled and the infusion goes to an “Infusion Complete” or “Empty Bag” status, the Air Sensor setting will automatically change to 2 mL.

9. **AUDIO (Audio Volume):** This feature allows you to adjust the audio volume of the pump’s alarm setting. You can enter a number from 9 (the loudest setting) to 1 (the quietest setting). When you select a number, the pump will beep at that audio level. Press the NO key and enter a new setting (1 – 9). Repeat until the desired setting is reached, and then press YES/ENTER to select and move the cursor to the next field.
10. **PWR CK? (Power Check):** This feature allows you to periodically recheck the battery charge. It will display the active power source of the pump, whether internal batteries or external power. If the active source is the batteries, a graph will indicate approximately how much charge remains in the batteries. To access this information, select “ON” and press YES/ENTER. The Power Source Screen will display for five seconds and return to the Options Menu.
11. **ACCEPT OPT?:** If the changes made in the Options Menu are acceptable, press the YES/ENTER key. If the changes are not acceptable, cursor up to any field for corrections or abort all changes made in this menu by pressing the NO key, and return to the previous screen.

There are two remaining selections to consider from the Run Options Screen, **NO** to **REVIEW** and **RUN** to **START**.


Reviewing or Changing the Program

To review or make any changes in the prescription programming, press the NO/CHANGE key at the Run Options Screen, and the prescription screens will reappear (changes may be inhibited by lock setting). Changes to the prescription can be made before the start of the therapy; however, once a therapy has started, no changes can be made to the Pre-Prescription Screen. It will appear only for review purposes. Attempts to change the Pre-Prescription Menu during a new program review will cause an alert screen to be displayed, notifying the programmer that changes will require reprogramming of the prescription.

Starting the Infusion

RUN: When the programming is complete and the pump has been primed properly and connected to the access site, press the RUN key at the Run Option screen to start the therapy. The pump will begin infusing, and except when a LOAD DOSE or CLIN DOSE is programmed, the following screen will display data on the status of the therapy:

PCA Run Screen

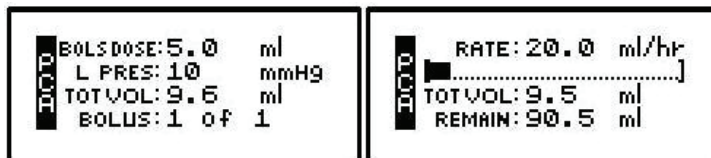

IMPORTANT INFORMATION

The amount of drug infused during the loading dose depends on the units chosen which may be in mL, mg, or mcg. A loading dose of 5 mg will infuse 50 mL if the concentration is 0.1 mg/mL.

This screen will continue to display as long as the infusion is running. Notice the information on each field:

1. The first field displays the “RATE” of the infusion toggled with the programmed Bolus Dose Amount. This field also displays “BOLUS xx (mg/mL/mcg)” whenever a bolus dose is being delivered. The programmed bolus amount displays and the BOLUS text flashes when the pump is delivering a bolus dose.
2. The second field is a graphic bar representing the percentage of the completed infusion. **The graphic bar will toggle with Line Pressure (“Line Press”) which displays the current patient side (downstream) pressure in the administration set in millimeter of mercury (mmHg) units.**
3. The third line displays the total volume “TOT VOL” infused and, if mg or mcg is the selected unit, this field will toggle to display the total mg or mcg infused “TOT INF” as well.
4. The fourth line displays the number of boluses delivered and number of boluses attempted per bag volume. If the patient attempts a bolus before the allowed amount of time set between boluses, an alert message will display “Bolus Requested,” and only the second number will increase. Also, toggling with number of boluses delivered/requested is the remaining volume left in the bag.
5. In addition to the display screen, information on the status of the infusion is provided by the three LED lights. The green LED light blinks whenever the pump is infusing. If the Basal rate is set at “0”, the yellow LED light will blink along with the green light until the bolus is actually being delivered. While the bolus is infusing, only the green light will blink.

The information on the run screen will alternate between a four-line display and a large font display of each individual line of information on the screen.

PCA Information Screens

Bolus 1 of 1	Bolus Dose 5.0 mL
Tot Vol 10.4 mL	Rate 20.0 mL/hr
Remain 90.1 mL	Line Pres 10 mmHg

Interrupting an Infusion

The infusion can be paused at any time by pressing the PAUSE key. The yellow standby LED light blinks, and the Resume/Repeat Rx/New Program Menu screen is displayed.

The screen offers the user three options:


Resuming the Current Container

Note: The top line of these running screens will display the name of the running protocol when programmed in the Library mode, or it will display “Unidentified” when programmed outside of the library.

1. RESUME: To resume the therapy in progress from where it left off, press the YES/ENTER key at this field. The pump will return to the Run Options Screen, and you can resume the therapy.

Note: If you wish to hang a new container, move the cursor to “REPEAT Rx” (see following section).

Run Options Screen

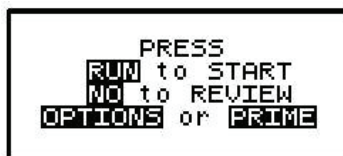

- a. Press the RUN key to resume the therapy.
- b. Press the NO key to go back to the programming screen to make changes in the therapy program (this feature may be blocked by a security lock, thus not allowing changes to the program unless an access code is given). Also, remember that no changes are allowed in the first prescription menu if an infusion is in progress.
- c. Press the OPTIONS or PRIME keys if either of these functions is desired at this time.


Hanging a New Container

2. REPEAT Rx: To repeat the present therapy and add another IV container with the same prescription, press the YES/ENTER key at this field. If more than one therapy is programmed, a Select Therapy Screen will list all preprogrammed therapies. Select which therapy to repeat by moving the cursor to the therapy of choice and pressing the YES/ENTER key.


The pump will then display the program screen with the last programmed values of the therapy and the cursor will automatically advance through the prescription menus. You can stop the auto-advance function by pressing any key (except ON/OFF) and then complete the confirmation of the prescription manually by pressing the YES key at the remaining fields.

Note: The previously programmed AIL setting is also retained


When the program is accepted, the display returns to the above Run Options Screen. If the current therapy is still in progress when the "REPEAT Rx" function is selected, the following screen will appear:

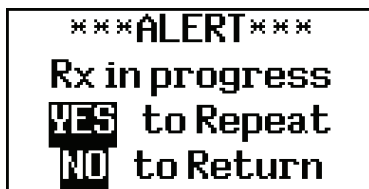


- a. This screen is a reminder that the infusion is not complete. If the therapy is to be terminated and a new IV container added, press YES/ENTER to go to the programming screen, confirm the program and proceed with the repeat infusion.

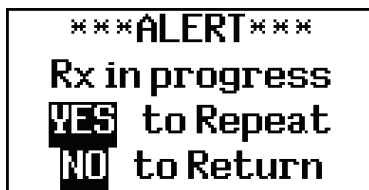

- b. This screen is a reminder that the infusion is not complete. If the incomplete therapy is to be continued, press NO/CHANGE and the Pause Menu Screen will be displayed, then press YES/ENTER and the Run Options Screen will be displayed. To resume the unfinished therapy, press RUN.
- c. If the therapy is to be terminated and a new IV container added, press YES/ENTER to repeat the infusion. The following screen will then appear, asking whether or not you wish to maintain the current Bolus or MedLMTS™ schedule.

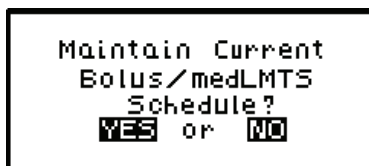

Press YES/ENTER to maintain the current schedules and continue with the programming of the pump. Press NO/CHANGE to start a new schedule. The following screen will then appear asking you to confirm the clearing of the schedules.

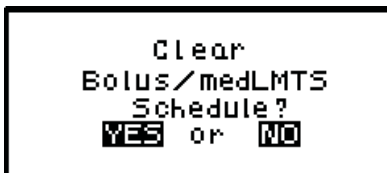


Press YES/ENTER to clear the current schedules and continue with the programming of the pump. Pressing NO/CHANGE will return you to the prior screen.

Entering a New Program

3. NEW PROGRAM: To program an entirely new prescription or different therapy, press the YES/ENTER key at this field (the ability to change these parameters may be controlled by the lock setting). The display will return to the Select Therapy Screen if more than one therapy mode is enabled, or to the Pre-Prescription Screen of the single therapy mode. If the current container has not completely infused, the following screen will appear:



- a. This screen is a reminder that the infusion is not complete. If you want to return to the current container and finish infusing it, press the NO/CHANGE key to return to the Pause Menu Screen.
- b. To abort the current container and enter a new program, press the YES/ENTER key. If a prescription has previously been entered for the therapy mode you wish to program, the following screen will appear:

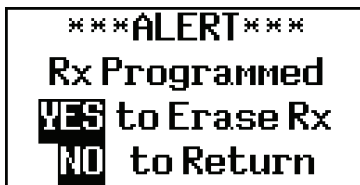

- c. If you want to save the current prescription, press the NO/CHANGE key to return to the Pause Menu Screen. If you wish to erase the current prescription and enter a new program, press the YES/ENTER key to proceed. All previous prescription values will be erased.


Note: Selecting "NEW PROGRAM" will also reset the OPTIONS selected to their default settings except for the LOCK and AUDIO settings which must be changed manually.

Infusion Complete

When the infusion is complete, the audio alarm will beep, and the following screen will display:

Note: To temporarily silence the alarm, press the SILENCE key.

PCA Infusion Complete

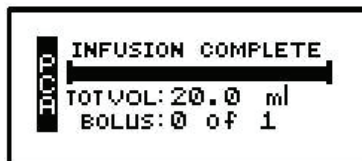

This screen displays:

1. "INFUSION COMPLETE."
2. A completely filled in bar graph indicating infusion complete.
3. "TOT Vol" (Total Volume): If units are mg or mcg, this field will also toggle to give the "Total Infused."
4. Infusion Complete is considered an alarm status, and the audio alarm will beep periodically every 3 seconds to alert the user to this condition. Additionally, the red LED light blinks to indicate that the "Infusion Complete" alarm exists.

Stopping the Infusion


At "Infusion Complete," press the PAUSE key and then choose to either:

1. Turn off the pump by pressing the OFF key, or
2. Select one of the options from the Pause screen:


Pause Menu Screen



- a. REPEAT Rx: To repeat the present therapy and add another IV container with the same prescription, press the YES/ENTER key at this field. The following screen will then appear, asking whether or not you wish to maintain the current Bolus or MedLMTS™ schedule.

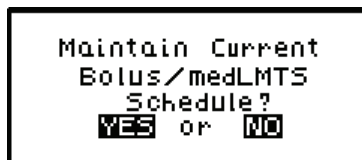

Press YES/ENTER to maintain the current schedules and continue with the programming of the pump. Press NO/CHANGE to start a new schedule. The following screen will then appear asking you to confirm the clearing of the schedules.

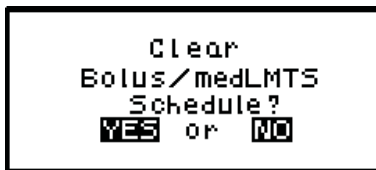

Press YES/ENTER to clear the current schedules and continue with the programming of the pump. Pressing NO/CHANGE will return you to the prior screen.

The pump will then display the program screens with the last programmed values of the therapy and the cursor will automatically advance through the prescription menus. You can stop the auto-advance function by pressing any key (except ON/OFF) and then complete the confirmation of the prescription manually by pressing the YES key at the remaining fields.

- b. NEW PROGRAM: To program an entirely new prescription or different therapy, press the YES/ENTER key when this field is highlighted and the display will return to the Select Therapy Screen. (The lock setting can control access to this function.)

Note: The therapy is complete; therefore, "RESUME" is not an option at this time and is not displayed.

TPN Therapy with Automatic Ramping

This therapy is designed to allow a level rate of infusion of parenteral nutritional products with the option of tapering at the beginning, end, or both of the infusion. It also has an early ramp-down feature.

Begin programming a TPN Therapy by selecting “TPN” from the Select Therapy Screen. The following display screen will appear, requiring you to enter information needed to program this therapy.

Prescription Menus

TPN Pre-Prescription Menu Screen


<div style="background-color: black; color: white; padding: 2px; text-align: center; font-weight: bold;">TPN</div> <div style="padding: 5px;"> DELAY: OFF NEXT? YES </div>	ON
---	----

This menu item allows you to customize the program to match each individual prescription. The pump's default setting is displayed in the **left** screen. Each field is individually explained below:

1. DELAY (Delay Start): This field allows you to delay the start of the infusion from one minute up to twenty-four hours. If the infusion is **NOT** to be delayed, select “OFF” and press the YES/ENTER to enter. The cursor will move to the next field. If the infusion **IS** to be delayed until a later time, select “ON” and press YES/ENTER to accept. The infusion start time screen will appear:

The time format (12 hr AM/PM or 24 hr Military) selected in the BIOMED SETUP Menu will apply in this table; and, if Military was selected, the AM/PM field will not appear here. If the entire delay start screen is to be aborted, press the NO key at the “ACCEPT” field. When the start time and date are entered and the data is accepted after the YES/ENTER key is pressed, the infusion is set to start at the programmed time. The display then returns to the next field in the Pre-Prescription Menu.


After the entire prescription program is accepted, if the pump is turned off and back on, press YES at “PROGRAM” and YES to “REPEAT Rx.” The cursor will automatically scroll through the program. No changes will be allowed in the Pre-Prescription Menu during this time, thus maintaining the originally programmed Delay Start time. To change the Delay Start time, press the NO/CHANGE key when the Run Options Screen is displayed:

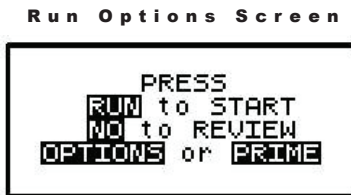

This will allow the clinician to go back to review the program and at this point a change can be made to the delay start time.

Note: If the pump is not connected to the patient’s access site at this time, the patient or caregiver should be taught proper techniques to access the infusion site before the scheduled start time.

When the RUN key is pressed, the green and yellow LED lights will flash to indicate that the pump will automatically start when the delay start countdown period is up. During the delay start time, the following screen appears:

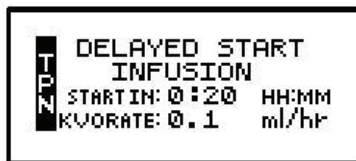

If the KVO rate is set to “0,” the pump will beep ten minutes prior to the beginning of the therapy start time. To stop the audible beep, press the SILENCE key.

IMPORTANT INFORMATION

During the delay start period, if the KVO rate is set to “0,” no fluid is infusing. Be sure the access device is patent and has been flushed per facility policy to maintain patency during this time period. If a KVO rate is programmed, then that KVO rate will be used in the Delay Start period.

2. NEXT?: If there are any changes to be made, use the up or down arrow keys to locate the field to be changed, use the numeric keys to enter the corrected information, and press YES/ENTER to accept the new data. If all the fields are acceptable, move the cursor to the “NEXT?” field and press the YES/ENTER key to move on to the next screen.

At the following screen, the prescription information is entered to complete the programming of the pump:

TPN Prescription Screen

The screenshot shows a two-part screen for TPN prescription. The top part has a vertical label 'ZDU' with a down arrow. The bottom part has a vertical label 'ZDU' with an up arrow. The fields are as follows:

BAG Vol:	0.0	ml
Vol TBI:	0.0	ml
INF RATE:	0.0	ml/hr
UP Ramp:	0:00	HH:MM
DN Ramp:	0:00	HH:MM
TOT Time:	0:00	HH:MM
KVO RATE:	0.1	ml/hr
DONE?	YES	

Note that the screen identifier gives the abbreviation “TPN” for Total Parenteral Nutrition and has a down arrow configuration, which indicates that there are more than four fields of text or information to be considered.

1. BAG Vol (IV Bag Volume): Use the numeric keys to enter the amount of fluid in the IV Bag or Medication Reservoir (include the overfill volume).

Note: The volume used for priming via the pump or for KVO is deducted from the Bag Volume. Therefore, Bag Volume should be equal to or greater than the Amount To Be Infused to start the infusion.

When the correct amount has been entered, press the YES/ENTER key to accept this field and move on to the next. The range for this field is 1 to 9999.

2. VOL TBI (Volume To Be Infused): Use the numeric keys to enter the actual volume of infusion to be delivered, excluding any KVO amounts. When the amount is correctly entered, use the YES/ENTER key to accept this field and to move to the next. (If VOL TBI entered is greater than Bag Volume, the pump will beep once. An alert message of “AMT TBI GREATER THAN BAG VOL” will appear to remind you to recheck and change the number before the YES/ENTER key will accept the value and move to the next field.) The range for this field is 1 to 9999.
3. Inf RATE (Infusion Rate or rate of the level infusion): The **Moog Curlin Infusion** pump can deliver from 0.1 mL to 400 mL per hour. Use the numeric keys to enter the prescribed rate. When this field is correct, use the YES/ENTER key to accept the rate and move to the next field. You may choose to let the pump calculate the rate by skipping this field and entering the up ramp time, down ramp, and total time.
4. UP Ramp: This optional feature allows the beginning of the infusion to be gradually tapered up to the level infusion rate over a specified period of time. Enter the up ramp time period using the numeric keys, and when correct, press the YES/ENTER key to accept this field and move to the next. Since this field is optional, “0” may be entered and accepted. The range for this field is 0:00 to 9:00 HH:MM.
5. DN Ramp (Down Ramp): This optional feature allows the infusion to be gradually tapered down over a specified period of time at the end of the level infusion. Enter the down ramp time period using the numeric keys, and when correct, press the YES/ENTER key to

accept and move the cursor to the next field. Since this field is optional, “0” may be entered and accepted. The range for this field is 0:00 to 9:00 HH:MM.

6. TotTime (Total Time): This field displays the entire time period over which the infusion will be delivered, including up and down ramp times. When the pump is given the Volume To Be Infused, the RATE at which to infuse, and the up and down ramp times, it will automatically calculate the Total Time and display it here. Or you may choose to program the Total Volume To Be Infused, Total Time and ramp times and let the pump calculate the level rate of infusion. If so, the cursor will go back to the “Inf RATE” field, display the rate, and require the operator to accept it by pressing the YES/ENTER key. If the time is not acceptable, enter the new TIME with the numeric keys. When the TIME field is satisfactory, press the YES/ENTER key and move to the final field. The range for this field is 0:01 to 288:00 HH:MM.

Note: The time calculated by the pump may be less than a minute; however, one minute is the minimum entry and display value.


7. KVO Rate (Keep Vein Open Rate): This field requires data entry using the numeric keys to set the rate at which fluid will be delivered in a “Keep Open” state. The pump defaults to a rate of 0.1 mL/hour can be set from 0 to 10 mL/hour, but cannot be set at a rate to exceed the therapy operating rate. When the desired KVO rate has been entered, press the YES/ENTER key to accept and move the cursor to the next field.
8. DONE?: If there are any changes to be made, use the up or down arrow keys to locate the field to be changed, enter the corrected information, and press YES/ENTER to accept the new data. If all the fields are acceptable, at the “DONE?” prompt, press YES/ENTER to move to the next screen.

**IMPORTANT
INFORMATION**

Before the infusion is started, it is important to consider a few remaining issues, such as checking the administration set to be sure it was primed or purged of all air, or perhaps the pump may need to be put into a security lock setting for additional safety.

The next screen that appears when the programming is complete offers four choices:

Run Options Screen

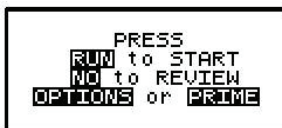

Note that you do not need to answer yes or no questions or enter data with the numeric keys as previously requested. This screen requires you to press the corresponding key on the keypad to begin the requested function. The following section explains each selection beginning with Prime:

Prime

The IV bag and administration set should be inspected and be free of all air and air bubbles before they are connected to the access site. Priming may have been done by gravity when the

administration set was first connected to the bag (see Chapter 1, “Preparing Medication for Infusion”). However, if it was not done then or if there are any remaining air bubbles to be removed, the pump’s “PRIME” function will assist in this process. To prime the set by using the pump, follow the directions below.

Note: The lock setting may restrict the ability to prime using the pump.

1. Press and release the PRIME key at the Run Options Screen.
2. The following screen appears:

Prime Direction Screen



Disconnect Pt.
Release Clamps
HOLD PRIME key
YES to EXIT

3. Disconnect the administration set from the access site using aseptic protocols

**IMPORTANT
INFORMATION**

Priming with the set connected to the patient could result in delivery of air or overdose and may cause injury or even death to the patient.

4. Release any clamps on the administration set.
5. Press and hold the PRIME key. Fluid will be pumped at a rate of 300 mL/hr through the administration set as long as the PRIME key is pressed or until 6 mL is pumped. The pump will then beep once and require repressing and holding the PRIME key again until 6 more milliliters is delivered. Continue this process until all air has been purged from the IV bag and the administration set.


Note: If the bag volume is depleted during priming, a subsequent priming event cannot be initiated.



PRIMING
Release PRIME
key to Stop
Prime Amt: 0.0 ml

While the PRIME key is depressed, the following screen appears:

The amount used for the prime session is accumulated and displayed on the bottom field of this screen.



PRESS
PRIME to Cont.
YES to EXIT
Prime Amt: 0.2 ml

6. When priming is complete, release the PRIME key. The following screen will display:
7. To continue priming, press the PRIME key again. If priming is complete, press YES/ENTER to exit and return to the Run Option Screen. The total amount of solution used in this priming session displays on the bottom field.

Note: The amount of fluid used in the priming process will be deducted from the Bag Volume Total but will NOT be counted in the amount infused because this amount is not to be delivered to the patient. The bag volume should still be equal to or greater than the Amount TBI before you can proceed with the infusion. If it is not, an alert will appear stating, “AMT TBI GREATER THAN BAG VOL.” You have the option to continue by pressing the YES/ENTER key or return to the Run Options Screen by pressing the NO key.

Options

Once the prescription is programmed, there are more customizing features available for each therapy. Press the OPTIONS key from the Run Options Screen to access these options. The OPTIONS menu can also be accessed while the pump is running.

Note: After the infusion has started, any changes made through the OPTIONS menu involving:

- “HRLY TOT” and
- “CLR HRLY”

will return you directly to the “ACCEPT OPT?” field.

The following screen displays the TPN Therapy Options available. When you open this menu, use the up and down arrows to move the highlight bar to the desired option. Lock settings may require an access code before the option can be changed. Any new setting, except “CLEAR HOURLY” screen acceptance becomes effective when the Options Screen is exited by pressing YES/ENTER at the “ACCEPT OPT?” field.

TPN Options Menu

<div style="display: flex; justify-content: space-between;"> <div style="text-align: center;"> ↑ ↓ </div> <div> OPTIONS Lock OFF LOCK: OFF HRLY TOT: OFF CLR HRLY: OFF </div> </div>	<div style="display: flex; justify-content: space-around;"> <div>1</div> <div>2</div> <div>3</div> </div> <div> ON ON </div>
<div style="display: flex; justify-content: space-between;"> <div style="text-align: center;"> ↑ ↓ </div> <div> DNR&mp: OFF DN Occlu: LOW AIR SENS: 0.5 ml AUDIO: 9 PWRCK? OFF ACCEPT OPT? YES/NO </div> </div>	<div> ON HIGH 2.0 OFF 0.1 1 - 9 ON </div>

Note: The pump’s default settings are indicated above in the **left** screen.

1. LOCK: When the lock is set to “OFF” the clinician has access to all of the pump functions. After the pump is completely programmed, change the lock to meet the security needs of this therapy. There are four choices: “OFF,” “1,” “2,” or “3.” Each level higher limits the available

options. To change the lock setting, press the NO/CHANGE key until the correct lock setting appears. Then, press YES/ENTER key to accept the new lock selection. Once the pump has been set into a locked condition, any subsequent request to change the lock will require a clinician access code. (See “Clinical Information and Access Code” chapter for access code information, lock setting table and available features for each lock setting.)


2. HRLY TOT (Hourly Totals): To view hourly infusion totals press the NO/CHANGE key to change OFF to ON and then press YES/ENTER. The following screen will appear:

Hourly Totals Screen



- a. The screen displays hourly infusion delivery for each hour (00-23) of the last 24 hours of therapy in the unit selected.
 - a. The highlighted field has a marker ◀ indicating the current hour in progress.
 - b. Use the up arrow key to scroll back through the last 24 hours.
 - c. If the infusion was not running, a series of ----- will appear for that hour.
 - d. To exit this screen, press the NO/CHANGE key.
3. CLR HRLY (Clear Hourly Totals): To clear Hourly Totals press the NO/CHANGE key to change OFF to ON and then press YES/ENTER. The following screen will appear:

Clear Hourly Totals Screen

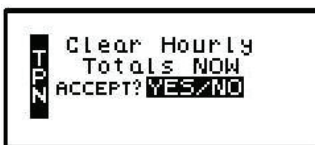

- a. To clear the Hourly Totals, press the YES/ENTER key. All Hourly Totals will be cleared and the cursor will return to the Options Menu.
 - b. To maintain the Hourly Totals, press the NO/CHANGE key. Hourly Totals will be maintained and the cursor will return to the Options Menu.
 - c. A lock level of 2 or 3 will require an access code entry before allowing Hourly Totals reset.
4. DN Ramp (Early Down Ramp): During a TPN infusion, it may be necessary or desirable to stop the level rate of infusion and begin an early down ramp process. The range for this field is 0:00 to 9:00 HH:MM. Selecting early ramp down by selecting “ON” and pressing the YES/



ENTER key when the highlight bar is on this field brings a second screen as seen below:

Note: Whatever Down Ramp time was accepted in the prescription menu will appear as the “DN Ramp” time on this screen. This time can be changed using the numeric keys or accepted as already displayed. When correct, press the YES/ENTER key. When the new time is accepted, the screen returns to the Options Menu. The Early Down Ramp phase begins when the Options Menu is exited and the pump is running.

5. DN Occlu (Down Occlusion): This feature allows the pump to sense an occlusion pressure in the line between the pump and the access site— e.g., a kink in the tubing between the pump and the patient, a blockage in the access site, or a clamp left on the distal tubing. There are two options for this field: the “Low” setting (default setting) triggers the alarm when it detects in-line pressures at 400 mmHg \pm 150 (8 \pm 3 psi “pounds per square inch”), or the “High” setting alarms when it detects in-line pressures at 900 mmHg \pm 150 (18 \pm 3 psi). Press NO/CHANGE to change the selection. When the correct pressure setting appears, press the YES/ENTER key to accept it and move the cursor to the next field.
6. AIR SENS (Air In Line Sensor): This feature allows you to select the amount of air the pump will detect in the administration set tubing before it goes into an alarm state. The choices are “0.1 mL,” “0.5 mL” (the default setting), “2 mL,” and “OFF.” Press the NO/CHANGE key to change the options until the desired setting appears on the display, and then press YES/ENTER to select and move the cursor to the next line.

**IMPORTANT
INFORMATION**

Whenever the Air-In-Line Sensor is set to OFF, Moog recommends the use of an air-eliminating filter.

When the Air-In-Line Sensor is set to “OFF” and the RUN key is pressed at the Run Option Screen, the alert message below will appear on the screen. It will also appear if the Air-In-Line is changed from the OPTIONS menu when an infusion is in progress and the AIR SENS field is accepted.

Air-In-Line Sensor Off Message Screen



```
AIR Sensor OFF
Using In Line
Air Filter?
YES/NO
```

This message alerts you to the Air-In-Line Sensor being set to “OFF.” The message will require answering yes or no to whether the administration set being used has an air-eliminating filter. If you press the YES key to indicate that there is an air-eliminating filter in use, the Air-In-Line Sensor will be completely disabled and the pump will start the infusion.

If you press the NO key to indicate a filter is not in use, the Air-In-Line Sensor will be set to detect the presence of any contiguous 2-milliliter air in the line. The alert message below will then briefly appear on the screen. If the alert occurred at the Run Options Screen, the pump will start the infusion.

Air-In-Line Sensor Set at 2mL Alert Screen

ALERT
PUMP WILL SENSE
2 ML OF AIR

Also, when the Air-In-Line Sensor is completely disabled and the infusion goes to an “Infusion Complete” or “Empty Bag” status, the Air Sensor setting will automatically change to 2 mL.

7. **AUDIO** (Audio Volume): This feature allows you to adjust the audio volume of the pump’s alarm setting. You can enter a number from 9 (the loudest setting) to 1 (the quietest setting). When you select a number, the pump will beep at that audio level. Press the NO key and enter a new setting (1 – 9). Repeat until the desired setting is reached, and then press YES/ENTER to select it and move the cursor to the next field.
8. **PWR CK?** (Power Check): This feature allows you to periodically recheck the battery charge. It will display the active power source of the pump, whether internal batteries or external power. If the active source is the batteries, a graph will indicate approximately how much charge remains in the batteries. To access this information, select “ON,” and press YES/ENTER. The Power Source Screen will display for five seconds and return to the Options Menu.
9. **ACCEPT OPT?**: If the changes made in the Options Menu are acceptable, press the YES/ENTER key. If the changes are not acceptable, use arrow keys to cursor up to any field for corrections, or abort all changes made in this menu by pressing the NO key and return to the previous screen.

There are two remaining selections to consider from the Run Options Screen: **NO** to **REVIEW** and **RUN** to **START**.


Reviewing or Changing the Program

To review or make any changes in the prescription programming, press the NO/CHANGE key at the Run Options Screen, and the prescription screens will reappear (changes may be inhibited by lock setting). Changes to the prescription can be made before the start of the therapy; however, once a therapy has started, no changes can be made to the Pre-Prescription Screen. It will appear only for review purposes. Attempts to change the Pre-Prescription Menu during a new program review will cause an alert screen to be displayed, notifying the programmer that changes will require reprogramming of the prescription.

Starting the Infusion

RUN: When all parameters are correct and the pump has been primed properly and connected to the access site, press RUN at the Run Options Screen to start the therapy. If a delayed start is programmed, the Delay Start Count Down Screen will appear. If no delay start is programmed or the delay has elapsed, the pump will begin infusing, and the following screen displays the status of the therapy:

TPN Run Screen

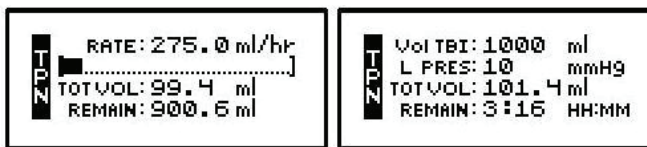

This screen will continue to display as long as the infusion is running. Notice the information on each field:

1. The first field displays the “RATE” of the infusion. If the therapy is in a ramp up or down phase, the rate will change accordingly to reflect the gradual increase or decrease in rate. The Rate toggles with the “Volume to Be Infused.”
2. The second field is a graphic bar representing the percentage of the completed infusion. It also depicts the up or down ramping phase with gradually taller or shorter lines on the bar graph. **The graphic bar will toggle with Line Pressure (“Line Press”) which displays the current patient side (downstream) pressure in the administration set in millimeter of mercury (mmHg) units.**
3. The third field displays “Total Volume Infused.”
4. The fourth field displays the remaining time to the completion of the infusion. This toggles with the “Remaining Volume.”
5. In addition to the display screen accumulating information regarding the status of the infusion, the green LED light blinks whenever the pump is infusing.

Tot Vol 107.2 mL	Remain 3:13 HH:MM
Vol TBI 1000 mL	Remain 882.3 mL
Rate 275.0 mL/hr	Line Pres 10 mmHg

Note: The top line of these running screens will display the name of the running protocol when programmed in the Library mode, or it will display "Unidentified" when programmed outside of the library.

The information on the Run Screen will alternate between a four-line display and a large-font display of each individual line of information on the screen.

TPN Information Screens

Interrupting an Infusion

The infusion can be paused at any time by pressing the PAUSE key. The yellow standby LED light blinks, and the Resume/Repeat Rx/New Program Menu screen is displayed.

The screen offers the user three options:

Resuming the Current Container

1. RESUME: To resume the therapy in progress from where it left off, press the YES/ENTER key at this field. The pump will return to the Run Options Screen, and you can resume the therapy.

Note: If you wish to hang a new container, move the cursor to “REPEAT Rx” (see following section).


- a. Press the RUN key to resume the therapy.
- b. Press the NO key to go back to the programming screen to make changes in the therapy program (this feature may be blocked by a security lock, thus not allowing changes to the program unless an access code is given). Also, remember that no changes are allowed in the first prescription menu if an infusion is in progress.
- c. Press the OPTIONS or PRIME keys if either of these functions is desired at this time.

Hanging a New Container

2. REPEAT Rx: To repeat the present therapy and add another IV container with the same prescription, press the YES/ENTER key at this field. If more than one therapy is programmed, a Select Therapy Screen will list all preprogrammed therapies. Select which therapy to repeat by moving the cursor to the therapy of choice and pressing the YES/ENTER key.

The pump will then display the program screen with the last programmed values of the therapy and the cursor will automatically advance through the prescription menus. You can stop the auto-advance function by pressing any key (except ON/OFF) and then complete the confirmation of the prescription manually by pressing the YES key at the remaining fields.

Note: The previously programmed AIL setting is also retained.

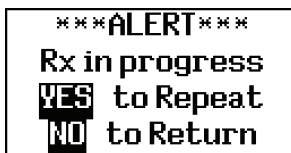

When the program is accepted, the display returns to the above Run Options Screen. If the current therapy is still in progress when the “REPEAT Rx” function is selected, the following

screen will appear:

- a. This screen is a reminder that the infusion is not complete. If the therapy is to be terminated and a new IV container added, press YES/ENTER to go to the programming screen, confirm the program and proceed with the repeat infusion.
- b. If the incomplete therapy is to be continued, press NO and the Pause Menu Screen will be displayed, then press YES and the Run Options Screen will be displayed. To resume the unfinished therapy, press RUN.


Entering a New Program

3. NEW PROGRAM: To program an entirely new prescription or different therapy, press the YES/ENTER key at this field (the ability to change these parameters may be controlled by the lock setting). The display will return to the Select Therapy Screen if more than one therapy mode is enabled, or to the Pre-Prescription Screen of the single therapy mode. If the current container has not completely infused, the following screen will appear:

- a. This screen is a reminder that the infusion is not complete. If you want to return to the current container and finish infusing it, press the NO/CHANGE key to return to the Pause Menu Screen.



- b. To abort the current container and enter a new program, press the YES/ENTER key. If a prescription has previously been entered for the therapy mode you wish to program, the following screen will appear:

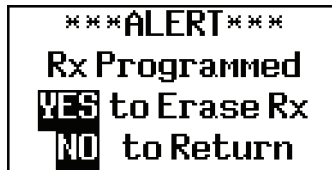

- c. If you want to save the current prescription, press the NO/CHANGE key to return to the Pause Menu screen. If you wish to erase the current prescription and enter a new program, press the YES/ENTER key to proceed. All previous prescription values will be erased.

Note: Selecting "NEW PROGRAM" will also reset the OPTIONS selected to their default settings except for the LOCK and AUDIO settings which must be changed manually.

Infusion Complete

When the infusion is complete, the audio alarm will beep, and the following screen will display:

Note: To temporarily silence the alarm, press the SILENCE key.

TPN Infusion Complete

This screen displays:



1. "INFUSION COMPLETE."
2. A graphic representation (completely filled in bar graph) indicating infusion complete, or the smaller "dot" configuration (shown here) if a KVO RATE is programmed.
3. "TOT Vol" (Total Volume): Displays the total volume infused.
4. "KVO Rate:" The preset KVO rate displays.
5. "Infusion Complete" is considered an alarm status, and the audio alarm will beep periodically (every 3 seconds if there is no KVO, every 20 minutes if there is a KVO) to alert the user to this condition. Additionally, the LED lights blink to alert you to the status of the pump. The green light blinks when the KVO rate is infusing (if one is programmed) and the red light blinks to indicate that the "Infusion Complete" alarm exists.

Stopping the Infusion

To halt the infusion at "Infusion Complete," press the PAUSE key and then choose to either:

1. Turn off the pump by pressing the OFF key, or
2. Select one of the options from the Pause Screen:

Pause Menu Screen



- a. REPEAT Rx: To repeat the present therapy and add another IV container with the same prescription, press the YES/ENTER key at this field. If more than one therapy is programmed, a Select Therapy Screen will list all preprogrammed therapies. Select which therapy to repeat by moving the cursor to the therapy of choice and pressing the YES/ENTER key.
The pump will then display the program screens with the last programmed values of the therapy and the cursor will automatically advance through the prescription menus. You can stop the auto-advance function by pressing any key (except ON/OFF) and then complete the confirmation of the prescription manually by pressing the YES key at the remaining fields.

- b. NEW PROGRAM: To program an entirely new prescription or different therapy, press the YES/ENTER key at this field and the display will return to the Select Therapy Screen. (The lock setting can control access to this function.)

Note: The therapy is complete; therefore, “RESUME” is not an option at this time and is not displayed.



Intermittent Therapy

Intermittent Therapy is designed to deliver programmed intervals and rates of specified amounts of infusates and to optionally deliver small amounts of the infusate between doses to keep the access site patent.

Begin programming an Intermittent Therapy by selecting “Intermittent” from the Select Therapy Screen. The following display screen will appear, requiring you to enter information needed to program this therapy.

Prescription Menus

Intermittent Pre-Prescription Menu Screen

Hz UNITS: mL DELAY: OFF NEXT? YES	UNITS: mg CONCEN: 0.0 mg/ml DELAY: OFF NEXT? YES	UNITS: mcg CONCEN: 0.0 mcg/ml DELAY: OFF NEXT? YES
--	---	---

These menu items allow you to customize the program to match each individual prescription. The pump's default settings are displayed in the **left** screen. Each field is individually explained below.

Note: Some of the lines are abbreviated to fit on the display screen; the full descriptions appear below.


1. **UNITS:** Allows you to select from units of mL, mg, or mcg, to match the prescription. The pump will allow a volume range of 0.1 to 400 mL/hr; therefore, if mg or mcg is selected, the concentration requested in the next field will determine the upper and lower programming limits for mg and mcg. Use the NO/CHANGE key until the desired unit appears on the display, and then press the YES/ENTER key to accept and move to the next field.
2. **CONCEN (Concentration):** This field is active whenever mg or mcg is selected in the “UNITS” field and requires data entry of the concentration of mg or mcg per mL. When the correct data is entered, press YES/ENTER to accept, and move the cursor to the next field. If mL is chosen in the “UNITS” field, this line will not display. The range for this field is 0.1 to 999.
3. **DELAY:** This field allows you to delay the start of the infusion from one minute up to twenty-four hours. If the infusion is **NOT** to be delayed, accept the default setting of “OFF” by pressing the YES/ENTER key, and the cursor will move to the next field. If the infusion **IS** to be delayed until a later time, press the NO/CHANGE key. The delay start

will change to “ON.” Now, when you press the YES/ENTER to accept, the infusion start time screen will appear:

The time format (12 hr AM/PM or 24 hr Military) selected in the BIOMED SETUP Menu will apply in this table; and, if Military was selected, the AM/PM field will not appear here. If the entire delay start screen is to be aborted, press the NO key at the “ACCEPT” field. When the start time and date are entered and the data is accepted after the YES/ENTER key is pressed, the infusion is set to start at the programmed time. The display then returns to the next field in the Pre-Prescription Menu.

After the entire prescription program is accepted, if the pump is turned off and back on, press YES at “PROGRAM” and YES to “REPEAT Rx.” The cursor will automatically scroll through the program. No changes will be allowed in the Pre-Prescription Menu during this time, thus maintaining the originally programmed Delay Start time. To change the Delay Start time, press the NO/CHANGE key when the Run Options Screen is displayed:


Run Options Screen

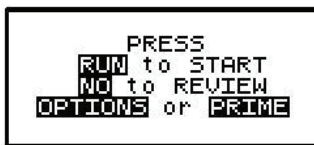

This will allow the clinician to go back to review the program and at this point a change can be made to the delay start time.

Note: If the pump is not connected to the patient’s access site at this time, the patient or caregiver should be taught proper techniques to access the infusion site before the scheduled start time.

When the RUN key is pressed, the green and yellow LED lights will flash to indicate that the pump will automatically start when the delay start countdown period is up. During the delay start time, the following screen appears:

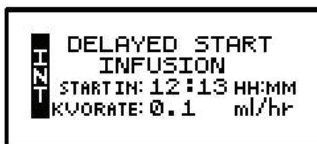

If the KVO rate is set to “0,” the pump will beep ten minutes prior to the beginning of the therapy start time. To stop the audible beep, press the SILENCE key.

IMPORTANT INFORMATION

During the delay start period, if the KVO rate is set to “0,” no fluid is infusing. Be sure the access device is patent and has been flushed per facility policy to maintain patency during this time period. If a KVO rate is programmed, then that KVO rate will be used in the Delay Start period.

4. NEXT?: If there are any changes to be made, use the up or down arrow keys to locate the field to be changed, use the numeric keys to enter the corrected information and press

YES/ENTER to accept the new data. If all the fields are acceptable, move the cursor to the “NEXT?” field and press the YES/ENTER key to move on to the next screen.

At the following screen, the prescription information is entered to complete the programming of the pump. Note that the screen identifier gives you the abbreviation “INT” for Intermittent Therapy and has a down arrow configuration, which indicates that there are more than four fields of text or information to be considered.

Intermittent Prescription Menu

INT BAG Vol: 0.0 ml AMT/DOSE: 0.0 ml DOSE RATE: 0.0 ml/hr DOSE Time: 0:00 HH:MM	BAG Vol: 0.0 ml AMT/DOSE: 0.0 mg DOSE RATE: 0.0 mg/hr DOSE Time: 0:00 HH:MM	BAG Vol: 0.0 ml AMT/DOSE: 0.0 mcg DOSE RATE: 0.0 mcg/h DOSE Time: 0:00 HH:MM
INT DOSE Freq: 0:00 HH:MM KVO RATE: 0.1 ml/hr DOSE/BAG: 0 REQ Vol: 0.0 ml TOT Time: 0:00 HH:MM DONE? YES		

1. BAG Vol (IV Bag Volume): Using the numeric keys to enter the amount of fluid in the IV Bag or Medication Reservoir (include the overfill volume).

Note: The volume used for priming via the pump or for KVO is deducted from the Bag Volume. Therefore, Bag Volume should be equal to or greater than the Amount To Be Infused to start the infusion.

When the correct amount has been entered, press the YES/ENTER key to accept this field and move on to the next. The range for this field is 1 to 9999.

2. AMT/DOSE (Amount per Dose): Using the numeric keys, enter the actual amount per dose to be delivered. The units selected in the prescription menu will automatically appear in this field whether mL, mg, or mcg. When the amount is correctly entered, use the YES/ENTER key to accept this field and to move to the next. The range for this field is 0.1 to 999.
3. DOSE RATE (Rate at which each dose is to be delivered): The **Moog Curlin Infusion** pump can deliver from 0.1 mL to 400 mL per hour. Use the numeric keys to enter the prescribed rate (again, the units chosen in the pre-infusion menu will appear here). When the correct rate displays, use the YES/ENTER key to accept the rate, and move to the next field.

Note: You may elect to skip the dose rate and enter the dose duration first to let the pump automatically calculate the dose rate.

4. DOSE DURA: Using the numeric keys, enter the amount of time over which each dose is to be infused. The range for this field is 0:01 to 99:59 HH:MM. When correct, use the YES/ENTER key to accept this field, and to move on to the next.

Note: If you have entered the dose amount and rate, the pump will automatically calculate the dose duration.

5. DOSE Freq (Dose Frequency): Represents the time from the start of one dose to the start of the next (e.g., every six hours). Use the numeric keys to enter the dose frequency, and when correct, press the YES/ENTER key to accept the cycle time and to move on to the next field. The range for this field is 0:01 to 99:59 HH:MM.
6. KVO Rate (Keep Vein Open Rate): This field requires data entry using the numeric keys to set the rate at which fluid will be delivered in a “Keep Open” state. The KVO Rate will be used between doses. The pump can be set from 0 to 10 mL/hour, but cannot be set at a rate to exceed the therapy-operating rate. When the desired KVO rate has been entered, press the YES/ENTER key to accept and move the cursor to the next field.

**IMPORTANT
INFORMATION**

When a KVO rate is programmed for the Intermittent Therapy, the patient is not disconnected between dosing. The pump automatically cycles through dosing and KVO. A single beep alerts the user to the change between dosing and KVO. If the KVO rate is set at “0,” the pump can be off between doses. The internal time clock of the pump continues to monitor the dosing cycle.

Note: The next three fields are automatically calculated based on the information already programmed and are helpful in providing additional information for managing an intermittent therapy. At each of these fields, when the information is correct, press the YES/ENTER key to accept and move to the next field.

7. DOSE/BAG: Displays the number of doses that can be delivered from the BAG Volume given. You can change this number to a lower number of doses per bag but cannot change to a greater number. When correct, press the YES/ENTER key to accept, and move to the next field. The range for this field is 1 to 200.
8. REQ Vol (Required Volume): The volume needed for the programmed number of doses, including the amount needed for KVO rates (if a KVO rate is programmed). The pump calculates this volume for you based on previously programmed information, and you cannot change the data in this field. Use the down arrow or the YES/ENTER key to move to the next field. The range for this field is 1 to 9999.
9. TOT Time (Total time): Time required to deliver the number of doses programmed. The pump calculates this time based on previously programmed information and you cannot change data in this field. Use the down arrow or the YES/ENTER key to move to the next field. The range for this field is 0:01 to 288:00 HH:MM.

Note: A DOSE is always followed by a KVO period. Therefore, after the last dose, the KVO will run until the dose frequency time has expired.


10. DONE?: If there are any changes to be made, use the up or down arrow keys to locate the field to be changed, use the numeric keys to enter the corrected information, and press YES/ENTER to accept the new data. (Remember that if a field is changed, subsequent fields will need reconfirmation to verify accuracy.) If all the fields are acceptable, move the cursor to the “DONE” field, and press the YES/ENTER key to move on to the next screen.

**IMPORTANT
INFORMATION**

Before the infusion is started, it is important to consider a few remaining issues, such as checking the administration set to be sure it was primed or purged of all air, or perhaps the pump may need to be put into a security lock setting for additional safety.

The following screen offers four choices:

Run Options Screen

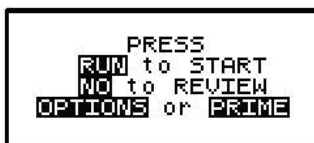

Note that you do not need to answer yes or no questions or enter data with the numeric keys as previously requested. This screen requires only that you press the corresponding key on the keypad to begin the requested function. The following explains each selection beginning with Prime:

Prime

The IV bag and administration set should be inspected and be free of all air and air bubbles before they are connected to the access site. Priming may have been done by gravity when the administration set was first connected to the bag (see Chapter 1, “Preparing Medication for Infusion”). However, if it was not done then or if there are any remaining air bubbles to be removed, the pump’s “PRIME” function will assist in this process. To prime the set by using the pump, follow the directions below.

Note: The lock setting may restrict the ability to prime using the pump.

1. Press and release the PRIME key at the Run Options Screen.
2. The following screen appears:

Prime Direction Screen



3. Disconnect the administration set from the access site using aseptic protocols.

**IMPORTANT
INFORMATION**

Priming with the set connected to the patient could result in delivery of air or overdose and may cause injury or even death to the patient.

4. Release any clamps on the administration set.
5. Press and hold the PRIME key. Fluid will be pumped at a rate of 300 mL/hr through the administration set as long as the PRIME key is pressed or until 6 mL is pumped. The pump will then beep once and require repressing and holding the PRIME key again until 6 more

milliliters is delivered. Continue this process until all air has been purged from the IV bag and the administration set.

Note: If the bag volume is depleted during priming, a subsequent priming event cannot be initiated.


While the PRIME key is depressed, the following screen appears:



```
PRIMING
Release PRIME
key to Stop
Prime Amt: 0.0 ml
```

The amount used for the prime session is accumulated and displayed on the bottom field of this screen.

6. When priming is complete, release the PRIME key. The following screen will display:

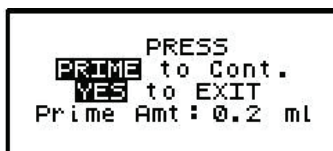

```
PRESS
PRIME to Cont.
YES to EXIT
Prime Amt: 0.2 ml
```

7. To continue priming, press the PRIME key again. If priming is complete, press YES/ENTER to exit and return to the Run Options Screen. The total amount of solution used in this priming session displays on the bottom field.

Note: The amount of fluid used in the priming process will be deducted from the Bag Volume Total but will NOT be counted in the amount infused because this amount is not to be delivered to the patient. The bag volume should still be equal to or greater than the Amount TBI before you can proceed with the infusion. If it is not, an alert will appear stating, “AMT TBI GREATER THAN BAG VOL.” You have the option to continue by pressing the YES/ENTER key or return to the Run Options Screen by pressing the NO key.

Options

Once the prescription is programmed, there are more customizing features available for each therapy. Press the OPTIONS key from the Run Options Screen to access these options. The OPTIONS menu can also be accessed while the pump is running.

Note: After the infusion has started, any changes made through the OPTIONS menu involving:

- “HRLY TOT” and
- “CLR HRLY”

will return you directly to the “ACCEPT OPT?” field.

The following screen displays the Intermittent Therapy Options available. When you open this menu, use the up and down arrows to move the highlight bar to the desired option. Lock settings may require an access code before the option can be changed. Any new setting, except “CLEAR HOURLY” screen acceptance becomes effective when the Options Screen is exited by pressing

YES/ENTER at the “ACCEPT OPT?” field.

Intermittent Options Menu		
123456789101112	OPTIONS Lock OFF LOCK: OFF HRLY TOT: OFF CLR HRLY: OFF	1 2 3 ON ON
	DN Occlu: LOW AIR SENS: 0.5 ml NXT DOSE: 0:00 HH:MM AUDIO: 9 PWRCK? OFF ACCEPT OPT? YES/NO	HIGH 2.0 OFF 0.1 1 - 9 ON

Note: The pump’s default settings are indicated above in the **left** screen.

1. LOCK: When the lock is set to “OFF” the clinician has access to all of the pump functions. After the pump is completely programmed, change the lock to meet the security needs of this therapy. There are four choices: “OFF,” “1,” “2,” or “3.” Each level higher limits the available options. To change the lock setting, press the “NO/CHANGE” key until the correct lock setting appears. Then, press YES/ENTER key to accept the new lock selection. Once the pump has been set into a locked condition, any subsequent request to change the lock will require a clinician access code. (See “Clinical Information and Access Code” chapter for access code information, lock setting table and available features for each lock setting.)
2. HRLY TOT (Hourly Totals): To view hourly infusion totals press the NO/CHANGE key to change OFF to ON and then press YES/ENTER. The following screen will appear:

Hourly Totals Screen		
123456789101112	HR VOL-ml 10 ----- 11 ----- 12 26.1	AMT-mg ----- 0.0
		AMTmcg ----- 0.0

The screen displays hourly infusion delivery for each hour (00-23) of the last 24 hours of therapy in the unit selected.

- a. The highlighted field has a marker ◀ indicating the current hour in progress.
- b. Use the up arrow key to scroll back through the last 24 hours.
- c. If the infusion was not running, a series of ----- will appear for that hour.
- d. To exit this screen, press the NO/CHANGE key.

3. CLR HRLY (Clear Hourly Totals): To clear Hourly Totals press the NO/CHANGE key to change OFF to ON and then press YES/ENTER. The following screen will appear:

Clear Hourly Totals Screen



- To clear the Hourly Totals, press the YES/ENTER key. All Hourly Totals will be cleared and the cursor will return to the Options Menu.
 - To maintain the Hourly Totals, press the NO/CHANGE key. Hourly Totals will be maintained and the cursor will return to the Options Menu.
 - A lock level of 2 or 3 will require an access code entry before allowing Hourly Totals reset.
4. DN Occlu (Down Occlusion): This feature allows the pump to sense an occlusion pressure in the line between the pump and the access site (e.g., a kink in the tubing between the pump and the patient, a blockage in the access site, or a clamp left on the distal tubing). There are two options for this field: the “Low” setting (default setting) triggers the alarm when it detects in-line pressures at 400 mmHg \pm 150 (8 \pm 3 psi “pounds per square inch”), or the “High” setting alarms when it detects in-line pressures at 900 mmHg \pm 150 (18 \pm 3 psi). Press NO/CHANGE to change the selection. When the correct pressure setting appears, press the YES/ENTER key to accept it and move the cursor to the next field.
5. AIR SENS (Air In Line Sensor): This feature allows you to select the amount of air the pump will detect in the administration set tubing before it goes into an alarm state. The choices are “0.1 mL,” “0.5 mL” (the default setting), “2 mL,” and “OFF.” Press the NO/CHANGE key to change the options until the desired setting appears on the display, and then press YES/ENTER to select and move the cursor to the next line.

**IMPORTANT
INFORMATION**

Whenever the Air-In-Line Sensor is set to OFF, Moog recommends the use of an air-eliminating filter.

When the Air-In-Line Sensor is set to “OFF” and the RUN key is pressed at the Run Option screen, the alert message below will appear on the screen. It will also appear if the Air-In-Line is changed from the OPTIONS menu when an infusion is in progress and the AIR SENS field is accepted.

Air-In-Line Sensor Off Message Screen



This message alerts you to the Air-In-Line Sensor being set to “OFF.” The message will require answering “yes” or “no” to whether the administration set being used has an air-eliminating filter. If you press the YES key to indicate that there is an air-eliminating filter in use, the Air-In-Line Sensor will be completely disabled and the pump will start the infusion.

If you press the NO key to indicate a filter is not in use, the Air-In-Line Sensor will be set to detect the presence of any contiguous 2-milliliter air in the line. The alert message below will then briefly appear on the screen. If the alert occurred at the Run Options Screen, the pump will start the infusion.

Air-In-Line Sensor Set at 2 mL Alert Screen

ALERT
PUMP WILL SENSE
2 ML OF AIR

Also, when the Air-In-Line Sensor is completely disabled and the infusion goes to an “Infusion Complete” or “Empty Bag” status, the Air Sensor setting will automatically change to 2 mL.

6. **NXT DOSE (Next Dose):** Allows the time of the next dose to be changed. The current time-remaining-until-the-next-dose is displayed. Use the numeric keys to change the time for the start of the next dose. If correct, press the YES/ENTER key to accept, and move to the next field. The range for this field is 0:00 to 24:00 HH:MM. Please refer to the Timing Rules section of this chapter for more information on bypassing the NXT DOSE delayed time.

Note: If there is a dose currently in progress and the NXT DOSE feature is used, the current dose will be terminated and the cycle will continue at the KVO rate until the time for the new next dose.

7. **AUDIO (Audio Volume):** This feature allows you to adjust the audio volume of the pump’s alarm setting. You can enter a number from 9 (the loudest setting) to 1 (the quietest setting). When you select a number, the pump will beep at that audio level. Press the NO/CHANGE key and enter a new setting (1 - 9). Repeat until the desired setting is reached, and then press YES/ENTER to select it and move the cursor to the next field.
8. **PWR CK? (Power Check):** This feature allows the user to recheck the battery charge. The display gives the active power source of the pump, whether internal batteries or external power. If the active source is the batteries, a bar graph will indicate approximately how much charge remains in the batteries. To access this information, use the NO/CHANGE key to change from the default setting of “OFF” to “ON,” and press YES/ENTER. The Power Source Screen will display for five seconds and return to the Options Menu.
9. **ACCEPT?:** If the changes made in the Options Menu are acceptable, press the YES/ENTER key. If the changes are not acceptable, use the arrow keys to cursor up to any field for corrections, or abort all changes made in this menu by pressing the NO key and return to the previous screen.

There are two remaining selections to consider from the Run Options Screen: **NO** to **REVIEW** and **RUN** to **START**.


Reviewing or Changing the Program

To review or make any changes in the prescription programming, press the NO/CHANGE key at the Run Options Screen, and the prescription screens will reappear (changes may be inhibited by lock setting). Changes to the prescription can be made before the start of the therapy; however, once a therapy has started, no changes can be made to the Pre-Prescription Screen. It will appear only for review purposes. Attempts to change the Pre-Prescription Menu during a new program review will cause an alert screen to be displayed, notifying the programmer that changes will require reprogramming of the prescription.

Starting the Infusion

RUN: When all parameters are correct and the pump has been primed properly and connected to the access site, press RUN on the Run Options Screen to start the therapy. If a delayed start is programmed, the Delay Start Count Down Screen will appear. If no delay start is programmed or the delay has elapsed, the pump will begin infusing and the following screen displays the status of the therapy:

Intermittent Run Screen

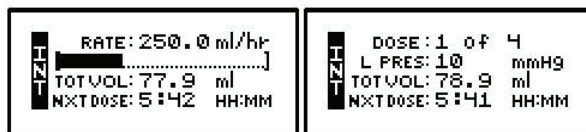

This screen continues to display as long as the infusion is in progress. Notice the information given in each field.

1. The first field displays the “RATE” of the infusion in whichever units/hour has been selected. The dose number in progress of the total number of doses programmed toggles with the infusion rate.
2. The second field is a graphic representation of the delivered amount of the **dose in progress** or that of the KVO cycle in progress. **The graphic bar will toggle with Line Pressure (“Line Press”) which displays the current patient side (downstream) pressure in the administration set in millimeter of mercury (mmHg) units.**
3. The third field toggles to give “Total Volume Infused,” “Dose Infused,” and, if units are mg or mcg, the “Total Amount Infused” also.
4. The fourth field gives the remaining time for the entire infusion (all doses in this IV bag), the remaining amount (in mg or mcg), the remaining volume, and the time the next dose is due.
5. In addition to the display screen accumulating information regarding the status of the infusion, the green LED light blinks whenever the pump is infusing.

The information on the run screen will alternate between a four-line display and a large-font display of each individual line of information on the screen.

Intermittent Information Screens

Tot Vol 30.6 mL	Remain 23:53 HH:MM
Dose 1 of 4	Dose Inf 37.5 mL
Remain 970.8 mL	Nxt Dose 5:51 HH:MM
Rate 250.0 mL/hr	Line Pres 10 mmHg

Note: The top line of these running screens will display the name of the running protocol when programmed in the Library mode, or it will display "Unidentified" when programmed outside of the library.

Interrupting an Infusion

The infusion can be paused at any time by pressing the PAUSE key. The yellow standby LED light blinks, and the Resume/Repeat Rx/New Program Menu screen is displayed.

The screen offers the user three options:

Resuming the Current Container

1. RESUME: To resume the therapy in progress from where it left off, press the YES/ENTER key at this field. The pump will return to the Run Options Screen, and you can resume the therapy.

Note: If you wish to hang a new container, move the cursor to "REPEAT Rx" (see following section).

- a. Press the RUN key to resume the therapy.
- b. Press the NO key to go back to the programming screen to make changes in the therapy program (this feature may be blocked by a security lock, thus not allowing changes to the program unless an access code is given). Also, remember that no changes are allowed in the first prescription menu if an infusion is in progress.
- c. Press the OPTIONS or PRIME keys if either of these functions is desired at this time.

Hanging a New Container


2. REPEAT Rx: To repeat the present therapy and add another IV bag or reservoir with the same prescription, press the YES/ENTER key at this field. If more than one therapy is

programmed, a Select Therapy Screen will list all preprogrammed therapies. Select which therapy to repeat by moving the cursor to the therapy of choice and pressing the YES/ENTER key.

The pump will then display the program screen with the last programmed values of the therapy and the cursor will automatically advance through the prescription menus. You can stop the auto-advance function by pressing any key (except ON/OFF) and then complete the confirmation of the prescription manually by pressing the YES key at the remaining fields.

Note: The previously programmed AIL setting is also retained.


When the program is accepted, the display returns to the above Run Options Screen. If the current therapy is still in progress when the “REPEAT Rx” function is selected, the following screen will appear:

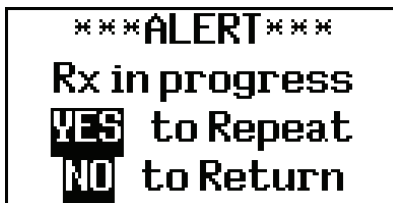

- This screen is a reminder that the infusion is not complete. If the therapy is to be terminated and a new IV container added, press YES/ENTER to go to the programming screen, confirm the program and proceed with the repeat infusion.
- If the incomplete therapy is to be continued, press NO and the Pause Menu Screen will be displayed, then press YES and the Run Options Screen will be displayed. To resume the unfinished therapy, press RUN.

Following the confirmation and acceptance of the prescription screens, a screen will appear to confirm the dosing schedule. This screen is displayed only if the previous INT infusion was terminated prior to the completion of the infusion.

I n t e r m i t t e n t D o s i n g S c h e d u l e

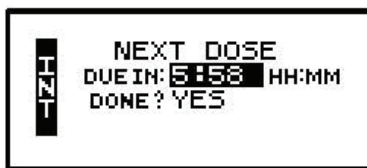

The dosing schedule of the previous container is retained and displayed on this screen. In Lock setting 0 and 1, you can change the time. In lock 2 or 3, the next dose time is displayed but requires an access code before allowing a change. When the next dose time is correct, press the YES/ENTER key. The cursor will move to the “DONE?” field, press the YES/ENTER key again and the Run Options Screen will display.

Entering a New Program

3. NEW PROGRAM: To program an entirely new prescription or a different therapy, press the YES/ENTER key at this field (the ability to change these parameters may be controlled by the lock setting). The display will return to the Select Therapy Screen if more than one therapy mode is enabled, or to the Pre-Prescription Screen of the single therapy mode. If the current container has not completely infused, the following screen will appear:

```

***ALERT***
Rx in progress
YES New Program
NO to Return
  
```

- This screen is a reminder that the infusion is not complete. If you want to return to the current container and finish infusing it, press the NO/CHANGE key to return to the Pause Menu screen.
- To abort the current container and enter a new program, press the YES/ENTER key. If a prescription has previously been entered for the therapy mode you wish to program, the following screen will appear:

```

***ALERT***
Rx Programmed
YES to Erase Rx
NO to Return
  
```

- If you want to save the current prescription, press the NO/CHANGE key to return to the Pause Menu screen. If you wish to erase the current prescription and enter a new program, press the YES/ENTER key to proceed. All previous prescription values will be erased.

Note: Selecting “NEW PROGRAM” will also reset the OPTIONS selected to their default settings except for the LOCK and AUDIO settings which must be changed manually.

Timing Rules when Interrupting Intermittent Therapies

IMPORTANT INFORMATION

Remember, this therapy is time dependent, and the following rules apply for interruptions in the therapy.

For the Intermittent Therapy the pump automatically cycles through dosing and KVO. If the KVO rate is set at “0” the pump can be off between those doses.

When the pump is idle or off, the internal time clock of the pump continues to monitor the dosing cycle and the following time considerations apply:

1. When the infusion is resumed thirty minutes or less past the scheduled “Next Dose Time,” the dose is ready for immediate delivery and no changes will be made in the original scheduling except to shorten the next KVO time between doses to comply with the dosing cycle.
2. When the infusion is resumed more than thirty minutes past the scheduled “Next Dose Time,” the next dose will be ready for delivery when the pump is turned on. All subsequent dosing will be shifted to keep the dosing cycle consistent unless the “NXT DOSE” Option is used to change subsequent dose times.

When a dose is interrupted for 30 minutes or more and a countdown appears on the pump screen, an immediate “NOW” DOSE may be programmed. There are two ways to program and deliver a “NOW” dose:

- Using the “NXT DOSE” Option: Press the Options key while the pump is running. In the Options Menu, enter the value of time of the NEXT DOSE HH:MM field. This option will deliver subsequent doses remaining in the program, and dose time will be shifted to keep the dosing cycle consistent.

- Using the REPEAT RX Option: Select Repeat Rx and press the YES/ENTER key as the program is scrolling. Confirm each field, until NEXT DOSE is displayed. Enter the value of time in the HH:MM field. This option will deliver subsequent doses until bag is empty.

3. If a dose in progress is interrupted for thirty minutes or less and the infusion is resumed, the dose in progress will resume, and the only adjustment made will be to shorten the next KVO time between doses. If a dose in progress is interrupted for more than thirty minutes and the infusion is resumed, the dose in progress will be terminated. If necessary, use the “NXT DOSE” Option to reset the next dose delivery time.
4. If the KVO cycle is interrupted before reaching the time for next dose, no changes occur in dosing time cycles.

IMPORTANT INFORMATION

If it is necessary to change the batteries during the delivery of an intermittent dose, be sure to press the PAUSE key and then turn the pump off before removing the batteries. If the pump is delivering a dose and the batteries are removed without first pausing the pump, the dose in progress will be aborted.

Infusion Complete

When the infusion is complete (all doses of this IV bag are administered and the last KVO cycle is complete), the following screen displays:

Intermittent Infusion Complete



This screen displays:

1. “INFUSION COMPLETE.”

2. A graphic representation (completely filled in bar graph) indicating infusion complete, or the smaller “dot” configuration (shown here) if a KVO RATE is programmed.
3. “TOT Vol” (Total Volume): If units are mg or mcg, this field will also toggle to give the “Total Infused.”
4. “KVO Rate:” The preset KVO rate displays.
5. Infusion Complete is considered an alarm status, and the audio alarm will beep periodically (every 3 seconds if there is no KVO, every 20 minutes if there is a KVO) to alert the user to this condition. Additionally, the LED lights blink to alert you to the status of the pump. The green light blinks when the KVO rate is infusing (if one is programmed) and the red light blinks to indicate that the Infusion Complete alarm exists.

Stopping the Infusion

To halt the infusion at “Infusion Complete,” press the PAUSE key and then choose to either:

1. Turn off the pump by pressing the OFF key, or
2. Select one of the options from the Pause Screen:



- a. REPEAT Rx: To repeat the present therapy and add another IV bag or reservoir with the same prescription, press the YES/ENTER key at this field. If more than one therapy is programmed, a Select Therapy Screen will list all preprogrammed therapies. Select which therapy to repeat by moving the cursor to the therapy of choice and pressing the YES/ENTER key.

The pump will then display the program screens with the last programmed values of the therapy and the cursor will automatically advance through the prescription menus. You can stop the auto-advance function by pressing any key (except ON/OFF) and then complete the confirmation of the prescription manually by pressing the YES key at the remaining fields.

- b. NEW PROGRAM: To program an entirely new prescription or a different therapy, press the YES/ENTER key at this field and the display will return to the Select Therapy Screen. (The lock setting can control access to this function.)

Note: The therapy is complete; therefore, “RESUME” is not an option at this time and is not displayed.


Variable Therapy

This therapy is designed to allow twenty-four specified programs of varying amounts, rates, and times of delivery.

Begin programming a Variable Therapy by selecting “Variable” from the Select Therapy Screen. The following display screen will appear, requiring you to enter information needed to program this therapy.

Prescription Menus

Variable Pre-Prescription Menu Screen

 UNITS: mL DELAY: OFF NEXT? YES	UNITS: mg CONCEN: 0.0 mg/ml DELAY: OFF NEXT? YES	UNITS: mcg CONCEN: 0.0 mcg/ml DELAY: OFF NEXT? YES
---	---	---

These menu items allow you to customize the program to match each individual prescription. The pump's default settings are displayed in the **left** screenshot above. Each field is individually explained on following pages.


Note: Some of the lines are abbreviated to fit on the display screen; the full descriptions appear below.

1. **UNITS:** Allows you to select from units of mL, mg, or mcg, to match the prescription. The pump will allow a volume range of 0.1 to 400 mL/hr; therefore, if mg or mcg is selected, the concentration requested in the next field will determine the upper and lower programming limits for mg and mcg. Use the NO/CHANGE key until the desired units appears on the display, and then press the YES/ENTER key to accept and move to the next field.
2. **CONCEN (Concentration):** This field is active whenever mg or mcg is selected in the “UNITS” field and requires data entry of the concentration of mg or mcg per mL. When the correct data is entered using the numeric keys, press YES/ENTER to accept, and move the cursor to the next field. If mL is chosen in the “UNITS” field, this line will not display. The range for this field is 0.1 to 999.
3. **DELAY Start:** This field allows you to delay the start of the infusion from one minute up to twenty-four hours. If the infusion is **NOT** to be delayed, select “OFF” and press “YES/ENTER” to enter. The cursor will move to the next field. If the infusion **IS** to be delayed until a later time, press the NO/CHANGE key. The delay start will change to

“ON.” Now, when you press YES/ENTER to accept, the infusion start time screen will appear: The time format (12 hr AM/PM or 24 hr Military) selected in the BIOMED SETUP Menu will apply in this table; and, if military was selected, the AM/PM field will not appear here. If the entire delay start screen is to be aborted, press the NO key at the “ACCEPT” field. When the start time and date are entered and the data is accepted after the YES/ENTER key is pressed, the infusion is set to start at the programmed time. The display then returns to the next field in the Pre-Prescription Menu.

After the entire prescription program is accepted, if the pump is turned off and back on, press YES at “PROGRAM” and YES to “REPEAT Rx.” The cursor will automatically scroll through the program. No changes will be allowed in the Pre-Prescription Menu during this time, thus maintaining the originally programmed Delay Start time. To change the Delay Start time, press the NO/CHANGE key when the Run Options Screen is displayed:


Run Options Screen

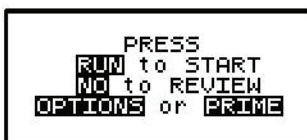

This will allow the clinician to go back to review the program and at this point a change can be made to the delay start time.

Note: If the pump is not connected to the patient’s access site at this time, the patient or caregiver should be taught proper techniques to access the infusion site before the scheduled start time.

When the RUN key is pressed, the green and yellow LED lights will flash to indicate that the pump will automatically start when the delay start count down period is up. During the delay start time, the following screen appears:

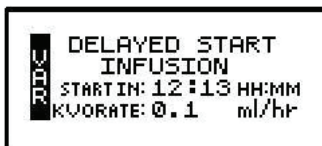

If the KVO rate is set to “0,” the pump will beep ten minutes prior to the beginning of the Continuous Therapy start time. To stop the audible beep, press the SILENCE key.

IMPORTANT INFORMATION

During the delay start period, if the KVO rate is set to “0,” no fluid is infusing. Be sure the access device is patent and has been flushed per facility policy to maintain patency during this time period. If a KVO rate is programmed, then that KVO rate will be used in the Delay Start period.


4. NEXT?: If there are any changes to be made, use the up or down arrow keys to locate the field to be changed, use the numeric keys to enter the corrected information, and press YES/ENTER to accept the new data. If all the fields are acceptable, move the cursor to

the “NEXT?” field and press the YES/ENTER key to move on to the next screen.

At the following screens, the prescription information is entered to complete the programming of the therapy.

Variable Therapies are programmed in individual dose screens, from one up to twenty-four. Before information about each individual dose is entered, the bag volume and number of total doses per bag are entered on the initial programming screen. Once the total number of doses is entered and accepted, the pump will subsequently display a screen for each dose to program the specific dose amount, rate, and time:

Variable Prescription Menu 1

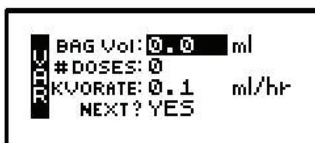

```
BAG Vol: 0.0 ml
#DOSES: 0
KVO RATE: 0.1 ml/hr
NEXT? YES
```

1. BAG Vol (IV Bag Volume): Use the numeric keys to enter the amount of fluid in the IV Bag or Medication Reservoir (include the overfill volume).


Note: The volume used for priming via the pump or for KVO is deducted from the Bag Volume. Therefore, Bag Volume should be equal to or greater than the Amount To Be Infused to start the infusion.

When the correct amount has been entered, press the YES/ENTER key to accept this field and move on to the next. The range for this field is 1 to 9999.

2. # Doses: The Variable Therapy can be programmed to deliver from one to twenty-four individual programs or “doses.” Enter the total number of doses to be delivered from the previously entered bag volume. This number dictates the number of dose programming screens that will subsequently display. When the correct number of doses has been entered, press the YES/ENTER key to accept this field and move on to the next.
3. KVO RATE: This field requires data entry using the numeric keys to set the rate at which fluid will be delivered in a “Keep Open” state. The pump defaults to a rate of 0.1 mL/hour, can be set from 0 to 10 mL/hour. All dose rates (mL/hr) must be greater than the KVO rate. When the desired KVO rate has been entered, press the YES/ENTER key to accept and move the cursor to the next field.
4. NEXT?: If there are any changes to be made, use the up or down arrow keys to locate the field to be changed, use the numeric keys to enter the corrected information, and press YES/ENTER to accept the new data. If all the fields are acceptable, move the cursor to the “NEXT?” field and press the YES/ENTER key to move on to the next screen.


At the following screens, the dose information is entered to complete the programming of the pump for a Variable Therapy. Each variable dose will have its own prescription screen:

Variable Prescription Menu 2

 DOSE: 1 of 24 AmtTBI: 0.0 ml RATE: 0.0 ml/hr TIME: 0:00 HH:MM	DOSE: 1 of 24 AmtTBI: 0.0 mg RATE: 0.0 mg/hr TIME: 0:00 HH:MM	DOSE: 1 of 24 AmtTBI: 0.0 mcg RATE: 0.0 mcg/h TIME: 0:00 HH:MM
<input type="checkbox"/> NEXT? YES		

Note: One dose per screen is entered.

Variable Prescription Menu 3

 DOSE: 2 of 24 AmtTBI: 0.0 ml RATE: 0.0 ml/hr TIME: 0:00 HH:MM	DOSE: 2 of 24 AmtTBI: 0.0 mg RATE: 0.0 mg/hr TIME: 0:00 HH:MM	DOSE: 2 of 24 AmtTBI: 0.0 mcg RATE: 0.0 mcg/h TIME: 0:00 HH:MM
<input type="checkbox"/> NEXT? YES		


1. DOSE: The dose number on the first field changes to indicate which dose is currently being programmed. The total number of doses is also displayed, and the pump will only prompt programming screens for the number of doses indicated on the initial programming screen.
2. Amt TBI: Use the numeric keys to enter the amount to be delivered in this dose. The units selected in the prescription menu will automatically appear in this field, whether mL, mg, or mcg. The amount to be infused is limited to the bag volume total, and each time a dose is programmed, the new amount to be infused is limited to the remaining bag volume total. When the amount is correctly entered, press the YES/ENTER key to accept, and move to the next field. The range for this field is 1 to 9999, or the remaining bag volume amount.
3. RATE: The **Moog Curlin Infusion** pump can deliver from 0.1 mL to 400 mL per hour. Use the numeric keys to enter the prescribed rate for this dose. When this field is correct, press the YES/ENTER key to accept, and move to the next field.
4. TIME (calculated by pump): Displays the time required to complete this dose. If the time is not acceptable, press the NO key to edit this field, and enter the new TIME. (Remember, when you change the time, a new RATE will be calculated and will have to be accepted before you exit this menu.) When the time is correct, press YES/ENTER to accept, and move to the next field. The range for this field is 0:01 to 288:00 HH:MM.

Note: The time entered is limited by what time is remaining out of the maximum time of 288 hours. The time calculated by the pump may be less than a minute, however, one minute is the minimum entry and display value.

5. NEXT?: If there are any changes to be made, use the arrow keys to locate the field to be changed, use the numeric keys to enter the corrected information, and press YES/ENTER to accept the new data. If all fields are acceptable, move the cursor to the “NEXT?” field and press the YES/ENTER key to accept and move on to the next dose programming screen. When the total number of doses is programmed, the pump will display the Variable Summary Screen.

The summary screen indicates the total number of doses programmed and the total time and volume required. The fields on this screen cannot be modified.

Variable Summary Screen

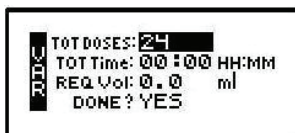


1. TOT DOSES: The field displays the total number of doses programmed. If correct, press the YES/ENTER key to accept and move to the next field.
2. TOT Time: The pump calculates the total time required to deliver the total number of doses. Press the YES/ENTER key to accept and move to the next field.
3. REQ VOL (Required Volume): The pump calculates the total volume required to deliver the doses. Press the YES/ENTER key to accept and move to the next field.
4. DONE?: If any of the information is NOT acceptable, press the YES key and then the NO key at the Run Options Screen. The display will return to the Pre-Prescription and Prescription screens to allow changes to be made. When all fields are acceptable, move the cursor to the “DONE?” field and press the YES/ENTER key to move on to the next screen.

IMPORTANT INFORMATION

Before the infusion is started, it is important to consider a few remaining issues, such as checking the administration set to be sure it was primed or purged of all air, or perhaps the pump may need to be put into a security lock setting for additional safety.

The following screen offers four choices:

Run Options Screen

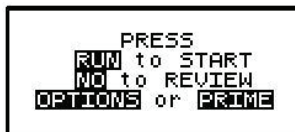

Note that you do not need to answer yes or no questions or enter data with the numeric keys as previously requested. This screen requires only that you press the corresponding key on the keypad to begin the requested function. The following explains each selection beginning with Prime:

Prime

The IV bag and administration set should be inspected and be free of all air and air bubbles before they are connected to the access site. Priming may have been done by gravity when the administration set was first connected to the bag (see Chapter 1, “Preparing Medication for Infusion”). However, if it was not done then or if there are any remaining air bubbles to be removed, the pump’s “PRIME” function will assist in this process. To prime the set by using the pump, follow the directions below.

Note: The lock setting may restrict the ability to prime using the pump.

1. Press and release the PRIME key at the Run Options Screen.
2. The following screen appears:

Prime Direction Screen



Disconnect Pt.
Release Clamps
HOLD PRIME key
YES to EXIT

3. Disconnect the administration set from the access site using aseptic protocols.


IMPORTANT INFORMATION

Priming with the set connected to the patient could result in delivery of air or overdose and may cause injury or even death to the patient.

4. Release any clamps on the administration set.
5. Press and hold the PRIME key. Fluid will be pumped at a rate of 300 mL/hr through the administration set as long as the PRIME key is pressed or until 6 mL is pumped. The pump will then beep once and require repressing and holding the PRIME key again until 6 more milliliters is delivered. Continue this process until all air has been purged from the IV bag and the administration set.

Note: If the bag volume is depleted during priming, a subsequent priming event cannot be initiated.


While the PRIME key is depressed, the following screen appears:

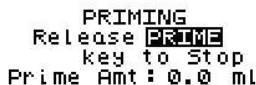

PRIMING
Release PRIME
key to Stop
Prime Amt: 0.0 mL

The amount used for the prime session is accumulated and displayed on the bottom field of this screen.

6. When priming is complete, release the PRIME key. The following screen will display:

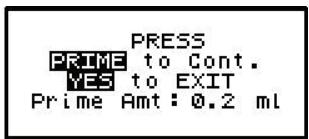

7. To continue priming, press the PRIME key again. If priming is complete, press YES/ENTER to exit and return to the Run Options Screen. The total amount of solution used in this priming session displays on the bottom field.

Note: The amount of fluid used in the priming process will be deducted from the Bag Volume Total but will NOT be counted in the amount infused because this amount is not to be delivered to the patient. The bag volume should still be equal to or greater than the Amount TBI before you can proceed with the infusion. If it is not, an alert will appear stating, AMT TBI GREATER THAN BAG VOL. You have the option to continue by pressing the YES/ENTER key or return to the Run Options Screen by pressing the NO key.

Options


Once the prescription is programmed, there are more customizing features available for each therapy. Press the OPTIONS key from the Run Options Screen to access these options. The OPTIONS menu can also be accessed while the pump is running.

Note. After the infusion has started, any changes made through the OPTIONS menu involving:

- “HRLY TOT” and
- “CLR HRLY”

will return you directly to the “ACCEPT OPT?” field.

The following screen displays the Variable Therapy Options available. When you open this menu, use the up and down arrows to move the highlight bar to the desired option. Lock settings may require an access code before the option can be changed. Any new setting, except “CLEAR HOURLY” screen acceptance becomes effective when the Options Screen is exited by pressing YES/ENTER at the “ACCEPT OPT?” field.

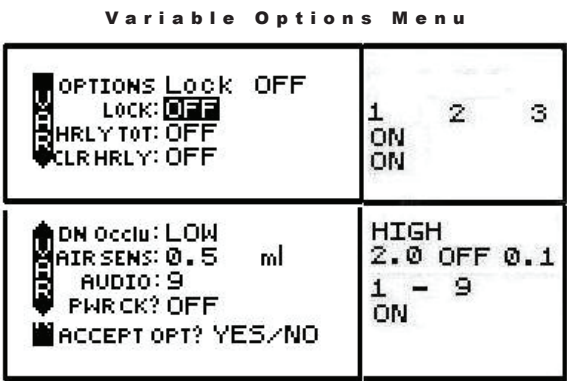

Note: The pump’s default settings are indicated above in the **left** screen.

1. **LOCK:** When the lock is set to “OFF,” the clinician has access to all of the pump functions. After the pump is completely programmed, change the lock to meet the security needs of this therapy. There are four choices: “OFF,” “1,” “2,” or “3.” Each level higher limits the available options. To change the lock setting, press the NO/CHANGE key until the correct lock setting appears. Then, press YES/ENTER key to accept the new lock selection. Once the pump has been set into a locked condition, any subsequent request to change the lock will require a clinician access code. (See “Clinical Information and Access Code” chapter for access code information, lock setting table and available features for each lock setting.)
2. **HRLY TOT (Hourly Totals):** To view hourly infusion totals press the NO/CHANGE key to change OFF to ON and then press YES/ENTER. The following screen will appear:

Hourly Totals Screen

<div style="display: flex; align-items: center;"> <div style="writing-mode: vertical-rl; transform: rotate(180deg); font-weight: bold; margin-right: 5px;">PUMP</div> <div> HR VOL-ml 00 ----- 01 ----- 024 15.9 </div> </div>	AMT-mg ----- -----	AMTmcg ----- -----

The screen displays hourly infusion delivery for each hour (00-23) of the last 24 hours of therapy in the unit selected.

- a. The highlighted field has a marker ◀ indicating the current hour in progress.
 - b. Use the up arrow key to scroll back through the last 24 hours.
 - c. If the infusion was not running, a series of ----- will appear for that hour.
 - d. To exit this screen, press the NO/CHANGE key.
3. **CLR HRLY (Clear Hourly Totals):** To clear Hourly Totals press the NO/CHANGE key to change OFF to ON and then press YES/ENTER. The following screen will appear:

Clear Hourly Totals Screen

PUMP	Clear Hourly Totals NOW
	ACCEPT? YES/NO

- a. To clear the Hourly Totals, press the YES/ENTER key. All Hourly Totals will be cleared and the cursor will return to the Options Menu.
- b. To maintain the Hourly Totals, press the NO/CHANGE key. Hourly Totals will be maintained and the cursor will return to the Options Menu.
- c. A lock level of 2 or 3 will require an access code entry before allowing Hourly Totals reset.

4. DN Occlu (Down Occlusion): This feature allows the pump to sense an occlusion pressure in the line between the pump and the access site— e.g., a kink in the tubing between the pump and the patient, a blockage in the access site, a clamp left on the distal tubing. There are two options for this field: the “Low” setting (default setting) triggers the alarm when it detects in-line pressures at 400 mmHg \pm 150 (8 \pm 3 psi “pounds per square inch”), or the “High” setting alarms when it detects in-line pressures at 900 mmHg \pm 150 (18 \pm 3 psi). Press NO/CHANGE to change the selection. When the correct pressure setting appears, press the “YES/ENTER” key to accept it and move the cursor to the next field.
5. AIR SENS (Air In Line Sensor): This feature allows you to select the amount of air the pump will detect in the administration set tubing before it goes into an alarm state. The choices are “0.1 mL,” “0.5 mL” (the default setting), and “OFF.” Press the NO/CHANGE key to change the options until the desired setting appears on the display, and then press YES/ENTER to select and move the cursor to the next line.

**IMPORTANT
INFORMATION**

Whenever the Air-In-Line Sensor is set to OFF, Moog recommends the use of an air-eliminating filter.

When the Air-In-Line Sensor is set to “OFF” and the RUN key is pressed at the Run Options Screen, the alert message below will appear on the screen. It will also appear if the Air-In-Line is changed from the Options Menu when an infusion is in progress and the AIR SENS field is accepted.

Air-In-Line Sensor Off Message Screen




```
AIR Sensor OFF
Using In Line
Air Filter?
YES/NO
```

This message alerts you to the Air-In-Line Sensor being set to “OFF.” The message will require answering “yes” or “no” to whether the administration set being used has an air-eliminating filter. If you press the YES key to indicate that there is an air-eliminating filter in use, the Air-In-Line Sensor will be completely disabled and the pump will start the infusion.

If you press the NO key to indicate a filter is not in use, the Air-In-Line Sensor will be set to detect the presence of any contiguous 2-milliliter air in the line. The alert message below will then briefly appear on the screen. If the alert occurred at the Run Options Screen, the pump will start the infusion.

Air-In-Line Sensor Set at 2mL Alert Screen

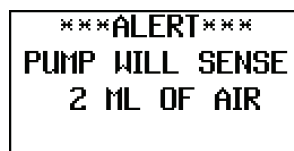

```
***ALERT***
PUMP WILL SENSE
2 ML OF AIR
```

Also, when the Air-In-Line sensor is completely disabled and the infusion goes to an “Infusion Complete” or “Empty Bag” status, the Air Sensor setting will automatically change to 2 mL.

6. **AUDIO (Audio Volume):** This feature allows you to adjust the audio volume of the pump’s alarm setting. You can enter a number from 9 (the loudest setting) to 1 (the quietest setting). When you select a number, the pump will beep at that audio level. Press the NO key and enter a new setting (1 - 9). Repeat until the desired setting appears on the display, and then press YES/ENTER to select it and move the cursor to the next field.
7. **PWR CK? (Power Check):** This feature allows you to periodically recheck the battery charge. It will display the active power source of the pump, whether batteries or external power. If the active source is the batteries, a graph will indicate approximately how much charge remains in the batteries. To access this information, select “ON” and press YES/ENTER. The Power Source screen will display for five seconds and return to the Options Menu.
8. **ACCEPT OPT?:** If the changes made in the Options Menu are acceptable, press the YES/ENTER key. If the changes are not acceptable, use arrow keys to cursor up to any field for corrections, or abort all changes made in this menu by pressing the NO key and return to the previous screen.

There are two remaining selections to consider from the Run Options Screen: **NO** to **REVIEW** and **RUN** to **START**.


Reviewing or Changing the Program

To review or make any changes in the prescription programming, press the NO/CHANGE key at the Run Options Screen, and the prescription screens will reappear (changes may be inhibited by lock setting). Changes to the prescription can be made before the start of the therapy; however, once a therapy has started, no changes can be made to the Pre-Prescription Screen. It will appear only for review purposes. Attempts to change the Pre-Prescription Menu during a new program review will cause an alert screen to be displayed, notifying the programmer that changes will require reprogramming of the prescription.

Starting the Infusion


RUN: When all parameters are correct and the pump has been primed properly and connected to the access site, press RUN at the Run Options Screen to start the therapy. If a delayed start is programmed, the Delay Start Count Down Screen will appear. If no delay start is programmed or the delay has elapsed, the pump will begin infusing and the following screen displays the status of the therapy:

Variable Run Screen 1

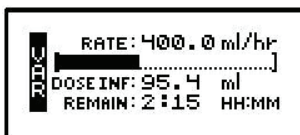

This information toggles with the following:

Variable Run Screen 2

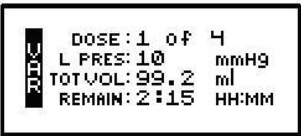

This screen displays as long as the infusion is running. Notice the information on each field:

1. The first field displays the “RATE” of the infusion in whichever units/hour has been selected. This field toggles to also display the current dose # of total doses programmed.
2. The second field is a graphic bar representing the percentage completed of the current dose. **The graphic bar will toggle with Line Pressure (“Line Press”) which displays the current patient side (downstream) pressure in the administration set in millimeter of mercury (mmHg) units.**
3. The third field displays “Total Volume Infused” (for entire therapy) and toggles with the amount infused of the dose in progress, and Total Infused if units are in mg or mcg.
4. The fourth field displays the remaining time to the completion of the total infusion and toggles with total remaining volume to be infused, and total remaining amounts if units are in mg or mcg.
5. In addition to the display screen giving information regarding the status of the infusion, the green LED light blinks whenever the pump is infusing.

The information on the run screen will alternate between a four-line display and a large-font display of each individual line of information on the screen.

Variable Information Screens

Tot Vol 142.2 mL	Remain 2:08 HH:MM
Dose 1 of 4	Rate 400.0 mL/hr
Remain 849.7 mL	Dose Inf 152.3 mL
Line Pres 10 mmHg	

Note: The top line of these running screens will display the name of the running protocol when programmed in the Library mode, or it will display "Unidentified" when programmed outside of the library.

Interrupting an Infusion

The infusion can be paused at any time by pressing the PAUSE key. The yellow standby LED light blinks, and the Resume/Repeat Rx/New Program Menu screen is displayed.

The screen offers the user three options:

Resuming the Current Container

1. RESUME: To resume the therapy in progress from where it left off, press the YES/ENTER key at this field. The pump will return to the Run Options Screen, and you can resume the therapy.

Note: If you wish to hang a new container, move the cursor to “REPEAT Rx” (see following section).

- a. Press the RUN key to resume the therapy.
- b. Press the NO key to go back to the programming screen to make changes in the therapy program (this feature may be blocked by a security lock, thus not allowing changes to the program unless an access code is given). Also, remember that no changes are allowed in the first prescription menu if an infusion is in progress.
- c. Press the OPTIONS or PRIME keys if either of these functions is desired at this time.


Hanging a New Container

2. REPEAT Rx: To repeat the present therapy and add another IV bag or reservoir with the same prescription, press the YES/ENTER key at this field. If more than one therapy is programmed, a Select Therapy Screen will list all preprogrammed therapies. Select which therapy to repeat by moving the cursor to the therapy of choice and pressing the YES/ENTER key.

The pump will then display the program screen with the last programmed values of the therapy and the cursor will automatically advance through the prescription menus. You can stop the auto-advance function by pressing any key (except ON/OFF) and then complete the confirmation of the prescription manually by pressing the YES key at the remaining fields.

Note: The previously programmed AIL setting is also retained

When the program is accepted, the display returns to the above Run Options Screen. If the current therapy is still in progress when the “REPEAT Rx” function is selected, the following screen will appear:

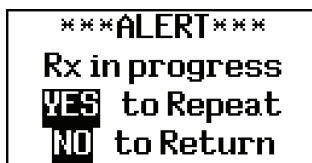

- a. This screen is a reminder that the infusion is not complete. If the therapy is to be terminated and a new IV container added, press YES/ENTER to go to the programming screen, confirm the program and proceed with the repeat infusion.

- b. If the incomplete therapy is to be continued, press NO and the Pause Menu Screen will be displayed, then press YES and the Run Options Screen will be displayed. To resume the unfinished therapy, press RUN.

Entering a New Program

3. NEW PROGRAM: To program an entirely new prescription or different therapy, press the YES/ENTER key at this field (The ability to change these parameters may be controlled by the lock setting). The display will return to the Select Therapy Screen if more than one therapy mode is enabled, or to the Pre-Prescription Screen of the single therapy mode. If the current container has not completely infused, the following screen will appear:

```

***ALERT***
Rx in progress
YES New Program
NO to Return
  
```

- a. This screen is a reminder that the infusion is not complete. If you want to return to the current container and finish infusing it, press the NO/CHANGE key to return to the Pause Menu screen.
- b. To abort the current container and enter a new program, press the YES/ENTER key. If a prescription has previously been entered for the therapy mode you wish to program, the following screen will appear:

```

***ALERT***
Rx Programmed
YES to Erase Rx
NO to Return
  
```

- c. If you want to save the current prescription, press the NO/CHANGE key to return to the Pause Menu Screen. If you wish to erase the current prescription and enter a new program, press the YES/ENTER key to proceed. All previous prescription values will be erased.

Note: Selecting “NEW PROGRAM” will also reset the Options selected to their default settings except for the LOCK and AUDIO settings which must be changed manually.

Infusion Complete

When the infusion is complete, the following screen will display:

Variable Infusion Complete

```

INFUSION COMPLETE
.....
TOTVOL: 200.0 ml
KVORATE: 0.1 ml/hr
  
```

This screen displays:

1. "INFUSION COMPLETE."
2. A graphic representation (completely filled in bar graph) indicating infusion complete, or the smaller dot configuration (shown here) if a KVO RATE is programmed.
3. "TOT Vol" (Total Volume): If units are mg or mcg, this field will also toggle to give the Total Infused.
4. "KVO Rate:" The preset KVO rate displays.
5. Infusion Complete is considered an alarm status, and the audio alarm will beep periodically (every 3 seconds if there is no KVO, every 20 minutes if there is a KVO) to alert the user to this condition. Additionally, the LED lights blink to alert you to the status of the pump. The green light blinks when the KVO rate is infusing (if one is programmed) and the red light blinks to indicate that the Infusion Complete alarm exists.

Stopping the Infusion

To halt the infusion at "Infusion Complete," press the PAUSE key and then choose to either:

1. Turn off the pump by pressing the OFF key, or
2. Select one of the options from the Pause Screen:

Pause Menu Screen



3. REPEAT Rx: To repeat the present therapy and add another IV bag with the same prescription, press the YES/ENTER key at this field. If more than one therapy is programmed, a Select Therapy Screen will list all preprogrammed therapies. Select which therapy to repeat by moving the cursor to the therapy of choice and pressing the YES/ENTER key.

The pump will then display the program screens with the last programmed values of the therapy and the cursor will automatically advance through the prescription menus. You can stop the auto-advance function by pressing any key (except ON/OFF) and then complete the confirmation of the prescription manually by pressing the YES key at the remaining fields.

4. NEW PROGRAM: To program an entirely new prescription or a different therapy, press the YES/ENTER key at this field and the display will return to the Select Therapy Screen. (The ability to change these parameters may be controlled by the lock setting.)

Note: The therapy is complete; therefore, "RESUME" is not an option at this time and is not displayed.

Troubleshooting

*This section assists in identifying and resolving alert and alarm conditions that may occur during the use of the **Moog Curlin Infusion pump**.*

Alerts, Alarms, and Error Codes

Alerts, Alarms, and Error Codes are classes of events that are relayed to the user and logged into the history file. The screen will display messages, the LED status lights will blink, and the audio alarm will sound. The backlight feature will illuminate the screen during an alert or alarm.

IMPORTANT INFORMATION

If the batteries have been dislodged or are so low in power that they cannot operate the pump, a solid tone may occur accompanied by a blank display screen and a constant red alarm light. In this event, replace the batteries.

ALERTS

Alerts report conditions that require attention from the user but may not be safety issues. When an alert occurs, a message is displayed on the LCD screen, and the audio alarm is sounded. The message displays on the screen for three seconds if it is a user-initiated alert, such as a wrong key press. If it is an automatic alert occurring while a therapy is infusing, such as “low battery,” the message displays for twenty seconds. If it is an alert that requires a user response, the alert remains displayed until the user presses one of the keys specified on the screen. The following is a list of the alerts in alphabetical order.

LCD Display: “***ALERT *** AIR Sensor OFF. Using In-Line Air Filter?”

Alert: Air-In-Line Sensor Disabled

Cause: The Air-In-Line Sensor has been disabled and either the RUN key has been pressed to run an infusion or the AIR SENS setting has been set to OFF on the Options Menu while the pump is running.

Resolution: If an air-eliminating filter is being used, press YES. The infusion will start/ resume and the Air-In-Line sensitivity will be disabled until the empty bag alert occurs or infusion complete is reached. No “AIR IN LINE” alarms will occur while the air sensitivity is disabled.

If no air-eliminating filter is being used, press NO. The infusion will start/ resume and the Air-In-Line sensitivity will be set to 2 mL for the remainder of the infusion.

Note: If the screen is not acknowledged with a YES or NO key press, the Air-In-Line sensitivity will be set to 2 mL when the screen is cleared (i.e. alarm occurs while screen is displayed).

LCD Display: “***ALERT *** PUMP WILL SENSE 2 ML OF AIR”

Alert: Air-In-Line sensitivity will be set to 2 mL.

Cause: NO was pressed at the “AIR SENS OFF. Using In Line Filter?” alert screen. The air in line sensitivity will be set to trigger an AIR IN LINE alarm if 2 mL of air is detected.

Resolution: If an air eliminating filter is being used, press YES when the “AIR SENS OFF. Using In Line Filter?” screen is displayed. This will disable air detection as indicated under “AIR SENS OFF. Using In Line Filter?” alert explanation.

LCD Display: “***ALERT *** AMT NOT 0 OR CURRENT DOSE AMT”

Alert: Unable to change clinician dose while clinician dose in progress.

Cause: A clinician dose is in progress, and a new clinician dose value is entered that is not “0” or the current dose amount.

Resolution: If the current clinician dose is to be continued, change the “Clin DOSE” value back to the original value and accept the field and menu. If the current clinician dose is to be terminated, which will allow a new clinician dose to be programmed, enter a value of 0 in the “Clin DOSE” field and accept the field and menu. The next time that the Options Menu is accessed a new “Clin DOSE” value can be entered.

LCD Display: “***ALERT *** AMT TBI GREATER THAN BAG VOL”

Alert: Value entered is greater than value in Bag Vol field.

Cause: Occurs when the amount entered for the “Amt TBI” or “Vol TBI” field is greater than the value in the bag volume field.

Resolution: Recheck the data entry.

LCD Display: “***ALERT*** AMT TBI GREATER THAN BAG VOL. YES to Continue NO to Return”

Alert: Insufficient volume

Cause: Occurs when bag volume is less than amount to be infused. Can occur when pump’s priming feature is used as bag volume is depleted by the amount used in priming.

Resolution: To modify the Bag Volume or Amt/Vol TBI, press NO to return to the Run Options Screen. The programming screens will then be accessible by pressing NO to REVIEW on the Run Options Screen.

Note: Access to the programming screens is lock level restrictive. If the actual amount in the source bag/container is sufficient to complete the programmed infusion, press YES to start the infusion.

Note: If the infusion is continued without sufficient volume, Air-In-Line or Up Occlusion alarms may occur later in the infusion.

LCD Display: “***ALERT *** BASAL RATE EQUAL OR GREATER THAN BAG VOL”

Alert: Value entered is greater than or equal to the value in Bag Vol field.

Cause: Occurs when the amount entered for the “Basal Rate” field is greater than or equal to the value in the bag volume field.

Resolution: Recheck the data entry.

LCD Display: “***ALERT *** AMT TBI LESS THAN AMT INFUSED”

Alert: Unable to change amount to be infused to an amount less than what has already infused.

Cause: Occurs when user tries to change the “Vol TBI/Amt TBI” field in the prescription menu to an amount lower than the amount which has already infused.

Resolution: Recheck the data entry. If alert recurs, resume the infusion and observe the “Vol Inf/Amt Inf” field on the run menu. The value for data entry must be greater than the value displayed on the run menu.

LCD Display: “***ALERT *** BOLUS DOSE IS NON-ZERO”

Alert: Unable to enter a bolus interval or # of boluses/hour of “0.”

Cause: Bolus interval or # of boluses/hour value entered is zero, and bolus dose value is non-zero. When a bolus value is entered, there must also be a value greater than 0 entered for bolus interval and # of boluses/hour.

Resolution: Recheck data entered for the prescription.

LCD Display: “***ALERT *** BOLUS DOSE NOT ALLOWED”

Alert: A bolus dose is not allowed.

Cause: Bolus key or button pressed when no bolus dose is programmed.

Resolution: If bolus doses are part of the prescription, modify the Rx/MedLMTS™ program to allow bolus doses.

LCD Display: “***ALERT *** BOLUS REQUESTED”

Alert: Not time for bolus.

Cause: Bolus button or key is pressed before the programmed time interval between bolus doses have elapsed.

Resolution: Advise patient of time limits between bolus doses per agency protocols.

LCD Display:	“***ALERT *** CALCULATED AMT OUT OF RANGE”
Alert:	Invalid calculated amount for infusion
Cause:	The calculated amount for the infusion, based on the values entered for rate and time, exceeds either the pump limit or the value entered for bag volume.
Resolution:	Recheck data entry for Rx.
LCD Display:	“***ALERT *** CALCULATED RATE OUT OF RANGE”
Alert:	Invalid calculated rate of infusion
Cause:	The calculated rate for the infusion, based on the values entered for Volume/ Amount and Time, exceeds either the minimum or maximum rate limit of the pump or the maximum rate programmed in MedLMTS™ limits.
Resolution:	Recheck data entry for Rx.
LCD Display:	“***ALERT *** CALCULATED TIME OUT OF RANGE”
Alert:	Invalid calculated time
Cause:	The calculated time for the therapy, based on the values entered for Volume/ Amount and Rate, exceeds the therapy time limit of 288:00 HHMM (167:59 HHMM Software Versions below R37C).
Resolution:	Recheck data entry. The time calculated by the pump cannot exceed the pump range.
LCD Display:	“***ALERT *** Changes Require Re-programming. YES to Continue NO to Return”
Alert:	Changes to the data will require re-entry of prescription.
Cause:	Attempted to change the Pre-Prescription Screen prior to start of infusion.
Resolution:	If a change to the programmed prescription is required, press YES. Note: Selecting YES will clear the <u>entire</u> prescription and require <u>re-entry</u> the prescription. If no change is required, press NO to preserve the current prescription and return back to the Pre-Prescription screen.
LCD Display:	“***ALERT *** DATE/TIME CHANGE NOT ALLOWED”
Alert:	Not allowed to change date or time.
Cause:	Attempt made to modify system clock while a therapy/delay is in progress.
Resolution:	If the therapy in progress is to be continued, wait until the infusion is complete and then change the date and time. If it is not necessary to preserve the therapy in progress or any other programmed therapy, use the “CLEAR Rx” feature of the Clinician

BioMed menu. This feature will erase all programmed therapies in the pump. The date and time can then be changed.

If the therapy is to be terminated but the prescription preserved (excluding Delayed Start), go to the Pause Menu, select “REPEAT Rx” and press YES at the “Rx in progress” alert screen. The pump can then be turned off and then on to access the Date/Time change option in the Clinician BioMed Menu. Selecting “NEW PROGRAM” and the name for the therapy in progress, instead of “REPEAT Rx”, may also be used but this selection will erase the prescription for the therapy.

LCD Display: “***ALERT *** DATE/TIME INVALID”

Alert: Unable to enter invalid date or time.

Cause: Date or time entry is not in a valid format.

Resolution: Recheck data entry. An invalid date or time will not be accepted.

LCD Display: “***ALERT *** DELAY START TIME NOT VALID”

Alert: Delay start time not valid.

Cause: Delay start time entered is less than one minute or greater than 24 hours from the current system time.

Resolution: Reset Delay Start Time to fall within pump limits. (To obtain the current date and time of the pump, turn the pump off and then on and observe the time on the Welcome Screen).

LCD Display: “***ALERT *** DOOR OPEN or SET NOT PROPERLY INSTALLED”

Alert: Door open or administration set is not properly installed.

Cause: Door latch is not completely closed or the administration set is not installed according to installation instructions.

Resolution: Recheck the door latch assembly and securely close the door. If the door was closed properly, re-install the set according to the installation instructions.

LCD Display: “***ALERT *** DOSE AMT IS LESS THAN DOSE INFUSED”

Alert: Dose amount entered is less than the amount already infused for the dose.

Cause: New dose value entered is less than the amount already infused for the dose.

Resolution: Recheck data entry. After the dose in progress finishes, the dose amount may be changed to the lower value.

If the above is not suitable, re-program the prescription.

LCD Display: “***ALERT *** DOSE CANCELLED. RESUME INFUSION THEN NEW DOSE”

Alert:	Cancellation of a clinician dose is pending and can only be processed with the pump running.
Cause:	Clinician dose in progress is cancelled while the infusion is paused; and the user attempts to enter a new clinician dose, prior to resuming the infusion.
Resolution:	Resume the infusion and then program the desired clinician dose.
LCD Display:	“***ALERT *** DOSE FREQ LESS THAN DOSE DURATION”
Alert:	Dose frequency time is less than the time required to infuse the programmed dose.
Cause:	The dose frequency value entered is less than the time required to infuse the programmed dose amount.
Resolution:	Recheck data entry. Dose frequency cannot be less than required time for dose to infuse.
LCD Display:	“***ALERT *** EMPTY BAG”
Alert:	Empty Bag
Cause:	Occurs when the volume infused for the prescription is equal to the “Bag Vol.” Bag Vol may have been reduced due to priming.
Resolution:	Verify that the volume remaining in the source container is sufficient to complete the current therapy. If the volume is insufficient, add another source container or stop the infusion.
LCD Display:	“***ALERT *** EXCEEDS POSSIBLE DOSES PER HR”
Alert:	Unable to enter # of boluses/hr that would be possible given the programmed bolus amount.
Cause:	# Boluses/hr value exceeds the possible # given the bolus amount.
Resolution:	Recheck data entered. The # of boluses per hour cannot exceed the time required to deliver the bolus amount programmed.
LCD Display:	“***ALERT *** EXTERNAL POWER DISCONNECTED”
Alert:	External power source disconnected.
Cause:	External power source is disconnected or external battery pack power is completely depleted during pump operation.
Resolution:	Reestablish external power source, or let pump continue to operate on batteries. Note: The battery level should be checked after the external source is disconnected to gage if additional batteries will be required before an external power source is available again.
LCD Display:	“***ALERT *** FIELD NOT ACCEPTED”
Alert:	Field Not Accepted.

Cause:	The field indicated by the flashing cursor has not been accepted.
Resolution:	Recheck the field and accept it by pressing the YES/ENTER key.
LCD Display:	“***ALERT *** GREATER THAN MAX FIELD LIMIT, MAX = _____”
Alert:	Unable to program a field with a value that exceeds the maximum limit.
Cause:	Value entered is greater than the allowable limit.
Resolution:	Recheck data entry. A value entered cannot exceed the field limit.
LCD Display:	“***ALERT *** GREATER THAN WHAT BOLS INT ALLOWS”
Alert:	Unable to enter a # of boluses/hr that exceeds the possible number allowed given the bolus interval.
Cause:	# Boluses/hr value exceeds the possible number given the bolus interval (“BOLS INT”) programmed.
Resolution:	Recheck data entry. The # of boluses/hr cannot be set for higher than the number allowed using the given bolus interval.
LCD Display:	“***ALERT *** INFUSION NOT IN PROGRESS”
Alert:	Infusion not in progress.
Cause:	Modification attempted to Next Dose or Early Down Ramp field without an infusion in progress.
Resolution:	Wait until infusion is in progress before modifying or modify original Rx programmed.
LCD Display:	“***ALERT *** INSUFFICIENT VOLUME MAX DOSES=_____”
Alert:	Bag Volume value is insufficient to deliver the remaining.
Cause:	Occurs whenever the prescription is changed so that Bag Volume is now less than amount required for the entire therapy (based on volume already infused plus volume required to deliver the remaining doses).
Resolution:	Recheck data entry. If the source container contains sufficient volume to complete the remaining dose cycles (dose volume + KVO volume), increase the “Bag Vol.” If the volume is insufficient, decrease the “DOSE/BAG” such that the therapy can continue with the new modified prescription. Note: If either “Bag Vol” or “DOSE/BAG” is modified and the therapy is repeated, these fields will have to be verified against the source container volume and prescribed prescription and changed accordingly.
LCD Display:	“***ALERT *** INVALID ENTRY”
Alert:	Invalid Entry
Cause:	Data entered does not fall within the defined limits of the program.

Resolution:	Enter data that falls within the limits of the program. Refer to the specific therapy section in the User's Manual.
LCD Display:	“***ALERT *** KVO RATE IS GREATER THAN INF RATE”
Alert:	Unable to enter KVO rate that is greater than infusion rate.
Cause:	Occurs when user enters a KVO rate that is greater than infusion rate.
Resolution:	Recheck data entry. KVO rate must be less than programmed or calculated infusion rate.
LCD Display:	“***ALERT *** LESS THAN MIN FIELD LIMIT, MIN = _____”
Alert:	Unable to program a field with a value that is less than the minimum limit.
Cause:	Value entered is less than the minimum limit for the pump or MedLMTS™ limit programmed.
Resolution:	Recheck data entry. A value entered cannot be less than the field's minimum limit.
LCD Display:	“***ALERT *** LOW C BATTERIES”
Alert:	Battery source power is low.
Cause:	The batteries that are powering the pump have reached levels that may prevent the continued operation of the pump beyond 1 hour (actual time is rate dependent). Alert applies to only C-cell batteries, not to the rechargeable battery pack.
Resolution:	Pause the therapy in progress, turn the pump off; and then replace the batteries. (See chapter 1 of the User's Manual for instructions on battery change.)
LCD Display:	“***ALERT *** MAX CHANGES FOR PERIOD REACHED”
Alert:	The maximum number of changes for the current delta time window has been reached.
Cause:	The maximum number of pump rate changes for the current delta time window has been reached. This occurs when the rate has been changed more than 12 times during any delta time window.
Resolution:	Allow more time to elapse before attempting to change the rate or reprogram the therapy with MedLMTS™ limits and prescription basal and bolus ranges that will reduce the number of times that the rate will have to be adjusted.
LCD Display:	“***ALERT *** MODIFICATIONS NOT ALLOWED”
Alert:	Modification Not Allowed.
Cause:	The selection cannot be modified at this time or the programmed lock setting does not permit modifications to this selection.

Resolution: Review the User's Manual for program restrictions or if lock level restricted, change lock setting to access area you wish to modify.

LCD Display: "***ALERT *** NEW DOSE IS LESS THAN CURRENT DOSE"

Alert: Unable to program a lower number of doses than number of doses already infused

Cause: While the Variable or Intermittent therapy is in progress, a lower number of doses to be delivered is entered than the number already infused.

Resolution: Wait until current therapy in progress is complete, or PAUSE pump and, using the "REPEAT Rx" function, modify the number of doses.

Note: Selecting to repeat the therapy in progress will terminate the current therapy.

LCD Display: "***ALERT *** NO BOLUS DOSE DEFINED"

Alert: Unable to enter bolus interval or # of bolus/hr because no bolus value has been programmed.

Cause: Entry of a value other than zero for the Bolus Interval or #Bolus per hour field when no bolus value has been programmed.

Resolution: Enter bolus value before entering bolus interval or # of boluses/hr.

LCD Display: "***ALERT *** NO KEYPAD ACTIVITY"

Alert: No Keypad Activity.

Cause: The pump has been displaying a non-running screen for two minutes without any keys being pressed. The alert will recur every 2 minutes as long as there are no key presses.

Resolution: Complete the activity for the displayed menu. The pump may also be turned off, however, all programming that was not complete will be discarded. If the pump was running and the Options menu displayed when the alert occurred, the run screen for the therapy will be displayed when the alert is cleared. Any changes to the Options menu are discarded and will have to be re-entered.

LCD Display: "***ALERT *** NO THERAPY ENABLED"

Alert: No Therapy Enabled.

Cause: Attempting to accept the Enable Therapy menu with all therapies set to "OFF." At least one therapy must be enabled or "ON" for menu to be accepted.

Resolution: Enable the therapy or therapies that are to be active for the pump.

LCD Display: "***ALERT *** NOT ALLOWED, DOSE IN PROGRESS"

Alert: Change is not allowed while a dose is in progress.

Cause:	Modification to the current field is not allowed while the dose is in progress.
Resolution:	Wait until the dose in progress completes before modifying field.
LCD Display:	“***ALERT *** NOT in PCA. NO BOLUS ALLOWED”
Alert:	Bolus requested when therapy is not PCA.
Cause:	Remote Bolus button or BOLUS key is pressed when not in PCA therapy.
Resolution:	Remove the remote bolus cord from the pump if connected. Refrain from pressing the BOLUS key.
LCD Display:	“***ALERT *** NUMBER OF DOSES INVALID”
Alert:	Number of doses entered exceeds maximum for therapy.
Cause:	The number of doses per bag in Intermittent therapy exceeds the allowable maximum.
Resolution:	Recheck data entry. The number of doses requested must be less than or equal to the maximum for the therapy.
LCD Display:	“***ALERT *** PERFORM ROUTINE MAINTENANCE”
Alert:	Maintenance Date (displayed only if date has arrived when pump is turned on)
Cause:	Current Date is beyond the Maintenance Date.
Resolution:	Return pump for routine maintenance.
LCD Display:	“***ALERT *** RAMP INVALID FOR AMT/RATE”
Alert:	Ramp time entered is invalid for amt/rate already programmed.
Cause:	Ramp value entered for TPN is unacceptable given the desired rate and amount programmed.
Resolution:	Recheck all data entered for prescription. If programming a new TPN therapy (“NEW PROGRAM” selected) and the ramp time is a requirement, enter the ramp time(s) and total time first followed by the next governing parameter (rate or volume to be infused).
LCD Display:	“***ALERT *** RATE EXCEEDS DELTA RATE/DELTA TIME”
Alert:	Rate entered exceeds the delta time limits.
Cause:	The rate entered falls outside of the allowable range for the delta rate / delta time limits programmed.
Resolution:	Recheck data entry or wait more time elapses then attempt to change the rate. Rate selected cannot exceed the range placed on MedLMTS™.
LCD Display:	“***ALERT *** RUNNING, PAUSE PUMP PRIOR TO PRESSING OFF”

- Alert:** Pause Infusion before turning off.
- Cause:** The OFF key was pressed while the pump was infusing or a running screen displayed.
- Resolution:** Press PAUSE key before pressing OFF key to stop an infusion.
- LCD Display:** “***ALERT *** Rx in progress, YES to Repeat – NO to Return”
- Alert:** REPEAT Rx selected while a therapy is already in progress.
- Cause:** REPEAT Rx was selected and there is a therapy currently in progress.
- Resolution:** To terminate the current therapy in progress, press YES. Resume will no longer be available. The Select Therapy menu will be displayed (if only one therapy is in programmed, the pre Rx menu for the therapy will be displayed). When the therapy to repeat is chosen, the last prescription data programmed will appear in the menu, excluding Delayed Start and Load Dose (if therapy was previously started). When the infusion is started the entire therapy will be repeated.
- To continue the current therapy in progress, press NO. The Pause Menu will be displayed.
- LCD Display:** “***ALERT *** Rx in progress, YES New Program – NO to Return”
- Alert:** NEW PROGRAM was selected while a therapy is already in progress.
- Cause:** NEW PROGRAM was selected and there is a therapy currently in progress.
- Resolution:** To terminate the current therapy in progress, press YES. “RESUME” will no longer be available. The Select Therapy menu will be displayed (if only one therapy is in programmed, an alert screen informing that the Rx will be erased will be displayed). When the therapy for the new program is chosen, entry of a new prescription will be required.
- To continue the current therapy in progress, press NO. The Pause Menu will be displayed.
- LCD Display:** “***ALERT *** Rx Programmed, YES to Erase – NO to Return”
- Alert:** “NEW PROGRAM” was selected for a therapy that has an existing prescription.
- Cause:** “NEW PROGRAM” was selected for a therapy that has an existing prescription.
- Resolution:** To erase the prescription for the selected therapy, press YES. The pre Rx menu will be displayed and entry of a new prescription will be required.
- To preserve the prescription for the selected therapy, press NO. The Pause Menu will be displayed.
- LCD Display:** “***ALERT *** SET NOT INSTALLED”
- Alert:** Set Not Installed.
- Cause:** Administration set is not properly installed.

Resolution: Install the appropriate **Moog Curlin Infusion** administration set for the desired therapy. Use *only* **Moog Curlin Infusion** administration sets. See Chapter 1, figures 1.9 and 1.10 of the User's Manual for proper installation of the administration set.

LCD Display: "***ALERT *** SYSTEM CLOCK NOT SET"

Alert: System Clock Not Set.

Cause: The clock for the pump is not set

Resolution: Use the BIOMED SETUP menu "Date and Time" option to set the date and time.

LCD Display: "***ALERT *** THERAPY TIME INVALID"

Alert: Invalid therapy time calculated.

Cause: While programming or modifying an Intermittent therapy, the combination of dose duration and dose frequency result in a prescription with a duration greater than 288:00 HH:MM.

Resolution: Recheck data entry. The changes to the field must comply with the total time limit.

LCD Display: "***ALERT *** TIME LESS THAN RAMP TIME"

Alert: Unable to enter a TPN therapy time that is less than the ramp times.

Cause: Time value entered for TPN infusion is less than ramp times.

Resolution: Recheck data entry. TPN infusion time cannot be less than specified ramp times.

LCD Display: "***ALERT *** VALUE PREVENTS A SUBSEQUENT DOSE"

Alert: Value entered will prevent the ability to program any more doses.

Cause: The value entered for the "Variable Dose Amt" or "Time field" will cause no remaining amount or time for a subsequent.

Resolution: Recheck data entry. If this is a new program, reprogramming of the therapy will be required if the error is in a previously programmed dose.
If the therapy is in progress or this is a repeat program that has not started and the error is in a previously programmed dose, turn the pump off and then on. Choosing to resume or repeat the therapy will display the last prescription that was completely programmed.

LCD Display: "***ALERT *** VALUE ↑ LIMIT"

Alert: Value entered is above the Library Limit.

Cause: The value entered is out of range of the soft limit setting(s) for a downloaded therapy.

Resolution: The screen will ask the user to override the soft limit value. Pressing YES will accept the new value.

- LCD Display:** “***ALERT *** VALUE ↓ LIMIT”
- Alert:** Value entered is below the Library Limit
- Cause:** The value entered is out of range of the soft limit setting(s) for a downloaded therapy.
- Resolution:** The screen will ask the user to override the soft limit value. Pressing YES will accept the new value.
-
- LCD Display:** “***ALERT *** EMPTY LIBRARY”
- Alert:** Empty Library
- Cause:** Occurs when “LIBRARY” is selected and no library is present.
- Resolution:** Load a Library or program the pump using the “PROGRAM” option.
-
- LCD Display:** “***ALERT *** PUMP DISABLED CALL PROVIDER”
- Alert:** The pump is disabled.
- Cause:** The pump calibration data is invalid.
- Resolution:** The pump needs to be returned for service.
-
- LCD Display:** “***ALERT*** DOOR OPEN or SET NOT PROPERLY INSTALLED”
- Alert:** Door is opened or administration set is not properly installed while the pump is running.
- Cause:** Door is opened or administration set is not properly installed while the pump is running
- Resolution:** Check the door latch assembly and securely close the door. If the door was closed properly, re-install the set according to the installation instructions. See Figure 1.8 for instructions on closing door.

ALARMS

Alarms are conditions that require correction by the user and may be a safety issue but are not system errors of the pump. All alarm conditions stop the infusion. The only exception to this rule is at the “Infusion Complete” alarm, which allows the KVO rate, when programmed, to continue infusing. When an alarm occurs, a message is displayed on the screen, the audio alarm is sounded, and the red LED light flashes. The message stays on the screen until the pause key is pressed. The SILENCE key can be pressed to disable the audio alarm for one-minute intervals. The cause of the alarm must be resolved, or the alarm will recur when the infusion is restarted. To exit an alarm screen, press the PAUSE key, resolve the alarm condition, and then resume the therapy by selecting “RESUME” from the Pause Menu.

The following is a selected and alphabetical list of the alarms, conditions that cause the alarms, messages displayed on the screen, and actions necessary to resolve them:

LCD Display: “***ALARM*** AIR IN LINE”

Alarm: Air-In-Line
Cause: Pump has detected air in the administration set during an infusion.
Resolution: Check air settings and remove air from administration set and IV bag or reservoir per protocol. (Disconnect from patient's access site before using prime function.)

LCD Display: "***ALARM*** DOWN OCCLUSION"

Alarm: Occlusion between pump and patient access site
Cause: Pump has detected excessive pressure in the administration set between the pump and the patient's access site.
Resolution: Check administration set from pump to patient access site for potential causes of occlusion. If tubing is kinked, straighten it. If filter is blocked, replace administration set. If patient access is obstructed, check it according to protocol. Consider changing the down occlusion pressure setting from Low to High. If the occlusion is cleared without the need to open the pump door, the pump automatically resumes its operation. Note: If the door of the pump is opened to clear the occlusion, (e.g. replacing the set), you must press the pause key, select "RESUME," and then press the RUN key to restart the infusion.

LCD Display: "***ALARM*** EMPTY BATTERY"

Alarm: Empty Battery
Cause: The C-cell battery no longer has sufficient power to run the pump. Alarm will not be displayed when external battery pack is depleted.
Resolution: Turn the pump off. Replace both of the C-cell batteries. Recharge or replace the battery pack if applicable.

LCD Display: "***ALARM*** HIGH UP PRESSURE"

Alarm: High positive pressure between the pump and IV reservoir
Cause: The pump has detected excessive positive pressure between the pump and the IV Source.
Resolution: Press PAUSE to clear alarm screen. Check for excessive pressure on the IV bag and relieve it. When ready to begin infusion, select "RESUME."

LCD Display: "***ALARM*** REPLACE SET 3"

Alarm: Replace Set
Cause: The pump has detected that the disposable administration set must be replaced.
Resolution: Replace the disposable administration set, prime the set if necessary, and when ready to begin infusing, select "RESUME" to continue the infusion.

LCD Display: "***ALARM*** REPLACE SET 4"

Alarm: Replace Set
Cause: The pump has detected excessive pressure in the disposable administration set.
Resolution: Make sure that all the IV clamps are open. Relieve pressure, open and close pump door.

- LCD Display:** “***ALARM*** SET NOT INSTALLED”
- Alarm:** Administration Set Not Installed.
- Cause:** The administration set has not been installed or has been installed improperly.
- Resolution:** Install appropriate **Moog Curlin Infusion** Administration set for desired therapy. Use **ONLY Moog Curlin Infusion** Administration sets. If the blue tubing guide on the administration set has been damaged in any way, replace the entire administration set. See Chapter entitled *Introduction*, Figures 1.9 and 1.10 for proper installation of administration set.
-
- LCD Display:** “***ALARM*** UNATTENDED”
- Alarm:** No key presses have occurred while the pump was paused.
- Cause:** When a therapy is in progress and a pump is placed in a paused state and left unattended for more than two minutes, this alarm will sound. If the prime menus are left unattended for one minute, this alarm will sound
- Resolution:** Press the “PAUSE” key to clear the alarm screen. If required, return to the screen displayed at the time of the alarm and complete the activity that was in progress, or resume the infusion, or turn the pump off.
-
- LCD Display:** “***ALARM*** UP OCCLUSION”
- Alarm:** Negative pressure detected between the pump and IV reservoir.
- Cause:** Pump has detected excessive negative pressure in the administration set between the medication reservoir or IV bag (“up stream”) and the pump.
- Resolution:** Pause the pump. Check administration set from IV bag to pump for potential causes of occlusion. Open the door of the pump and remove the administration set from the pumping chamber. If an area of the tubing is flattened, massage that area to a more rounded shape. If a clamp is in place, open it; if tubing is kinked, straighten it. Replace administration set back in the pump. When ready to begin infusing, select RESUME. If the alarm continues, use a new administration set or if the IV bag is almost empty, replace it.
-
- LCD Display:** “COMM TIMEOUT”
- Alarm:** Communication Timeout occurred during pump communication.
- Cause:** The communication link between the pump and the PC was broken (i.e., the cable between them was unplugged).
- Resolution:** Check the cable connection between the pump and the PDA or PC.
-
- LCD Display:** “INFUSION COMPLETE”
- Alarm:** Infusion Complete
- Cause:** Programmed amount to be infused is depleted to 0.
- Resolution:** If the therapy is to be repeated, press PAUSE, add another IV bag or reservoir and select the “REPEAT Rx” function.

If the therapy will not be repeated, press PAUSE key and turn the pump off by pressing the OFF key.

LCD Display: “STUCK BOLUS KEY”

Alarm: Bolus Key is stuck.

Cause: The Remote Bolus Cord is not functioning correctly.

Resolution: Replace the Remote Bolus Cord.

LCD Display: “***ALARM *** LOW MOTOR SUPPLY”

Alarm: Low Motor Supply Voltage.

Cause: The voltage to the Motor Supply is too low.

Resolution: Connect the AC adaptor or replace the batteries.

LCD Display: “***ALARM *** MOTOR STALLED”

Alarm: Motor Stalled

Cause: The motor is not turning at the expected rate.

Resolution: Connect the AC adaptor or replace the batteries and resume the infusion.

LCD Display: “***ALARM *** INVALID UNITS FOR PUMP CONFIGURATION”

Alarm: Invalid Units for Pump Configuration.

Cause: The Library Protocol units are not valid for the pump configuration

Resolution: Select a different Library Protocol or revise the Library Protocol.

LCD Display: “***ALARM *** INVALID ROUTE FOR PUMP CONFIGURATION”

Alarm: Invalid Route for Pump Configuration.

Cause: The Library Protocol Route is not valid for the pump configuration.

Resolution: Select a different Library Protocol or revise the Library Protocol.

LCD Display: “***ALARM *** SYSTEM TIMEOUT”

Alarm: System Timeout

Cause: Empty lithium battery

Resolution: Send pump to Moog or licensed service provider

ERROR CODES

If a pump goes into an error code, a numeric code will appear on the display screen. If this has occurred, turn the pump off and back on. If the error code message does not reappear, continue using the pump.

If the pump passes its start-up diagnostic tests, resume the therapy. If an error code recurs, turn the pump off, replace it, and report the error code to the healthcare provider.

Error Code	LCD Displayed Message	Probable Cause	Corrective Action
11	Turn pump OFF/ON	Keyboard Buffer Full	Report Error Code To Factory
12	Air Sensor Turn pump OFF/ON or Call provider	Air-In-Line Test Circuit Failure	Report Error Code To Factory
13	Pressure Sens DN Check Admin Set Turn Pump OFF/ON	Bad Down Strain Beam	Replace Set And Resume
14	Pressure Sens UP Check Admin Set Turn Pump OFF/ON	Bad Up Strain Beam	Replace Set And Resume
15	Turn Pump OFF/ON or Call Provider	Bad Encoder	Report Error Code To Factory
16	Turn Pump OFF/ON or Call Provider	Bad Analog to Digital Conversion	Report Error Code To Factory
17	Turn Pump OFF Call Provider	Bad Clock	Report Error Code To Factory
18	Call Provider	Bad Air In Line Comparator Register Reading From Fpga.	Report Error Code To Factory
19	Turn Pump OFF Call Provider	Bad Door Sensor	Report Error Code To Factory
20	Empty Lithium Battery Call Provider	Empty Lithium Battery	Report Error Code To Factory
21	Turn Pump OFF Call Provider	Bad Tristate Pin	Report Error Code To Factory
22	Turn Pump OFF Call Provider	Bad Test Pin	Report Error Code To Factory
23	Turn Pump OFF Call Provider	Bad Reverence Voltage	Report Error Code To Factory
24	Turn Pump OFF Call Provider	Bad Motor Supply	Report Error Code To Factory
25	Stalled Pump Turn Pump OFF/ON Call Provider	Broken Motor Shaft	Turn Power Off/On And Resume
26	Turn Pump OFF Call Provider	Bad Motor Inhibit	Report Error Code To Factory
27	Turn Pump OFF Call Provider	Bad Motor Supply Enable	Report Error Code To Factory
29	Low Alarm Batt Connect AC Adapt Turn pump OFF/ON	Low Auxiliary Battery	Connect Pump To Ac Battery Eliminator And Turn Power Off/On To Pump. May Require Turning Pump OFF/ON A Few Times
30	Turn Pump OFF Call Provider	Bad Auxiliary Battery	Report Error Code To Factory
31	Turn Pump OFF Call Provider	Audio Main Speaker Low	Report Error Code To Factory
32	Turn Pump OFF Call Provider	Audio Main Speaker High	Check/Replace D31. Replace PCB
33	Low Alarm Audio Connect AC Adapt Turn Pump OFF/ON	Audio Auxiliary Speaker Low	Connect Pump to AC Battery Eliminator and Turn Power OFF/ ON to Pump. May Require Turning Pump OFF/ON a Few Times
34	Turn Pump OFF Call Provider	Audio Auxiliary Speaker High	Report Error Code To Factory
35	History File Turn Pump OFF/ON	History Log Erased	Report Error Code To Factory

Error Code	LCD Displayed Message	Probable Cause	Corrective Action
36	Bad Cal Data Turn Pump OFF Call Provider	Bad EEPROM Data CRC value	Report Error Code To Factory
37	Turn Pump OFF Call Provider	Analog to Digital Converter did not Run	Report Error Code To Factory
38	Turn Pump OFF Call Provider	Air In Line Circuit not Tested	Report Error Code To Factory
39	Turn Pump OFF Call Provider	Bad EEPROM Data CRC 2	Report Error Code To Factory
41	Bad Cal Range Turn Pump OFF Call Provider	Bad EEPROM Limit	Report Error Code To Factory
45	Turn pump OFF/ON Call Provider	Motor Control Loss	Report Error Code To Factory
46	Turn Pump OFF/ON Call Provider	Motor Control Loss	Report Error Code To Factory
76	Turn Pump OFF/ON Call Provider	Battery Task Timeout	Report Error Code To Factory
100	Turn Pump OFF/ON Call Provider	Menu Not Found	Report Error Code To Factory
101	Turn Pump OFF/ON Call Provider	Mode Not Found	Report Error Code To Factory
102	Turn Pump OFF/ON Call Provider	Item Not Found	Report Error Code To Factory
103	Turn Pump OFF/ON Call Provider	Field Not Found	Report Error Code To Factory
104	Turn Pump OFF/ON Call Provider	Data Mismatch	Report Error Code To Factory
105	Turn Pump OFF/ON Call Provider	No Menu Pointer	Report Error Code To Factory
106	Turn Pump OFF/ON Call Provider	Count Out Of Range	Report Error Code To Factory
107	Turn Pump OFF/ON Call Provider	Display Task Queue Overflow	Report Error Code To Factory
108	Turn Pump OFF/ON Call Provider	Divide By Zero	Report Error Code To Factory
109	Turn Pump OFF/ON Call Provider	Concentration Zero Value	Report Error Code To Factory
110	Turn Pump OFF/ON Call Provider	Invalid Occurrence	Report Error Code To Factory
111	Turn Pump OFF/ON Call Provider	Multiple Param Struct Access	Report Error Code To Factory
112	Turn Pump OFF/ON Call Provider	Multiple Status Struct Access	Report Error Code To Factory
140	Turn Pump OFF/ON Call Provider	Infusion Volume Out Of Range	Report Error Code To Factory
141	Turn Pump OFF/ON Call Provider	Infusion Rate Out Of Range	Report Error Code To Factory
142	Turn Pump OFF/ON Call Provider	Infusion Time Out Of Range	Report Error Code To Factory
143	Turn Pump OFF/ON Call Provider	Volume Conversion Error	Report Error Code To Factory
144	Turn Pump OFF/ON Call Provider	Rate Conversion Error	Report Error Code To Factory
145	Turn Pump OFF/ON Call Provider	Time Conversion Error	Report Error Code To Factory
146	Turn Pump OFF/ON Call Provider	Bad PCA Dose Ratio	Report Error Code To Factory
160	Stuck PERIOD Key Turn pump OFF/ON or Call Provider	Stuck Period / Silence Key	If Key Was Pressed, Release It. Otherwise Report Error Code To Factory
161	Stuck ON OFF Key Turn Pump OFF/ON or Call Provider	Stuck ON/OFF Key	If Key Was Pressed, Release It. Otherwise Report Error Code To Factory

Error Code	LCD Displayed Message	Probable Cause	Corrective Action
162	Stuck 0 Key Turn Pump OFF/ON or Call Provider	Stuck 0 Key	If Key Was Pressed, Release It Otherwise Report Error Code To Factory
163	Stuck 1 Key Turn Pump OFF/ON or Call Provider	Stuck 1 Key	If Key Was Pressed, Release It Otherwise Report Error Code To Factory
164	Stuck 2 Key Turn Pump OFF/ON or Call Provider	Stuck 2 Key	If Key Was Pressed, Release It Otherwise Report Error Code To Factory
165	Stuck 3 Key Turn Pump OFF/ON or Call Provider	Stuck 3 Key	If Key Was Pressed, Release It Otherwise Report Error Code To Factory
166	Stuck 4 Key Turn Pump OFF/ON or Call Provider	Stuck 4 Key	If Key Was Pressed, Release It Otherwise Report Error Code To Factory
167	Stuck 5 Key Turn Pump OFF/ON or Call Provider	Stuck 5 Key	If Key Was Pressed, Release It Otherwise Report Error Code To Factory
168	Stuck 6 Key Turn Pump OFF/ON or Call Provider	Stuck 6 Key	Otherwise Report Error Code To Factory
169	Stuck 7 Key Turn Pump OFF/ON or Call Provider	Stuck 7 Key	If Key Was Pressed, Release It Otherwise Report Error Code To Factory
170	Stuck 8 Key Turn Pump OFF/ON or Call Provider	Stuck 8 Key	If Key Was Pressed, Release It Otherwise Report Error Code To Factory
171	Stuck 9 Key Turn Pump OFF/ON or Call Provider	Stuck 9 Key	If Key Was Pressed, Release It Otherwise Report Error Code To Factory
172	Stuck UP Key Turn Pump OFF/ON or Call Provider	Stuck Up Arrow Key	If Key Was Pressed, Release It Otherwise Report Error Code To Factory
173	Stuck NO Key Turn pump OFF/ON or Call Provider	Stuck No / Change Key	If Key Was Pressed, Release It Otherwise Report Error Code To Factory
174	Stuck YES Key Turn Pump OFF/ON or Call Provider	Stuck Yes / Enter Key	If Key Was Pressed, Release It Otherwise Report Error Code To Factory
175	Stuck DOWN Key Turn Pump OFF/ON or Call Provider	Stuck Down Arrow Key	If Key Was Pressed, Release It Otherwise Report Error Code To Factory
176	Stuck RUN Key Turn Pump OFF/ON or Call Provider	Stuck Run / Pause Key	If Key Was Pressed, Release It Otherwise Report Error Code To Factory
177	Stuck HELP Key Turn Pump OFF/ON or Call Provider	Stuck Help / Options Key	If Key Was Pressed, Release It Otherwise Report Error Code To Factory
178	Stuck PRIME Key Turn Pump OFF/ON or Call Provider	Stuck Prime / Bolus Key	If Key Was Pressed, Release It Otherwise Report Error Code To Factory
182	Turn Pump OFF/ON Call Provider	Invalid Key	Report Error Code To Factory
200	Turn Pump OFF/ON Call Provider	Invalid System State	Report Error Code To Factory
201	Turn Pump OFF/ON Call Provider	Bad Power Up Test Number	Report Error Code To Factory
202	Turn Pump OFF/ON Call Provider	Bad Power Up State	Report Error Code To Factory
203	Turn Pump OFF/ON Call Provider	Bad A to D Max Address	Report Error Code To Factory
204	Turn Pump OFF/ON Call Provider	Bad Run State	Report Error Code To Factory
205	Turn Pump OFF/ON Call Provider	Bad Pump Status	Report Error Code To Factory

Error Code	LCD Displayed Message	Probable Cause	Corrective Action
206	Turn pump OFF/ON Call Provider	Bad Volume Type	Report Error Code To Factory
207	Turn pump OFF/ON Call Provider	Bad Rate Type	Report Error Code To Factory
208	Turn pump OFF/ON Call Provider	Bad Motor Test Number	Report Error Code To Factory
209	Turn pump OFF/ON Call Provider	Bad Hist Type	Report Error Code To Factory
210	Turn pump OFF/ON Call Provider	Bad EEPROM Data ID	Report Error Code To Factory
211	Turn pump OFF/ON Call Provider	Bad run time CRC state	Report Error Code To Factory
212	Turn Pump OFF Call Provider	Bad EEPROM Write	Report Error Code To Factory
213	Turn Pump OFF Call Provider	Bad Total Volume	Report Error Code To Factory
214	Turn Pump OFF Call Provider	Bad Total Time	Report Error Code To Factory
215	Turn Pump OFF Call Provider	Bad Time Type	Report Error Code To Factory
216	Turn Pump OFF Call Provider	Bad Audio Level	Report Error Code To Factory
300	Turn Pump OFF Call Provider	Stack Overflow	Report Error Code To Factory
301	Turn Pump OFF Call Provider	Bad Flash	Report Error Code To Factory
302	Turn Pump OFF Call Provider	Bad Run Time Flash	Report Error Code To Factory
303	Turn Pump OFF Call Provider	Bad Ram	Report Error Code To Factory
305	Turn Pump OFF Call Provider	Bad Active Data	Report Error Code To Factory
307	Turn Pump OFF Call Provider	Therapy data CRC Failure	Report Error Code To Factory
308	Watch Dog Call Provider	Bad Watch Dog Test	Report Error Code To Factory
309	Turn Pump OFF Call Provider	Stopped Motor Motion Count	Report Error Code To Factory
310	Turn Pump OFF Call Provider	Stopped Motor Motion Volt	Report Error Code To Factory
311	Turn Pump OFF Call Provider	Bad Active Flags	Report Error Code To Factory
312	Turn Pump OFF Call Provider	Bad Active Flags 2	Report Error Code To Factory
313	Turn Pump OFF Call Provider	Negative Motor Motion	Report Error Code To Factory
314	Turn Pump OFF Call Provider	Therapy Params Write Violate	Report Error Code To Factory
315	Turn Pump OFF Call Provider	Therapy Status Write Violate	Report Error Code To Factory
316	Turn Pump OFF/ON Call Provider	Configuration Settings May Have Been Compromised	Try To Turn Off Pump Then On If Ok, Continue, If Error Recurs, Return For Service
317	Turn Pump OFF/ON Call Provider	Configuration Settings May Have Been Compromised	Try To Turn Off Pump Then On If Ok, Continue, If Error Recurs, Return For Service
320	Turn Pump OFF/ON Call Provider	Display String Oversize	Try To Turn Off Pump Then On If Ok, Continue, If Error Recurs, Return For Service
321	Turn Pump OFF/ON Call Provider	Critical Data Failure	Try To Turn Off Pump Then On If Ok, Continue, If Error Recurs, Return For Service
322	Turn Pump OFF/ON Call Provider	Task Execution Overrun	Try To Turn Off Pump Then On If Ok, Continue, If Error Recurs, Return For Service

Error Code	LCD Displayed Message	Probable Cause	Corrective Action
323	Turn Pump OFF/ON Call Provider	Interrupt Depth Overflow	Try To Turn Off Pump Then On If Ok, Continue, If Error Recurs, Return For Service
324	Turn Pump OFF/ON Call Provider	Invalid Malfunction Code	Try To Turn Off Pump Then On If Ok, Continue, If Error Recurs, Return For Service

Accessories

*There are a number of helpful accessories for use with the **Moog Curlin Infusion** pumps, and their descriptions and diagrams for use are included in this chapter.*

Soft Carry Packs

Please refer to our website at www.moog.com/medical to review the latest carry packs for use with Moog Curlin Infusion pumps.

Detachable Pole Clamps

The pole clamps attach to the Lockable Safety Shells or the Pump Holster and can accommodate a pole of .75 to 1.25-inch diameter. There is a **Standard Pole Clamp (340-0104)** as well as a **Lockable Pole Clamp (340-0125)** available.

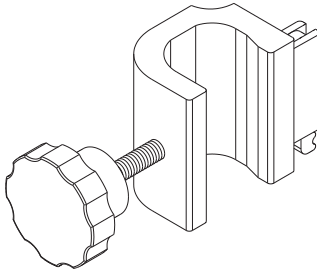


Figure 10.3 Detachable Pole Clamp

Remote Bolus Cord (340-2005)

This five-foot flexible cable offers a remote push button switch for the patient's use in administering bolus dosing in the PCA therapy. This accessory cannot be used in any therapy except PCA; and if it is left connected for any other therapy, an alert message will display on the pump screen, instructing the operator to remove this cable.

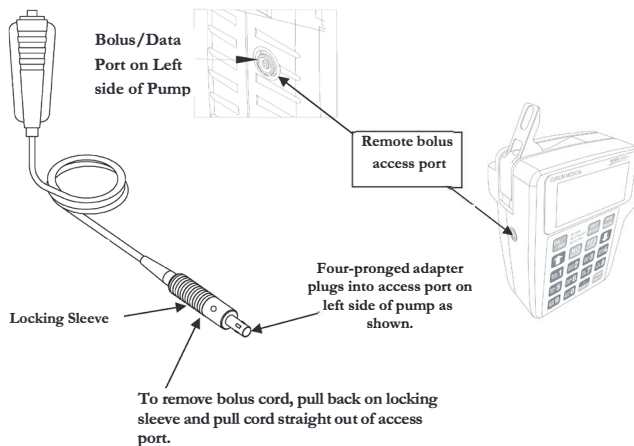


Figure 10.4 Remote Bolus Cord

Data Interface Cable (340-2011)

The data interface cable is used to download the history file of the pump into a PC with Windows 95 or greater using the hyperterminal communications program of MS Windows¹. An instruction booklet is shipped with the Interface Cable. It is also used for communications between the **Moog Curlin Infusion** CMS pump and the PC running the CMS program.

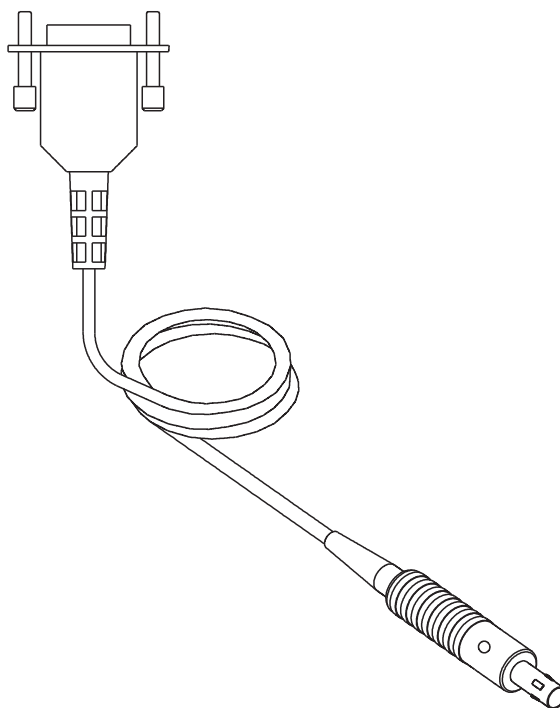


Figure 10.5 Data Interface Cable

¹ MS Windows is a registered trademark of Microsoft Corporation

Cleaning, Environmental Conditions, and Maintenance

*Following the guidelines for care listed below will assure the best appearance and longest life of the **Moog Curlin Infusion** pump.*

Cleaning

The **Moog Curlin Infusion** pumps are designed to be resistant to fluid spillage from any direction but are *not* to be immersed in solutions. Keep the pump and the pumping surfaces clean, dry, and free of fluid spillage at all times. Do not use the pump in the shower, sauna, or steam bath and do not position the pump where it could accidentally be dropped into a container of fluid, (e.g. basin, tub, or toilet). The pumping surfaces can be cleaned with a cloth dampened with any of the following solutions, and dried with a soft cloth without harming the pump.

Clean the pump, remote bolus cord, AC adaptor, safety shells, holster and battery pack with a soft cloth dampened in any of the following:

- Warm soapy water (do not submerge)
- Isopropyl alcohol
- Household bleach, diluted 9:1 with water
- Commercial disinfectant

(Do NOT spray the cleaning solution directly on the pump.)

IMPORTANT INFORMATION

The following cleaning/disinfecting methods are specifically prohibited:

- Do not use full strength bleach
- Do not autoclave or use steam sterilization
- Do not clean in a dishwasher
- Do not use acetone, lacquer thinner, or any other solvents or abrasive cleansers to clean the pump

With proper care, the pump will retain its finish for many years of service.

Transport and Storage

During transport or storage, the batteries should be removed, the pump should be cleaned and kept in its hard-shell packing case. The environmental conditions should be within the range of:

- An ambient temperature -20° F to 140°F
- A relative humidity range of 5%-95%; 375 to 795 mmhg
- An atmospheric pressure range of 500 Hpa to 1060 Hpa

All accessories should be stored in their original packing containers to ensure the longest service.

Operating Environment

The following conditions of ambient temperature range and relative humidity are recommended for the fluid delivered accuracy of:

- Temperature: ±5% Accuracy: 55 to 90°F
 ±10% Accuracy: 45 to 55°F
 ±10% Accuracy: 90 to 104°F
- Relative Humidity: 15 to 95% Non-condensing

Maintenance

Yearly preventive maintenance checks are recommended and will be prompted in the Setup Menu and on the display when the date occurs. This date accrues from the date of factory release and is reset when each annual maintenance check is completed.

Any time an error code message occurs repeatedly or any time a pump appears to be tampered with or shows any indication of not performing to standards, the pump should be returned to the healthcare provider dispensing the pump.

At no time should unauthorized personnel open the hard case of the pump.

Limited Warranty

ZEVEEX, Inc. (the manufacturer) warrants to the original purchaser that the infusion pump (the pump), not including accessories, shall be free from defects in material and workmanship under normal use, if used in accordance with this User's Manual, for a period of one year from the actual date of sale to the original purchaser. THERE ARE NO OTHER WARRANTIES.

The warranty does not cover normal wear and tear and maintenance items, and specifically excludes batteries, administration sets, extension sets or any other accessory items or equipment used with the pump.

Subject to the conditions of and upon compliance with this limited warranty, the manufacturer will repair or replace at its option without charge (except for a minimal charge for postage and handling) any pump (not including accessories) which is defective if a claim is made during such one year period.

The following conditions, procedures, and limitations apply to the manufacturer's obligation under this warranty:

1. **PARTIES COVERED BY THIS WARRANTY:** This warranty extends only to the original purchaser of the pump. This warranty does not extend to subsequent purchasers. The original purchaser may be a patient, medical personnel, or a medical facility that purchases the pump for treatment of patients. The original purchaser should retain the invoice or sales receipt as proof as to the actual date of purchase.
2. **WARRANTY PERFORMANCE PROCEDURE:** Notice of the claimed defect must be made in writing or by telephone to the manufacturer as follows: **Customer Service Department, ZEVEX, Inc., 4314 ZEVEX Park Lane, Salt Lake City, UT 84123, (801) 264-1001.** Notice to the Manufacturer must include date of purchase, model and serial number, and a description of the claimed defect in sufficient detail to allow the manufacturer to determine and facilitate any repairs that may be necessary. **AUTHORIZATION MUST BE OBTAINED PRIOR TO RETURNING THE PUMP,** and confirmed with issuance of an RMA (Return Materials Authorization) number. When authorized, the pump must be properly and carefully packaged and returned to the manufacturer, postage prepaid. Any loss or damage during shipment is at the risk of the sender.
3. **CONDITIONS OF WARRANTY:** The warranty is void if the pump has been 1) repaired by someone other than the manufacturer or its authorized agent; 2) altered so that its stability or reliability is affected; 3) misused; or, 4) damaged by negligence or accident. Misuse includes, but not limited to, use not in compliance with the User's Manual or use with non approved accessories. The pump is a sealed unit, and the fact that the seal has been broken will be considered conclusive evidence that the pump has been altered or misused. Removal or damage to the pump's serial number will invalidate this warranty.
4. **LIMITATIONS AND EXCLUSIONS:** Repair or replacement of the pump or any component part thereof is the **EXCLUSIVE** remedy offered by the manufacturer. The following and limitations shall apply:
 - a. No agent, representative, or employee of the manufacturer has authority to bind the manufacturer to any representation or warranty, expressed or implied.
 - b. **THERE IS NO WARRANTY OF MERCHANTABILITY OR FITNESS OR USE OF THE PUMP FOR ANY PARTICULAR PURPOSE.**
 - c. The pump can only be used under the supervision of medical personnel whose skill and judgment determine the suitability of the pump for any particular medical treatment.
 - d. All recommendations, information, and descriptive literature supplied by the manufacturer or its agents are believed to be accurate and reliable, but do not constitute warranties.

The manufacturer disclaims responsibility for the suitability of the pump for any particular medical treatment or for any medical complications resulting from the use of the pump. The manufacturer shall not be responsible for any incidental damages or consequential damages to property, loss of profits, or loss of use caused by any defect or malfunction of the pump.

This warranty gives the original purchaser specific legal rights, and the original purchaser may have other legal rights, which may vary from state to state.

Procedure for Checking Pumps Between Patients

Purpose

This procedure is provided for use by caregivers to ensure pumps meet all critical performance specifications prior to use of the product on new patients. This procedure is for use by technicians to inspect, test, and prepare a pump for use by a new patient. The test checks for volumetric accuracy and all pump safety systems. Also, various parameters are initialized.

Note: Between patient testing is not required for Moog Curlin Infusion pumps when anti-siphon valve administration sets are being used.

This procedure is intended to be used by technicians familiar with operation of the Moog Curlin 6000-series infusion pump.

Note: This procedure does not replace the recommended Annual Preventive Maintenance. The pump should receive preventive maintenance by a qualified technician at least annually.

Outpatient/Home Care/ Alternate Care Site Environment

If the pump is used in an outpatient, home care, or alternate Care setting, the pump must be checked between patients using the following procedure.

Inpatient/Hospital Environment

Moog Medical offers delivery sets with an anti-siphon valve for use in clinical situations where in between patient checks may not be practical.

Note: Between patient testing is not required for Moog Curlin Infusion pumps when *integral* anti-siphon valve administration sets are being used.

Moog Medical highly recommends that the hospital conduct in between patient checks using the following procedure ***if integral anti-siphon valve administration sets are not being used***. Should the hospital determine, for whatever reason, that the in between patient check is not practical, Moog Medical recommends performing volumetric accuracy checks at regular intervals, to ensure the pump continues to meet specifications.

Required Equipment

The following equipment is required for this procedure

- A NEW Administration Set (old or used sets are not suitable for the volumetric delivery and pressure measurements tests)
- Volumetric measurement device (i.e. calibrated weight scale, burette or syringes)

- Beaker with clean, fresh water or IV container.

Note: If the administration set contains a filter, sterile saline or sterile water should be used.

Maintenance Between Patients


This procedure does not replace the recommended preventive maintenance. The pump should receive preventive maintenance by a qualified technician annually.

1. CLEAN AND INSPECT PUMP FOR DAMAGE:

- a. Clean the pump with a soft cloth dampened with a designated antiseptic, disinfectant solution or a solution of household bleach diluted 9:1 with water. Be sure to clean all surfaces, including the pumping chamber.
- b. Inspect pump for damage. Check all case surfaces for cracks. Open the pumping chamber door. Examine the finger seal to confirm that no cuts or tears exist. Confirm that the door is well aligned by closing it with no unreasonable interference. Verify that the power and Bolus/Data connectors are securely mounted on the case (no loose nuts). Verify that all labels are legible and in good condition. Check that Keypad is securely attached to case and that no corners are peeling up. Verify that the battery door is in place and that the battery contacts in the battery compartment are not deformed.

2. VOLUMETRIC DELIVERY TEST:

Refer to the picture below when executing the following instructions.

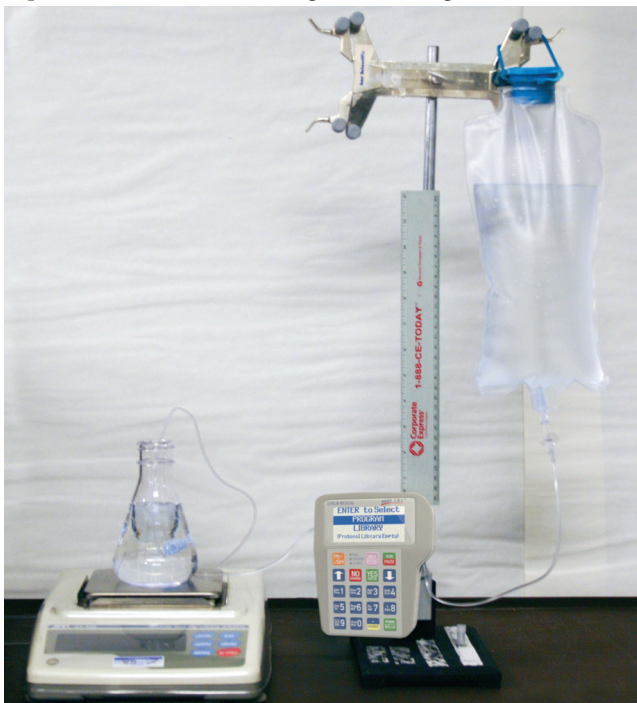

3. Volumetric Delivery Test:

- a. Install a new primed Administration Set in the pump and latch the door.
- b. Connect the up-stream side of administration set to a water source 12 inches +/- 2 inches above the pump.
- c. Connect the downstream side of the administration set to a volumetric accuracy measurement device (i.e. a calibrated scale or burette) at the same level as the pump +/- 2 inches.
- d. Program the following CONTINUOUS Infusion:

Pre Rx:		
UNITS:		mL
DELAY:		OFF
Medlimit:		OFF
NEXT?		YES
Prescription:		
BAG VOL:	100	mL
VOL TBI:	10.0	mL
RATE:	125	mL/hr
TIME:	0:04	HH:MM
KVO Rate:	0.0	mL/hr
DONE?		YES

- e. Start the continuous therapy running by pressing RUN button.
 - f. When the 10 mL infusion is over, observe the amount volume infused as measured on the measurement device. Passing measurement is: 9.5 g to 10.5 g (9.5mL to 10.5mL).
 - g. When "Infusion Complete" occurs verify that the audio alarm is beeping and that the green and red LEDs are flashing.
4. CHECK ALL SAFETY SYSTEMS:
- a. **Low Down Stream Occlusion** - Repeat the same Continuous Infusion as above. Go to the Option Screen and confirm that the Down Occlusion level is set to "LOW." Start the infusion and while the pump is running pinch off or close the down stream side of the administration set. Confirm the pump goes into Down Occlusion alarm. Open or clear the down stream occlusion and confirm the pump resumes infusion.
 - b. **High Down Stream Occlusion** – Go to the Option Screen and change the Down Occlusion level to "HIGH." Again pinch off or close the clamp on the down-stream side of the administration set. Confirm the pump goes into Down Occlusion. Open or clear the down stream occlusion and confirm the pump resumes infusion.
 - c. **Up Stream Occlusion** - Resume the infusion pumping then pinch off or close the up-stream side of the administration set. Confirm that the pump goes into an Up Occlusion alarm. Open or clear the up stream occlusion, press PAUSE then resume the infusion.
 - d. **Door Sensor** - With infusion in progress open the door latch and observe "ALARM DOOR OPEN OR SET NOT INSTALLED."

- e. **Air-In-Line Sensor** - Reinstall the administration set properly and resume the infusion. From the Options Menu set the AIL SENS (AIR IN LINE sensitivity) to 0.1mL. With the pump running introduce an air bolus of approximately 0.5mL into the fluid path at the input side of the pump. Once approximately 0.1mL of air has past the AIL detector, an "ALARM AIR IN LINE" will occur. PAUSE and PRIME the air bolus past the AIL sensor, then resume the infusion.
 - f. **Auxiliary Alarm** - With the pump running observe the "Vol INF" (volume infused on the run screen) and disconnect the pump from all power sources (unplug the AC Adaptor eliminator and remove one C-cell battery). Observe that the Auxiliary Alarm System is activated, with the red LED and beeper on constant (not intermittently beeping). With the battery still out of the pump, press the ON/OFF button to turn off the auxiliary alarm. Replace the battery in the battery compartment and turn on the pump. Resume the therapy and confirm that the infusion continues where it stopped due to Auxiliary Alarm occurrence.
5. SET BIOMED CLINICIAN MENU ITEMS:
- a. Turn the pump off and back on. While pump is performing Self Tests confirm that the pump is set to the correct date and time. If date and time are not correct, program the correct date and time from the BioMed/Clinician Menu.
 - b. Enter the BioMed/Clinician Menu using the Clinician Code.
 - c. Set the Lock Level to OFF in the CHANGE LOCK Menu.
 - d. Clear the Pt DATA from the pump.
 - e. Observe the pump maintenance date, top of the BioMed/Clinician Menu, to ensure adequate time for the next use by patient.
 - f. Sign, date and return the check list to your record file. Pumps that pass can be used for new patients. Pumps that fail are to be returned for service.

Technical Specifications





The following specifications apply to the **Moog Curlin Infusion** Ambulatory Infusion Pump:





ITEM	SPECIFICATION
Pumping Mechanism	Curvilinear peristaltic finger action
Pump Size Height: Width: Depth: Weight:	5 inches 4 inches 2.5 inches 17.5 ounces
Case Material:	UL rated flame-retardant, non-toxic high impact plastic
Operating Environment: Operating Temperature: Operating Relative Humidity:	± 5% Accuracy: 55° to 90°F ±10% Accuracy: 45° to 55°F ±10% Accuracy: 90° to 104°F 15 to 95% non condensing
Storage Environment: Storage Temperature Range: Storage Relative Humidity:	-20° to 140°F 5 to 95% non-condensing





Power Requirements	2 C-cell Single-Use Batteries AC Power										
ITEM	SPECIFICATION										
Operating Time with Full Charge Batteries	Approximately 10 hours at a rate of 400 mL/hr Approximately 30 hours at a rate of 125 mL/hr Approximately 85 hours at a rate of 2 mL/hr										
Internal Memory Battery	5 years life										
Display	Graphic LCD, 4 text fields, 16 characters per field										
Flow Rates	0.1 to 400 mL/hr										
KVO Rates	0 to 10 mL/hr. Can be set at "0" in Continuous, TPN, Variable and Intermittent Therapies										
Volume Accuracy	<p>±5% for volume delivered at rates ≥1 mL/hr. under the following conditions:</p> <ul style="list-style-type: none"> • Temperature 55° to 90° F • Back pressure to 300 mmHg (5.8 PSI) <p>±10% for volume delivered at rates ≥1 mL/hr under the following conditions:</p> <ul style="list-style-type: none"> • Temperature 45° to 55°F • Temperature 90° to 104°F • Back pressure to 750 mmHg (14.5 PSI) <p>±15% for volumes delivered at rates < 1 mL/hr.</p> <p>Sets containing air eliminating filters, when used at delivery rates below 1 ml/hr. may impact delivery accuracy an additional -3%</p> <p>The maximum volume that may be infused under Single Fault Conditions is 0.8 mL.</p> <p>Refer to administration set IFU for recommended time usage.</p> <p>*Follow standards of practice for set changes</p>										
Administration Sets	Proprietary Moog Curlin Infusion non-DEHP, PVC administration sets available in different configurations.										
Fill Volume for Administration Sets	Nonfiltered microbore sets ± 4 mL. See individual administration set label for specific fill volume.										
Up Occlusion Detection Time Microbore tubing with occlusion below spike	<table> <tr> <th>Rate</th><th>Time to Alarm</th></tr> <tr> <td>0.1 mL/hr</td><td><30 minutes</td></tr> <tr> <td>1 mL/hr</td><td>< 4 minutes</td></tr> <tr> <td>10 mL/hr</td><td><20 seconds</td></tr> <tr> <td>50 mL/hr</td><td><10 seconds</td></tr> </table>	Rate	Time to Alarm	0.1 mL/hr	<30 minutes	1 mL/hr	< 4 minutes	10 mL/hr	<20 seconds	50 mL/hr	<10 seconds
Rate	Time to Alarm										
0.1 mL/hr	<30 minutes										
1 mL/hr	< 4 minutes										
10 mL/hr	<20 seconds										
50 mL/hr	<10 seconds										





ITEM	SPECIFICATION		
Down Occlusion Detection Time Microbore tubing with occlusion at distal Luer	For non-filtered sets:		
	<u>Rate</u>	<u>Low Pressure</u>	<u>High Pressure</u>
	0.1 mL/hr	≤ 1 hr. 30 minutes	≤ 2 hrs. 15 minutes
	1 mL/hr	≤ 7 minutes	≤ 16 minutes
	10 mL/hr	≤ 40 seconds	≤ 1 min. 30 seconds
	50 mL/hr	≤ 6 seconds	≤ 15 seconds
	125 mL/hr	≤ 3 seconds	≤ 5 seconds
	For filtered sets:		
	<u>Rate</u>	<u>Low Pressure</u>	<u>High Pressure</u>
	0.1 mL/hr	≤ 9 hr. 15 minutes	≤ 20 hrs. 30 minutes
	1 mL/hr	≤ 50 minutes	≤ 2 hrs. 5 minutes
	10 mL/hr	≤ 4 min. 30 seconds	≤ 10 min. 30 seconds
	50 mL/hr	≤ 50 seconds	≤ 3 minutes
125 mL/hr	≤ 20 seconds	≤ 50 seconds	
Programmable Time (HH:MM) Continuous TPN Intermittent Variable PCA	00:01 to 288:00 HH:MM		
	00:01 to 288:00 HH:MM		
	00:01 to 288:00 HH:MM		
	00:01 to 288:00 HH:MM		
	No time limit		
Bolus Volumes Microbore tubing with occlusion at distal Luer	Low Pressure: <0.1 mL		
	High Pressure: <0.1 mL		

Guidance and manufacturer's declaration – electromagnetic emissions		
The Curlin Infusion Pump is intended for use in the electromagnetic environment specified below. The customer or the user of the Curlin Infusion Pump should assure that it is used in such an environment.		
Emission test	Compliance	Electromagnetic environment- guidance
RF emissions CISPR 11 EN 55011	Group 1	The Curlin Infusion Pump must emit electromagnetic energy in order to perform its intended function. Nearby electronic equipment may be affected.
RF emissions CISPR 11 EN 55011	Class B	The Curlin Infusion Pump is suitable for use in all establishments including domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

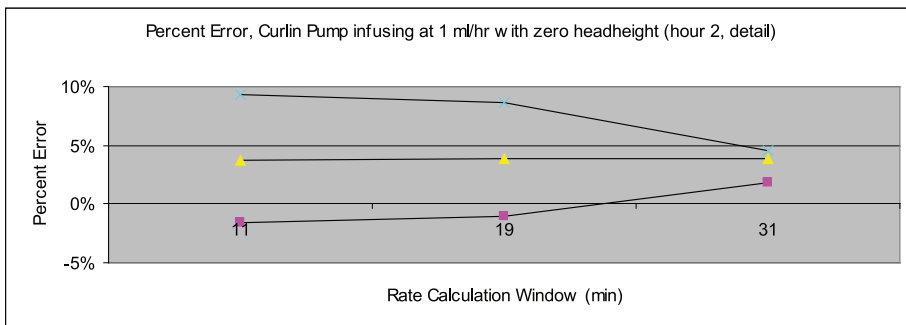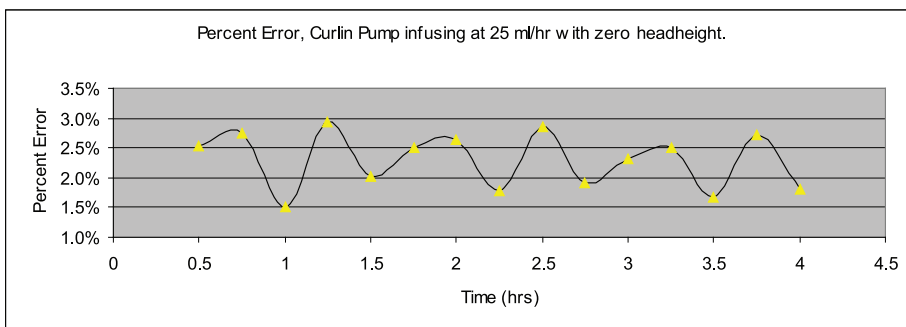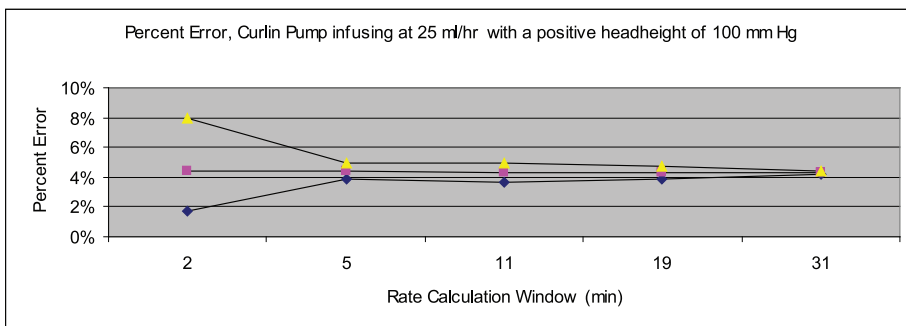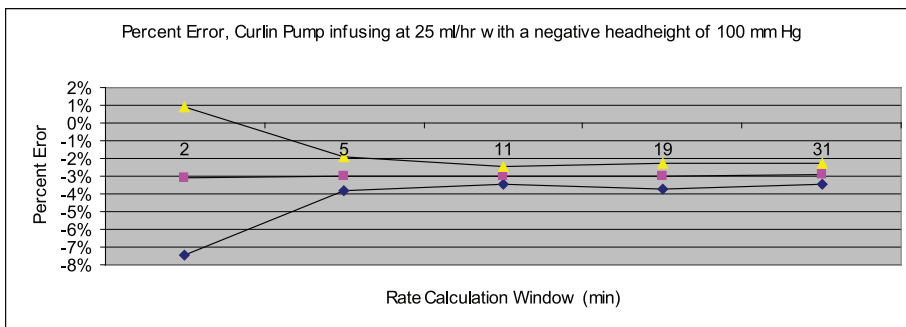


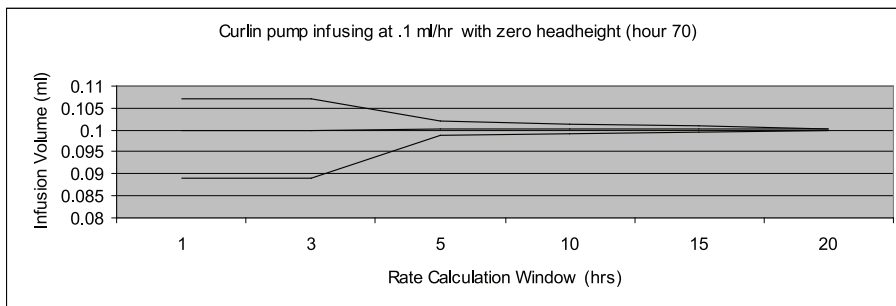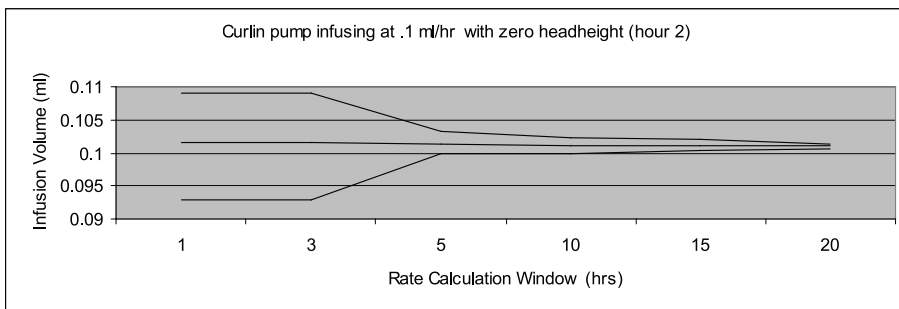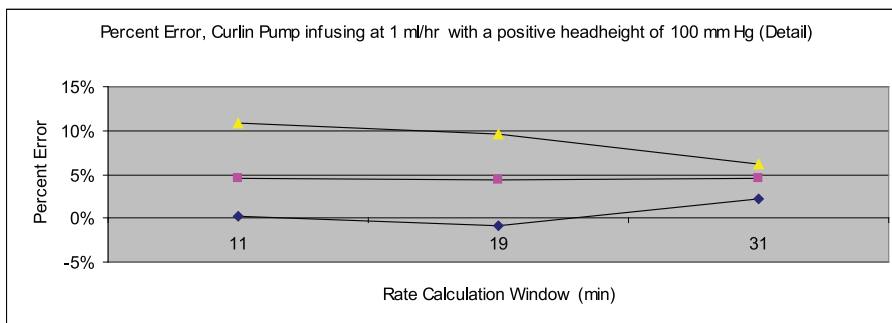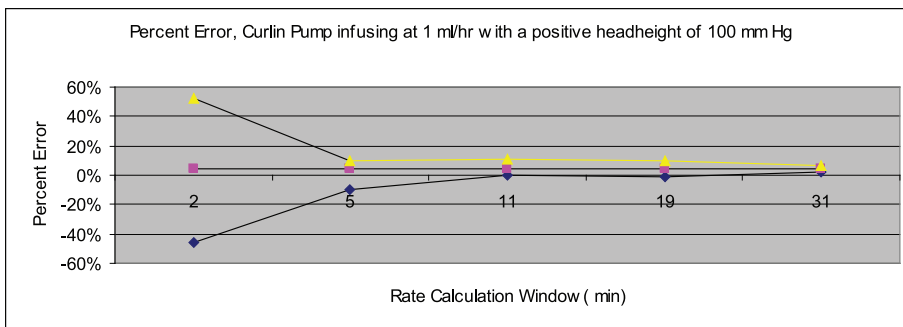

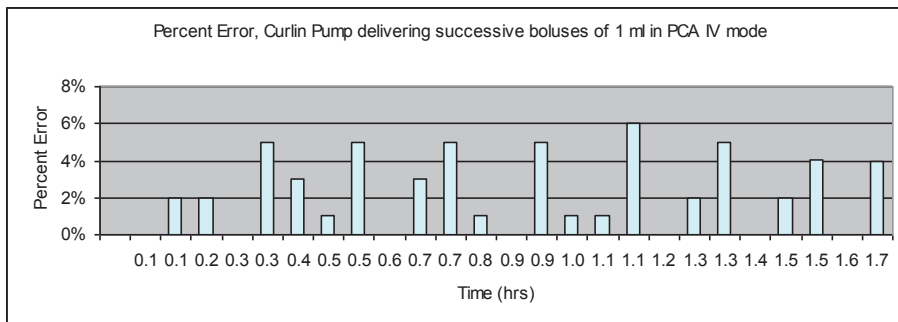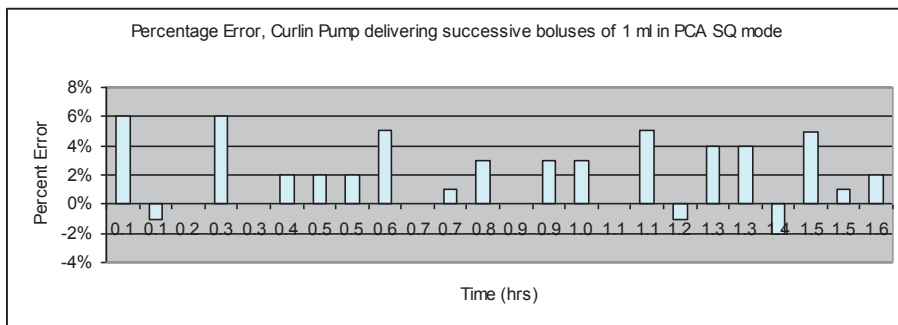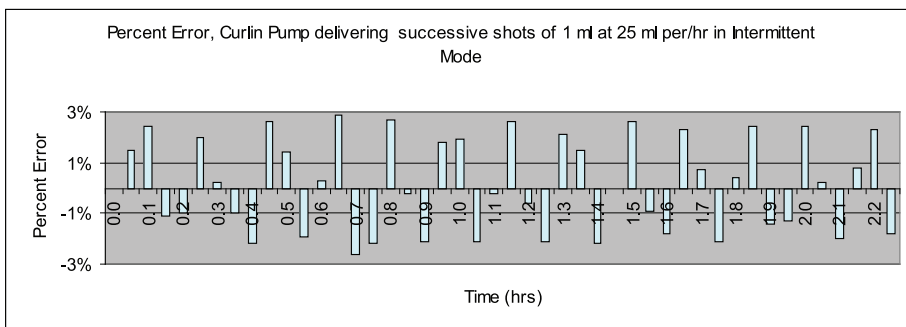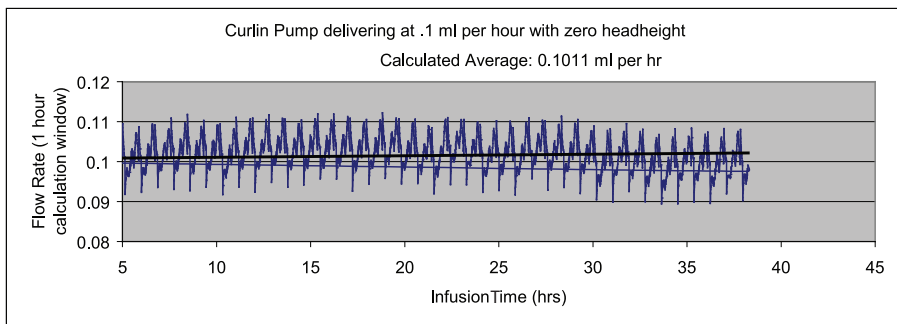


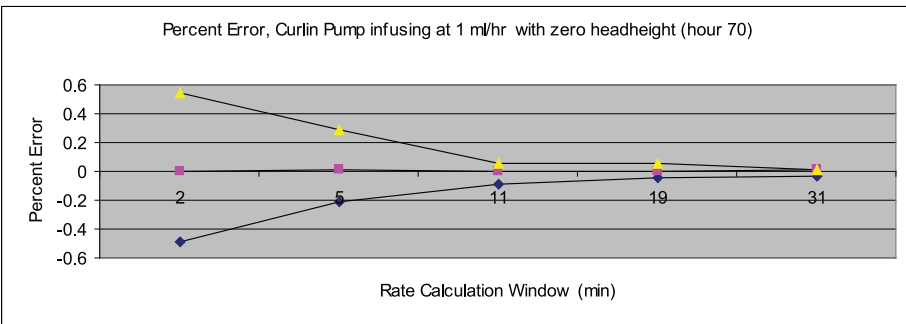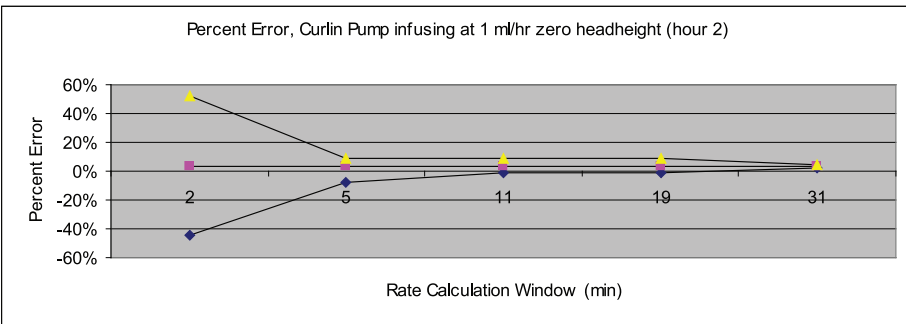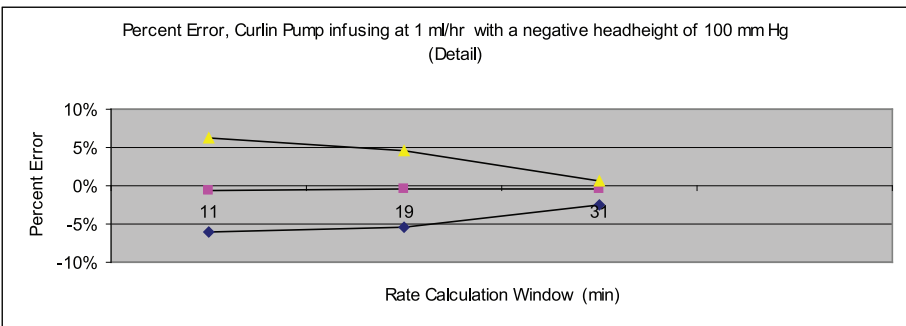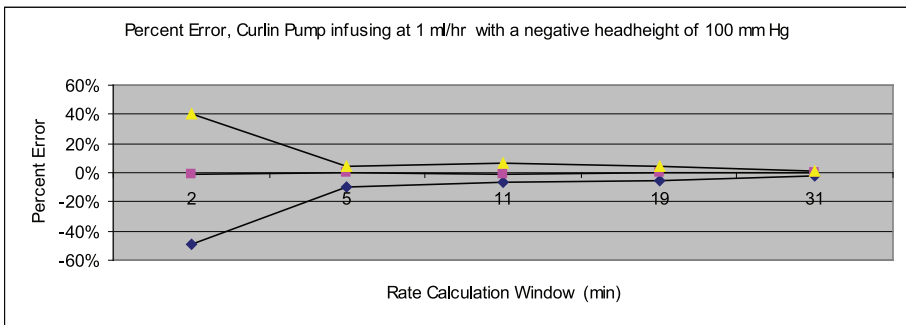

Curlin Infusion pump delivering in TPN mode at 25 ml/hr with 1 hour ramp up and 1 hour ramp down

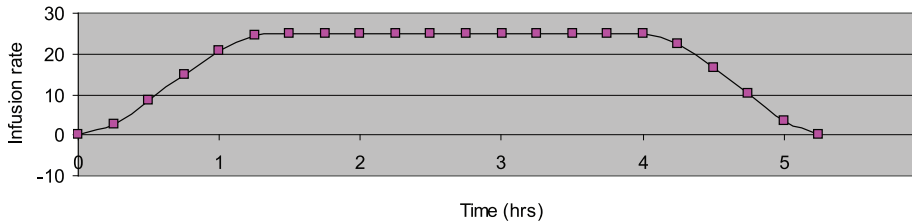




Percent Error, Curlin Pump infusing at 1 ml/hr no headheight (hour 70, detail)

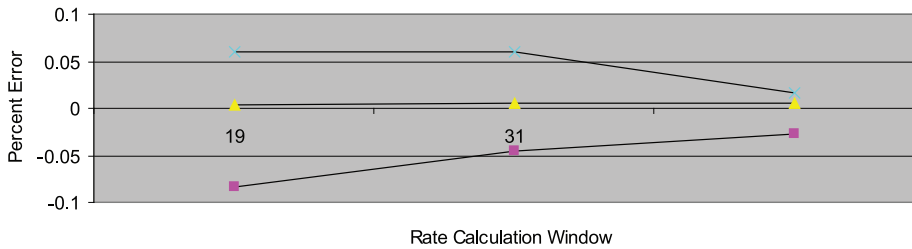

Curlin Pump delivering at .1 ml per hour with zero headheight (detail)


Calculated Average: 0.1011 ml per hr

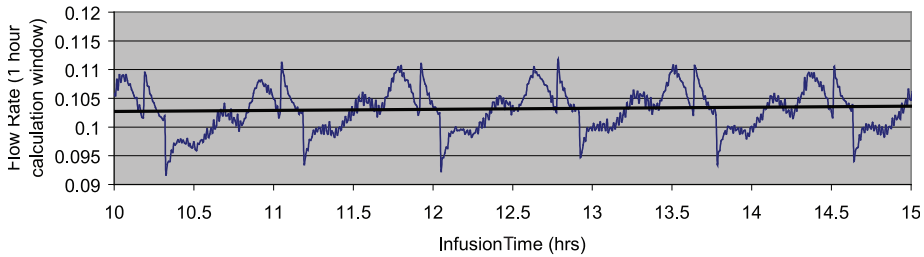

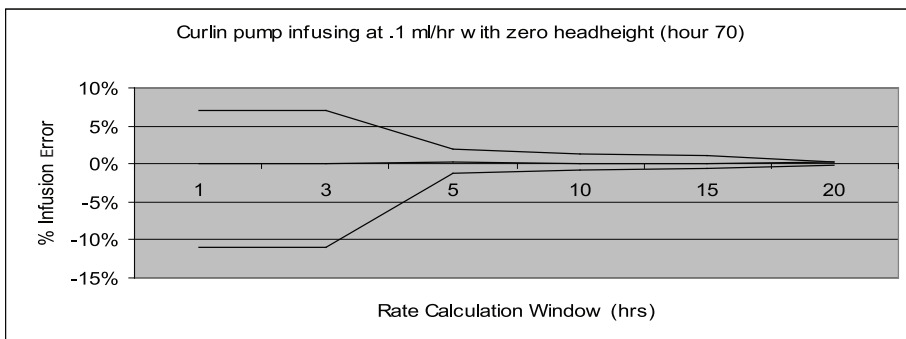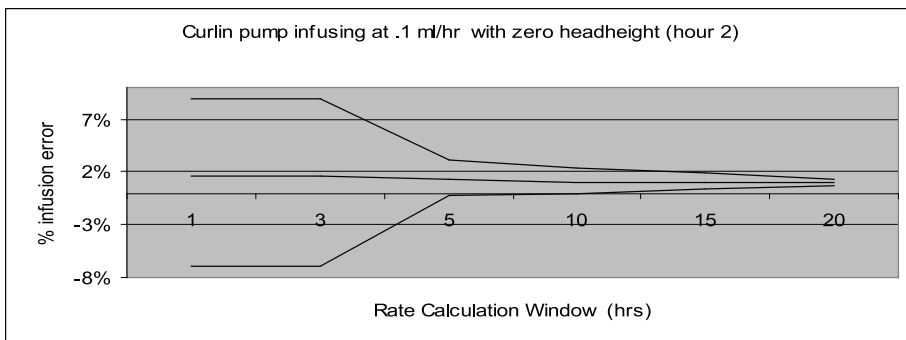

Guidance and manufacturer's declaration - electromagnetic immunity			
The PainSmart™ IOD, 6000 CMS™ and 6000 CMS™ IOD™ is intended for use in electromagnetic environment specified below. The customer or the user of the PainSmart™ IOD, 6000 CMS™ and 6000 CMS™ IOD™ should assure that it is used in such an environment.			
IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic Discharge (ESD) IEC 61000-4-2	2, 4, 6 kV contact 2, 4, 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast Transient / burst IEC 61000-4-4	± 2 kV AC mains ± 1 kV for signal or data	± 2 kV for power supply lines ± 1 kV for for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	Mains: ± 1 kV line(s) to line(s) ± 2 kV line(s) to earth Signal ports: 1 kV lines(s) to earth DC I/O ports: 0.5 kV line(s) to line(s) 0.5 kV line(s) to earth	1 kV line(s) to (lines) ± 2 kV line(s) to earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short Interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5 % U _i (>95% dip in UT) for 0,5 cycle 40 % U _i (60 % dip in UT) for 5 cycles 70 % U _i (30 % dip in UT) for 25 cycles <5 % U _i (>95% dip in UT) for 5 s	<5 % U _i (>95% dip in UT) for 0,5 Cycle 40 % U _i (60% dip in UT) for 5 cycles 70 % U _i (30% dip in UT) for 25 cycles <5 % U _i (>95% dip in UT) for 5 s	Mains power quality should be that of a typical commercial or hospital environment. If the user of the PainSmart™ IOD, 6000 CMS™ and 6000 CMS™ IOD™ requires continued operation during power mains interruptions, it is recommended that the PainSmart™ IOD, 6000 CMS™ and 6000 CMS™ IOD™ be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) Magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE: U _i is the a.c. mains voltage prior to application of the test level.			

Recommended separation distances between portable and mobile RF communications equipment and the Painsmart™ IOD, 6000 CMS™ and 6000 CMS™ IOD™			
The Painsmart™ IOD, 6000 CMS™ and 6000 CMS™ IOD™ is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Painsmart™ IOD, 6000 CMS™ and 6000 CMS™ IOD™ can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Painsmart™ IOD, 6000 CMS™ and 6000 CMS™ IOD™ as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter		
	150 KHz to 80 MHz $d = \left[\frac{3,5}{V_1} \right] \sqrt{P}$	80 MHz to 800 MHz $d = \left[\frac{3,5}{E_1} \right] \sqrt{P}$	800 MHz to 2,5 GHz $d = \left[\frac{3,5}{E_1} \right] \sqrt{P}$
0.01	0.116	0.12	0.23
0.1	0.369	0.34	0.74
1	1.16	1.7	2.3
10	3.69	3.7	7.4
100	11.7	11.7	23.3
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where p is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer. Note 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies. Note 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			


Guidance and manufacturer's declaration - electromagnetic immunity			
The PainSmart™ IOD, 6000 CMS™ and 6000 CMS™ IOD™ is intended for use in electromagnetic environment specified below. The customer or the user of the PainSmart™ IOD, 6000 CMS™ and 6000 CMS™ IOD™ should assure that it is used in such an environment.			
IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz 1 KHz 80% AM modulation	[V _i] V 3V / m	Portable and mobile RF communications equipment should be used no closer to any part of the Painsmart™ IOD, 6000 CMS™ and 6000 CMS™ IOD™ including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = \left\lceil \frac{3.5}{E_1} \sqrt{P} \right\rceil$ d = 1.17 · √P 80 MHz to 800 MHz $d = \left\lceil \frac{3.5}{E_1} \sqrt{P} \right\rceil$ 80 MHz to 800 MHz $d = \left\lceil \frac{7}{E_1} \sqrt{P} \right\rceil$ 800 MHz to 2,5 GHz d = 2.33 · √P 80 MHz to 2.56 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range Interference may occur in the vicinity of equipment marked with the following symbol: 
Radiated RF IEC 61000-4-3	2 V / m 150 MHz to 2,5 GHz 1 KHz 80% AM modulation	[E _i] V / m 3V / m	
NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.			
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
<ul style="list-style-type: none">Field strengths from transmitters, such as base stations for radio (cellular / cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the [ME EQUIPMENT or ME SYSTEM] is used exceeds the applicable RF compliance level above, the [ME EQUIPMENT or ME SYSTEM] should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the [ME EQUIPMENT or ME SYSTEM].Over the frequency range 150 kHz to 80 MHz, field strengths should be less than [V_i] V / m.			

Clinician Information and Access Code

*This chapter should be removed before initial patient training and instruction on the **Moog Curlin Infusion** pump so the intended locked functions or secured functions of the pump cannot be accessed except by a trained clinician. Removal of this chapter will assist in patient safety.*

Whenever it is necessary for the clinician to “unlock” a secured lock setting the following screen will appear (NOTE: For access code, contact the dispensing pharmacy.)

Clinician Access Code

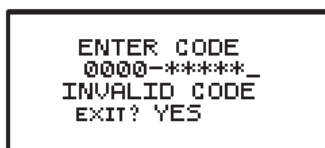

The clinician enters a four-digit personal identification number followed by the five-digit access code. If the five-digit code is valid, the display will automatically move to the requested screen or field. The four-digit personal identification number appears on the screen but the access code numbers are displayed as five asterisks. **The personal identification number is recorded in the history file for the purpose of providing an audit trail.** If an invalid access code is accidentally entered, a message displays: “Invalid Code” and the clinician can reenter the correct code.

Note: After entering the access code, DO NOT press the YES/ENTER key. The pump will automatically recognize the correct code and give you access to the secured section. Entering the clinician access code **does not** change the lock setting of the pump but does allow the clinician to perform the locked-out function.

Lock Setting Table

The following table illustrates which features are controlled under which lock settings:

“No” indicates that the correct access code must be entered in order to perform the pump operation.

Pump Operation	Active Therapy	Lock OFF	Lock 1	Lock 2	Lock 3
Change Lock Settings	All	All settings with correct access code			
Turn Pump Off/On	All	Yes	Yes	Yes	Yes
Pause/Run Therapy	All	Yes	Yes	Yes	Yes
Access Help/Options	All	Yes	Yes	Yes	Yes
Check Power Source	All	Yes	Yes	Yes	Yes
Resume Therapy	All	Yes	Yes	Yes	Yes
View Rx Screens at “No to Review”	All	Yes	Yes	Yes	Yes
Repeat Therapy	All	Yes	Yes	Yes	No
Prime Pump	All	Yes	Yes	No*	No
Change Audio Level	All	Yes	Yes	Yes	No
Change Rx Screen Information	All	Yes	Yes	No	No
Change Rx Screen at “No to Review”	All	Yes	Yes	No	No
Access New Program	All	Yes	No	No	No
Change Air-In-Line Setting	All	Yes	No	No	No
Change Down Pressure	All	Yes	No	No	No
Change KVO Rate	CON,TPN, INT, VAR	Yes	Yes	No	No
Delay Start	CON,TPN, INT, VAR	Yes	No	No	No
Change Units (New Program)	CON,PCA, INT, VAR	Yes	No	No	No
Change Concentration (New Program)	CON,PCA, INT, VAR	Yes	No	No	No
Utilize MedLMTS™ Option	CON, PCA	Yes	Yes	No	No
Program MedLMTS™ Limits	CON, PCA	Yes	No	No	No
Change Administration Route (New Program)	PCA	Yes	No	No	No
Program Loading Dose	PCA	Yes	No	No	No
Program Clinician Dose	PCA	Yes	No	No	No
Access Early Down Ramp	TPN	Yes	Yes	Yes	Yes
Change Next Dose Time	INT	Yes	Yes	No	No

* Lock Level 2 only requires an access code to prime in the PCA mode

Note: To determine the software version of the pump, turn the pump on and review the screen that appears after the “Moog 6000” screen and immediately before the “POWER SOURCE INTERNAL POWER” screen appears.

Whenever a Lock Level setting is placed in the pump it is retained unless:

1. The setting is changed by using either the “BioMed Setup” function or the OPTIONS function.

Note: Entering a New Program will NOT change the Lock Level from its current setting.

- 2. The user selects a preprogrammed protocol from a Protocol Library Safety System enabled pump. In this case, the pump defaults to the Protocol and erases the Lock Level setting.**

If a locked-out function is requested, such as PRIME in lock setting 3, a screen will appear requesting the entry of the access code. When the correct code is entered, the locked out function can be performed by the clinician.

Note: In any function outside of the field to change the Lock Level, entering the clinician access code will not change the Lock Level setting. Once the pump recognizes the clinician access code, it authorizes the function to be performed without the need to change the Lock Level setting.

