

## PCA Mode

### Programming the Pump

- Turn the pump on and wait for the self-test to complete.
- Press **YES** to Highlighted **LIBRARY**. (*Enter code if asked*)
- Select **NEW PROGRAM**.
- Highlight the desired category and press **YES**.
- Select drug or solution from listed protocols by highlighting and press **YES**.
- Press **YES** on screen that says: Alert Rx PROGRAM, press **YES** to erase Rx.
- If advisory listed make note and press **YES**.
- Accept units: Press **YES** to **mg**, unless **mcg** or **mL** is desired.
- Accept **Concentration of drug**, press **YES**.
- Accept Admin Route, press **YES**.
- Enter/Accept **Load dose** if ordered, and then press **YES**. (*If none then press YES to 0.0mL.*)
- At NEXT? Press **YES**.
- Accept **Bag volume default**, and then press **YES**.
- Accept **Basal rate** (if pre- programmed) then press **YES**.
- Accept **Pt. Bolus** then press **YES**.
- Accept **Bolus interval** (lockout) then press **YES**.
- Accept **# of Boluses/hr** then press **YES**.
- At **Done?** press **YES**.

Press **NO** to Review the Program.

Press **YES** to confirm each setting.

(Review the program prior to Priming.)

### Priming the line

- At the RUN/OPTIONS screen, press the **Prime key** and release.
- Press **Prime key** and Hold.
- Release the **Prime key** when priming is completed.
- Press **YES** to exit priming.

### Changing an existing program.

- Press the **Pause** key.
- Press **YES** to Resume.
- Press **NO** to Review.
- Press **YES** to confirm all settings until field to be changed is reached.
- At the field to change, enter the new value, then press **YES** to accept the change.
- Press **YES** to all other highlighted areas if no other changes are to be made.
- At **Done?** press **YES**.
- Press **RUN** to restart.

### Changing the Solution Container

- Press **PAUSE** key.
- Scroll down to **REPEAT Rx**, press **YES**.
- Answer any on screen questions.
- Note advisory, if any. Press **YES**.
- Press **NO** to Review the program as needed.
- Change the solution container.
- Press **RUN** to start.

## Clinician Dose (Clinician Given Dose)

While pump is running, press the “HELP/Options” key.

- Scroll down to **Clin Dose**.
- Press **NO** to turn OFF to ON, then press **YES**.
- Enter Access Code if asked, then press **NO** again to turn on then press **YES** to accept.
- Enter desired dose, then press **YES**.
- Press **YES** to Accept.
- Press **YES** again to Accept Option?. Pump will then give programmed dose.

## Hourly Totals

Using the IOD keys, press the **No. 5** key while the pump is infusing and view the hourly totals displayed on the screen for several seconds. (BG=Boluses Given, BA=Boluses Attempted.) Or-

While pump is running, press the “HELP/Options” key.

- Scroll down to **HRLY TOT**.
- Press **NO** to turn OFF to ON then press **YES**.
- Review Hourly amounts infused by using the **UP** arrow on the key pad.
- Press **NO** to escape the hourly menu.
- Press **YES** to Accept Options?

## Shift Totals/Clear Shift Totals

Using the IOD keys, press the **No. 9** key while the pump is running and press **YES** to clear totals while **Clear?** is flashing on the screen. Or-

While pump is running, press the “HELP/Options” key.

- Scroll down to **SHIFT TOT**.
- Press **NO** turn OFF to ON, then press **YES**.
- After reviewing, press **NO** to **Exit?**
- Then **CLEAR?** will flash. Press **YES** to clear the shift totals.
- Now press **YES** to **Exit?**
- Press **YES** to Accept Options?

Moog Terms	Definition
Loading Dose	Dose given at start of therapy only
Basal Rate	Continuous Rate
Pt Bolus	PCA Demand Dose
Bolus Interval	Lockout Time between bolus doses
# of Boluses/Hr	Number of total boluses allowed per hour
Clinician Dose	Clinician given dose

Moog is a registered trademark of Moog, Inc. and its subsidiaries. All trademarks indicated herein are the property of Moog, Inc. and its subsidiaries. © 2011 Moog, Inc. All rights reserved. 46279 Rev. A

# MOOG®

MEDICAL DEVICES GROUP

## Clinician Pocket Guide



**Curlin PainSmart IOD**  
**Library Enabled (V2.01)**

**PCA MODE**

**(Clinical support 1-888-287-5999)**

[www.moog.com/medical](http://www.moog.com/medical)