

## **Commercial Airline Travel with Moog Curlin Ambulatory Pumps**

Use of the Moog Curlin Ambulatory Pump is allowed and will not be adversely affected when traveling on commercial airlines. Please note the following considerations when traveling:

- Obtain a physician note prior to traveling if the Curlin Ambulatory Pump is clinically required.
- When going through security, notify security personnel that you are traveling with a medical device. The pump can be x-rayed for the purpose of screening without adversely affecting the pump.
- The Moog Curlin Ambulatory Pump can be used in a pressurized airline cabin. Moog does not recommend use in non-pressurized cabins.
- The Moog Curlin Ambulatory pump will radiate electronic frequencies that are similar to a computer. It is possible that the airline crew could request the infusion pump be turned off during take-off and landing. If the patient's healthcare provider determines it is clinically unacceptable to interrupt the infusion during these travel segments, the patient / caregiver should have a signed letter from their physician.
- The patient should not experience any nuisance alarms during take-off or landing of the aircraft.

Please contact the Customer Service Group at 1-800-970-2337 if there are further questions regarding this issue.