



MOOG[®]

CURLIN 6000 & PAINSMART[®] INFO ON DEMAND

Basal Rate: Background continuous rate
Bolus: Patient activated bolus dose
Load Dose: Given one time at start of infusion
Clin Dose: Dose given by clinician

INFORMATION ON DEMAND KEYS:

Information On Demand (IOD) is only available when the pump is infusing.

- 1** Displays the date and time set in the pump
- 2** Displays the battery level of the C cell batteries
- 3** Displays Lock Level, Down Occlusion setting, Air Sensor setting and Audio Level
- 4** Holds the current screen 6 seconds for ease of review
- 5** Displays up to 24 previous hours of infusion data (BG=boluses given, BA=boluses attempted)
- 6** Displays the drug name (if applicable), units, concentration, and administration route
- 7** Displays the basal, bolus, bolus interval and # boluses/hr
- 8** Displays the bolus volume as a % of the total volume infused
- 9** Displays the infused totals since last cleared. Enables the user to clear shift totals*
- 0** Displays an approximation of the total amount of medication available within one hour

** TO CLEAR SHIFT TOTALS: Press the CLR Shft/#9 key. Record needed information (amount inf, volume inf, clin doses, boluses given of boluses attempted). Then press the YES/ENTER button in response to the "CLEAR" question. If ":CLEAR" is not displayed on the screen, press #9 key again and press YES. The pump will go back to the running screen. Press the CLR Shft/#9 key again to verify that the values are reset or cleared.*



HANGING NEW CONTAINER:

- Press **PAUSE**
- When cursor is on **REPEAT RX**, press **YES**, Pump will auto-scroll through prescription
- When new container is in place press **RUN**
(Values on running screen start over from zero)

RESUMING THE SAME CONTAINER:

- Press **PAUSE**
- When cursor is on the **RESUME** press **YES**
- Press **RUN** to resume infusion

CHANGING AN EXISTING RX

Bag Volume, Basal, Pt Basal, Bolus Interval and/or # Boluses/hr

- Press **PAUSE**
- Press **YES** to **RESUME**
- Press **NO** key to **REVIEW** program
- Press **YES** to accept each line and advance cursor
- Press **NO** to make changes to program as needed (If locked, enter access code)
- Change data as needed and press **YES** to accept
- Continue confirming each line by pressing **YES**
- Press **YES** when cursor is on **DONE**
- Press **RUN** to start infusion

GIVING A CLINICIAN DOSE

- Press **HELP/OPTIONS** key (pump running)
- Move cursor to **CLIN DOSE**
- Press **NO/CHANGE** key to change from “OFF” to “ON” (If locked, enter access code and repeat previous step)
- Press **YES** to access Clinician Dose Screen
- Enter Clinician Dose Amount and press **YES**
- Press **YES** to accept and exit the screen
- Press **YES** at ACCEPT OPT to exit Options

To cancel a Clinician Dose

- Press **PAUSE**
- Press **YES** to **RESUME**
- Press **OPTIONS**
- Cursor down to Clin Dose, press **NO** to **OFF**, **YES** to **ON**
- Enter Clinician Dose Amount to Zero and Press **YES**
- Press **YES** to accept and exit the screen
- Press **YES** at **ACCEPT OPT** to exit Options

STARTING A NEW PROGRAM

- Press **PAUSE**
- Cursor down to **NEW PROGRAM** press **YES**
(if Access Code is required, enter it)
- Enter new values and press **YES** at each field
- Press **YES** when cursor is on **DONE**
- Press **RUN** to start infusion

*Moog Medical Devices Group,
4314 Zevek Park Lane, Salt Lake City, UT 84123*

Moog is a registered trademark of Moog, Inc. and its subsidiaries. All trademarks indicated herein are the property of Moog, Inc. and its subsidiaries.

© 2018 Moog, Inc. All rights reserved. LS 49237 Rev. C