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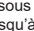
Infinityorange®
Small Volume
Enteral Feeding Pump
by MOOG

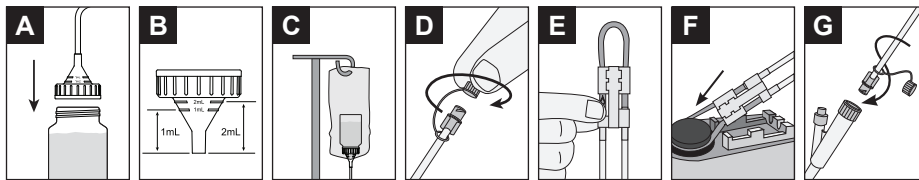
REF INFOCAP-E

Zevey, Inc. Salt Lake City, Utah 84123 USA
USA toll free: (800) 970-2337 tel: (801) 264-1001
www.infinityfeedingpump.com

Screw Cap Enteral Pump Delivery Set with ENFit® Connector

Use INFOCAP-E with Infinity Orange Enteral Feeding Pump only

INSTRUCTIONS FOR USE: To ensure proper operation of the delivery set, please follow all instructions and illustrations below.

- Fill a feeding bottle/container with a 40mm opening and secure to the Infinity Orange screw cap (image A).
Note: The screw cap includes 1ml and 2ml graduation markings on the funnel. These markings indicate an approximate volume in the screw cap; they do not include volume in the tubing (image B).
- Invert bottle/container, place it in the suspension bag and secure bag to an IV pole (image C).
- Unscrew the protective cover counterclockwise from the ENFit connector (image D).
- Gently pinch the orange colored tubing **below** the  symbol (image E). Hold this position until air is expelled from tubing. If fluid does not flow, pinch may be too tight.
- To expel air from tube using pump prime feature, refer to pump and Operator's Manual for instructions.
- Position silicone tubing around pump wheel stretching lightly. Seat cassette into the pump (image F). Close pump door.
- Srew the ENFit connector clockwise into feeding tube firmly by hand (image G). Be careful not to overtighten.

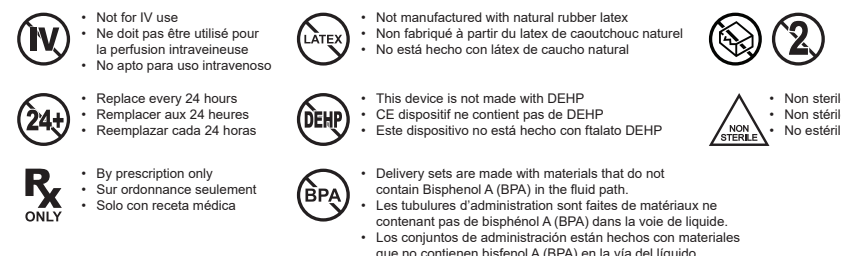
Note: Priming with the pump is recommended when using thick formulas because it is faster than priming by gravity.

WARNINGS AND PRECAUTIONS

- It is important to keep the air sensor free of debris including spilled fluids or food that may adhere to the sensor and prevent detection of air in the tubing. Use a cotton swab to clean the pathways of the cassette receptacle. (Refer to the pump Operator's Manual for further cleaning instructions).
- Use of INFOCAP-E in the Infinity® pump may result in frequent alarms and/or inaccurate fluid delivery.
- Device contains small parts. Dispose of or store connector cap when not in use to avoid choking hazard for children under 3 years.
- Avoid leaving tubing and cords where infants, children, or those deemed at high risk for medical line entanglement, can become entangled. Be aware that if these items become wrapped around a patient's neck, there is an increased risk of strangulation or death. Caregivers of patients who are at risk for entanglement should discuss with their health care provider how to properly manage their lines and properly monitor patients based on their needs while in use.

WARNING: This product can expose you to chemicals including vinyl chloride, which is known to the State of California to cause cancer. For more information go to www.P65Warnings.ca.gov.

THE INFINITY DELIVERY SET IS INTENDED TO CONNECT TO ENTERAL FEEDING TUBES

© 2022 Zevey, Inc. ENFit® is a registered trademark of GEDSA. Infinity® and Infinity Orange® are registered trademarks of Zevey, Inc. Moog® is a registered trademark of Moog, Inc. For patent information, visit www.moogmedical.com/patents. Assembled in Costa Rica. Manufactured by Zevey, Inc. LS-83181 Rev. B

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