CURLIN Infusion Pump Administration Set

For use with Curlin Ambulatory Infusion Pumps.

NOTE:

- Do not use if package is opened or damaged as sterility may be compromised.
- Aseptic technique required for all fluid pathway connections. If alcohol is used to disinfect junctions, allow for evaporation time prior to connecting.
- Patients and caregivers must be instructed by a qualified clinician before use.
- To prevent occurrences of false Air In Line Alarms, bring the solution to room temperature prior to infusion.
- Prior to the start of any infusion, refer to the Curlin pump "User's Manual" for any applicable precautionary instructions.

WARNING: Avoid leaving tubing and cords where infants, children, or those deemed at high risk for medical line entanglement, can become entangled. Be aware that if these items become wrapped around a patient's neck, there is an increased risk of strangulation or death. Caregivers of patients who are at risk for entanglement should discuss with their health care provider how to properly manage their lines and properly monitor patients based on their needs while in use.

CAUTION: Do not prime while the tubing is connected to the patient's access device. Doing so could result in overdose and may cause injury.

Recommended administration set usage not to exceed 72 hours

Directions for use:

By gravity:

- Twist and remove breakaway tab from "Flow-Stop"(See Step 1 on page 2). If your set has a pre-filled medication reservoir, move to item 2 below. Spike the IV bag or luer lock to reservoir and invert to

remove air. Note: If a Check Valve Adapter is provided, connect it to the distal end of the set.

- Squeeze "Flow-Stop" between thumb and forefinger to allow fluid to flow through tubing until all air is removed from the reservoir and administration set. To prime a filtered set, hold the filter in a vertical position with the flow downwards to allow the filter to completely fill with fluid. Release the "Flow-Stop" to automatically clamp the tubing and manually close slide clamp.
- Insert the administration set into the Curlin pump. (See Steps 2 and 3 on page 2.)
- Close the pump door securely. Proceed to items 5 - 10 below.

By using the pump "Prime" key:

- Remove the breakaway tab from the "Flow-Stop." (See Step 1 on page 2.)
- Insert the administration set into the Curlin pump. (See Steps 2 and 3 on page 2.)
- Close the pump door securely.
- Follow the pump "User's Manual" or "Patient Information Guide" instructions for priming with the pump. Proceed to items 5 - 10 below.

By gravity and using the pump "Prime" key:

- Attach the set to the IV access device according to your healthcare provider's policies.
- Open all clamps and begin the infusion.
- To pause the infusion refer to the owner's manual.
- When the infusion is complete, stop the pump and close the slide clamp prior to opening the pump door.
- Disconnect the set from the IV access site according to your healthcare provider's policies.
- Remove the set from the pump and dispose of it properly.

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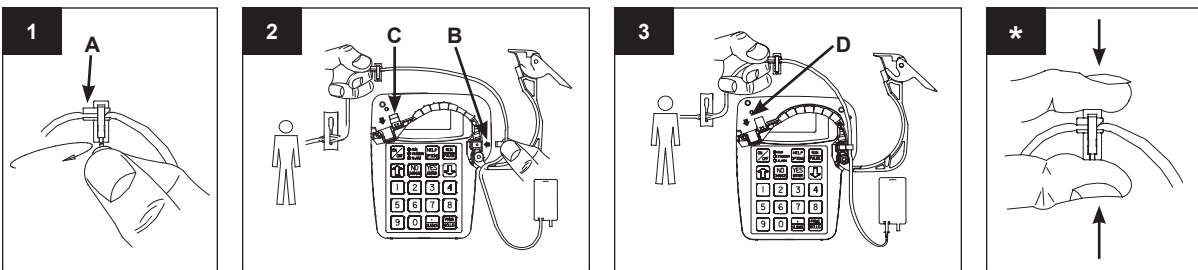
USA English Page 2 of 2

- First, twist and remove breakaway tab from the "Flow-Stop".
- Second, insert blue "Tubing Guide" on right side of pump. The blue arrow (B) on the background surface of pump head indicates direction and location for the blue "Tubing Guide."

CAUTION: Be sure the tubing is in front of the door as shown and that the "Tubing Guide" is inserted all the way into its receptacle before closing the pump door.

- Third, insert amber-colored "Flow-Stop" into the cavity on the left side of the pump so that the tubing lies in the middle of the pumping fingers. Then, push the tubing slightly into the black Air Detector slot. (C) The

yellow arrow (D) on the background surface of the pump head indicates the direction and location for the amber "Flow-Stop".

CAUTION: Hold the "Flow-Stop" by the small handle (A) as shown above. **DO NOT PRESS** down on the top of the "Flow-Stop" when inserting it into the receptacle. This action could inadvertently open the "Flow-Stop" and allow fluid to flow through the tubing.

Note: To allow fluid to flow through the tubing after the breakaway tab has been removed, squeeze the "Flow-Stop" between the thumb and forefinger. (*)

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