

PRIMING THE INFINITY DELIVERY SET

UNDERSTANDING THE DIFFERENT METHODS AVAILABLE TO PRIME INFINITY ENTERAL DELIVERY SETS

PRIMING OPTIONS

There are several different methods for priming enteral delivery sets for use with Infinity pumps - each with different benefits. This guide will help you understand these differences, and in what scenario each method can be used.

This guide is not intended to replace the operator's manual. For detailed instructions for use, refer to the operator's manuals by visiting the links provided:



Infinity
moogmedical.com/
infinity-manual




Infinity Orange
moogmedical.com/
orange-manual




USING THE HOLD TO PRIME FEATURE


This option is used for easy priming of the Infinity set if it will be hung on an IV pole. It requires holding your finger on the button throughout the priming process, allowing you to stop priming whenever needed.

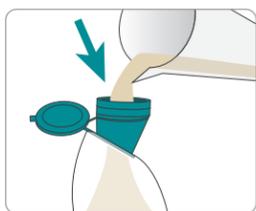


1. Fill the delivery set with formula and hang it on an IV pole. Attach the pump to the IV pole.

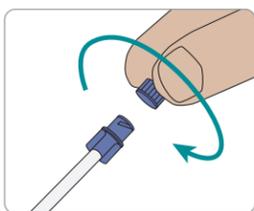


2. Remove the protective cover from the delivery set, and make sure that the delivery set is not connected to the patient.

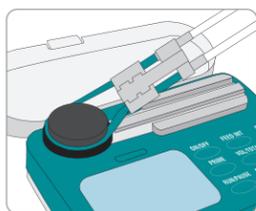

3. Press the ON/OFF button for 1.5 seconds to turn the pump on. Seat the cassette in the pump and close the pump door.

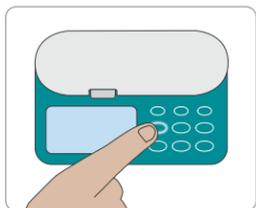


4. Press and hold the PRIME button. The alarm will sound once and the pump will prime the set for as long as the button is pressed. The display will read "TO STOP LET GO."

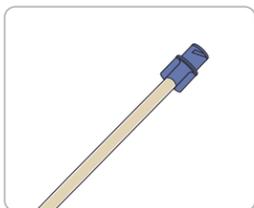


5. Once air is removed from the entire downstream tubing, release the PRIME button. The delivery set is now fully primed.

USING THE AUTO PRIME FEATURE


Auto prime is the easiest priming method as it allows you to prime the set with a touch of a button. This method is best used when the pump and set will be hung on an IV pole for delivery.

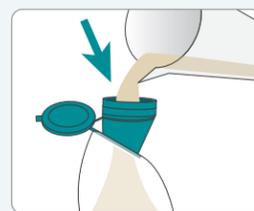

1. Fill the delivery set with formula and hang it on an IV pole. Attach the pump to the IV pole.

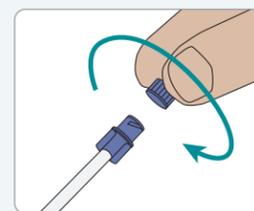


2. Remove the protective cover from the delivery set, and make sure that the delivery set is not connected to the patient.

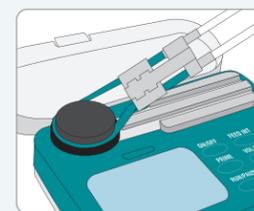


3. Press the ON/OFF button for 1.5 seconds to turn the pump on. Seat the cassette in the pump and close the pump door.



4. Press and hold the PRIME button for 1.5 seconds. Alarm will sound once and the pump will begin to prime the set. When auto prime is finished, the pump will stop and an alarm will sound.

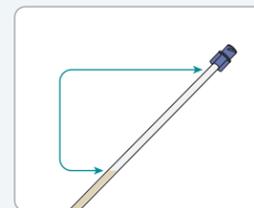

5. Remove any remaining air in the tubing by pressing and holding the PRIME button again for 1.5 seconds. Alarm will sound once and pump will re-initiate auto prime.

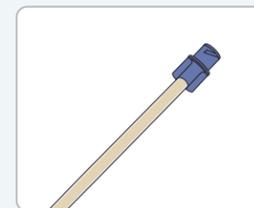


6. When the formula reaches the end of the tubing, press the PRIME button again to stop the pump. All air should now be removed from the entire downstream tubing, and the delivery set is fully primed.


NOTE: To use the auto prime feature, your pump must have the latest software installed.

MANUAL PRIMING FOR IV POLE USE


Manually priming the delivery set is the fastest way to prepare the set for use. Manual priming can be used when the pump and set will be hung on an IV pole for use.

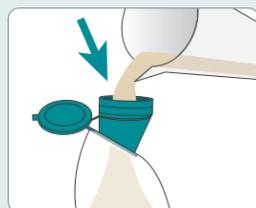


1. Fill the delivery set with formula and hang it on an IV pole.

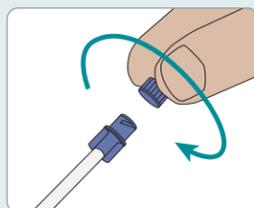


2. Remove the protective cover from the delivery set, and make sure that the delivery set is not connected to the patient.

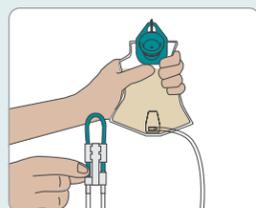


3. Grasp the bag and in one hand and the delivery set cassette in the other hand.

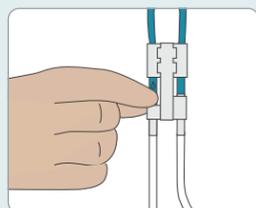

4. Gently pinch the colored tubing below the "O" symbol. Hold this position until air is removed from the tubing. Gently squeeze the bag at the same time to assist fluid flow.

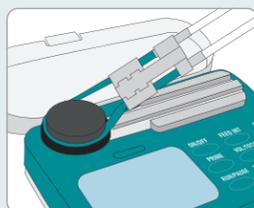


5. Seat the cassette in the pump and close the pump door.

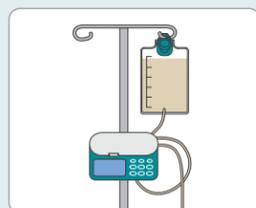


6. The delivery set is now fully primed. Both the pump and delivery set can be hung on an IV pole.

MANUAL PRIMING FOR BACKPACK USE


Using the pump and delivery set with a backpack requires the removal of all air in the delivery set bag and tubing. This can easily be achieved using the manual prime method.

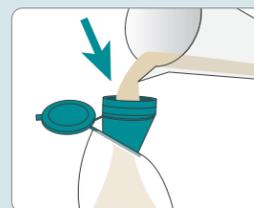


1. Fill the delivery set with formula.

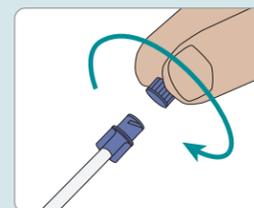


2. Remove the protective cover from the delivery set, and make sure that the delivery set is not connected to the patient.

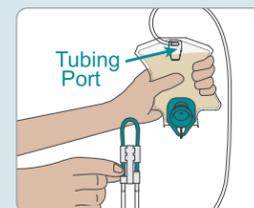

3. Turn the bag upside down so that the tubing port is at the top. Grasp the bag with one hand while holding the delivery set cassette in the other hand.

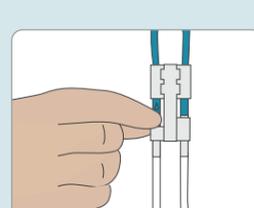

4. Gently pinch the colored tubing below the "O" symbol. Hold this position until air is removed from the tubing. Gently squeeze the bag at the same time to assist fluid flow.

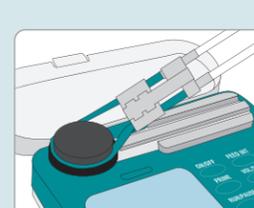

5. Seat the cassette in the pump and close the pump door.



6. The delivery set is now fully primed. Both the pump and delivery set can be placed into a backpack for use.