

Infinity[∞]® Enteral Feeding Pump BY MOOG[®]



From hospital care to anywhere.
The feeding pump for living.



Accurate and easy nutrition delivery anytime, anywhere.

When patients are able to be mobile they can live their lives with fewer restrictions and better feeding schedule compliance – *the Infinity Pump helps make this happen.*



Small & lightweight: Weighing less than 1 pound, the Infinity Pump is portable for extended use, which minimizes restrictions on a patient's lifestyle.



Portable: Designed to easily remove all the air from the bag and tubing, *allowing 100% mobile operation without air-in-line alarms.* Infinity pumps are the only feeding pumps with this capability.



Accurate: The Infinity Pump maintains $\pm 5\%$ accuracy at any flow rate and *when operated in any orientation*, even upside down!



Convenient: The Lithium Ion battery provides 24 hours of continuous on-the-go battery operation. A standard wall outlet is used to charge the pump between uses.

"I am beyond grateful for all the positive changes my Infinity pump has brought! It has stabilized my health in a way I desperately needed and provided the nutrition I need to live my best life."

- Infinity Patient



An everyday pump with an extraordinary impact.

The Infinity pump is trusted by clinicians to meet the relentless demands of daily use by patients of all ages.



With a new auto prime feature, Infinity pumps are more convenient and user friendly than ever!



Proven: Infinity pumps have been successfully used day in and day out for over 15 years by both pediatric and adult tube fed patients.



Easy-to-use: The simple and intuitive interface can be easily mastered by patients of all ages, including kids!



Safe: Infinity delivery sets are not manufactured with DEHP, BPA, or natural rubber latex and include an integrated in-line occluder for automatic free flow protection.



Rugged: Durable construction with consistent performance under a variety of conditions; from hospital to home.



Easy cleaning: Infinity pumps can be easily cleaned by rinsing under a stream of warm water and standard dish soap.



Maintenance free: Infinity pumps do not require any regularly scheduled maintenance or calibration.





The company behind the pump can make all the difference.



Focus: Our sole focus is on providing the best pumps, delivery sets, accessories, and support to our customers.



Our team: Our sales and clinical teams consist of industry experts who specialize exclusively in enteral products.




Collaboration: We work closely with your organization to find tailored support and education solutions that meet your specific needs.



Trust: We believe that earning your trust is crucial, and that's why we prioritize timely and transparent communication in every situation.



Our promise to you: We are committed to providing unrivaled support at all points of contact and transparency in all our communications. This is who we are, and this is what we do.

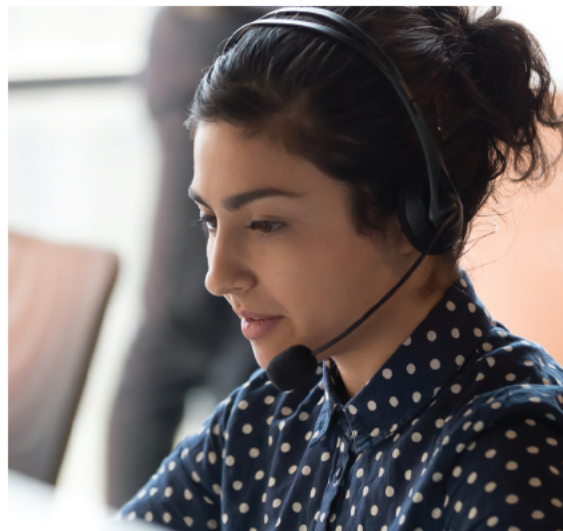

The kind of clinical support that other manufacturers can only talk about.

That's a bold statement. But we invite you to try it out. Pick up your phone any time and call us at **800.970.2337**. We are committed to being there for you and your patients whenever it is needed; **24 hours per day, 7 days per week**.



Live stream video: Connect with our clinical team in real-time directly from your own location via live stream video sessions.



e-Learning: Learn pump programming and troubleshooting techniques at your convenience with self-paced online learning tools.



On-site education: For larger groups, our clinical team can work with you to organize and develop in-person training at your location.



Online resources: Visit **moogmedical.com** for a broad selection of training videos, resource library, virtual training.



CE courses: Moog offers a variety of courses and training workshops on various topics which award CE credits upon completion.



Call us at any time with any question. We're here for you.
800.970.2337

Infinity Pump Specifications

Size:	4.05" H x 5.65" W x 1.95" D (10.3 cm H x 14.4 cm W x 4.9 cm D)
Weight:	14.4 oz (408 g)
Flow Rate Range:	0.1 – 600 mL/hr
Flow Rate Accuracy:	± 5%
Flow Rate Increment:	0.1 mL/hr from 0.1 – 10 mL/hr; 1 mL/hr from 10 – 600 mL/hr
Dose Range:	0.1 mL – 3000 mL, or infinite dose
Dose Increment:	0.1 mL from 0.1 mL - 10mL; 1 mL from 10 – 3000 mL
Priming Volume:	~15.0 mL
Residual Volume:	~11.5 mL
Battery:	Lithium ion; 24 hours at 125 mL/hr; 6 hour recharge time
Operating Orientation:	Any
Maintenance:	No annual maintenance or calibration required
Cleaning:	Washable under running water (water resistant)
Warranty:	2 years

Ordering Information

Order No.	Units / Box	Description
INFKIT2	1	Infinity Enteral Feeding Pump Kit - includes Infinity Pump, Compact Wall Charger, Multi-Position Pole Clamp, and operator's manual
INF0020-E	30	Infinity Spike Set
INF0500-E	30	Infinity 500 mL Bag Set
INF1200-E	30	Infinity 1200 mL Bag Set
23401-001	1	Compact Wall Charger
11981-001	1	Multi-Position Pole Clamp
PCK1001	1	Mini Backpack – Red
PCK1002	1	Mini Backpack – Light green and gray
PCK1003	1	Mini Backpack – Black and gray
PCK2001	1	Super-Mini Backpack – Black and gray
PCK2002	1	Super-Mini Backpack – Light green and gray
PCK3001	1	Waist Pack – Black and gray
PCK4001	1	Backpack – Black and gray

Note: Infinity Orange Delivery Sets cannot be used with the Infinity Enteral Feeding Pump



1) Infinity delivery sets, 2) Multi-Position Pole Clamp, 3) Compact Wall Charger, 4) Carry packs



Infinity delivery sets are not manufactured with DEHP, BPA, or natural rubber latex.

Every set features the ENFit® connector system. Learn more about ENFit connectors by scanning the code below with a smartphone, or visit moogmedical.com/enfit.





Speak live with a Moog Clinical Representative for pump questions and troubleshooting guidance 24 hours per day, 7 days per week.

Clinical & Customer Support
800.970.2337

Visit moogmedical.com/infinity for additional information about Infinity enteral feeding pumps.



Tel: **801.264.1001**
Toll Free: **800.970.2337**



4314 Zevex Park Lane
Salt Lake City, UT 84123 USA



moogmedical.com/infinity
medcustomerservice@moog.com



Infinity® is a registered trademark of Zevex, Inc. Moog® is a registered trademark of Moog, Inc. ENFit® is a registered trademark of GEDSA. © 2023 Moog, Inc. Moog Medical, 4314 Zevex Park Lane, Salt Lake City, UT 84123 USA. MD-41258-001 Rev. E

MOOG®

Enhancing Healthcare.
Enriching Lives.™